Bathroom Safety Rail

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Multi-use at sink or as toilet safety frame

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⚠ WARNING: Read **ALL** warnings and instructions on enclosed insert prior to operating device.

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Save these instructions.

Please refer to warning insert for additional information.

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Carex Health Brands

INDICATIONS FOR USE

This device is intended to provide stability assistance for end users with moderate stability or balance issues.

CAUTION! Read ALL instructions and warnings prior to assembly and use. Failure to use this device as intended may increase the risk of injury to the user or damage the device.

- If you do not understand these instructions, contact a healthcare professional before attempting to use.
- Inspect all parts for shipping damages before use. If there is shipping damage – **DO NOT USE**. Contact Compass Health Brands.

WARNINGS

- The device is to be used as a safety frame only. It is not designed to support the entire weight of the individual.
- Periodically check frame screws to make sure rail is securely attached. Tighten if necessary.
- This safety rail has a weight limit of 300 lbs. (136kg). **ALWAYS** observe the weight limit on the labeling of your device.
- **ALWAYS INSPECT SAFETY RAIL PRIOR TO USE.** Properly inspecting and maintaining your device can reduce the risk of injury. Make sure that all parts are secure and are in good working order.
- **ALWAYS** check the fasteners for the tightness prior. Make sure that if device is not in good working order or fasteners cannot be tightened, **DISCONTINUE USE OF THE DEVICE**.
- **DO NOT** use this device if you are unable to evenly balance your weight between the two handles.
- **DO NOT** use the device if you have a history of dizziness, balance problems, or a need for a device that can support heavy leaning. Consult a healthcare provider to determine if this device is right for you.
- **ALWAYS** make sure the device is fully opened and the folding legs are locked in position before use.
- This is not a toy. **DO NOT** let children play on/with safety rail.
- **BE AWARE** of your surroundings when operating the safety rail. Look for hazards and avoid them. Avoid surfaces with water, sand, gravel, dirt, leaves and other debris. Wet, slick, uneven and/or rough surfaces may impair traction and contribute to possible accidents. Loose cords and unsecured rugs may move suddenly and cause a loss of balance. **DO NOT** use this device in mud, ice, or puddles.
- **NEVER** use near steps, sloped driveway/hills with a steep incline (never greater than 15 degrees). **DO NOT** walk backwards down a slope.

- **DO NOT** use stairs or escalators while using this safety rail. **DO NOT** use near swimming pool areas.
- **ALWAYS** take additional care and move slowly when moving from carpeted to hard surface floors or cracks/separation in walkway surfaces; avoid sharp bumps, drainage grates, and sudden surface changes.
- **DO NOT** use while under the influence of mind-altering drugs, alcohol or while suffering from dizziness.
- **DO NOT** store in direct sunlight or outdoors. Exposure to extreme temperatures (heat, cold, moisture, etc.) may make surfaces (seat, hand grips, etc.) hot to the touch, weak or brittle.
- Ensure hand grips and skin contact surfaces are cool to the touch before use.
- **DO NOT** use this device as a “walker” – it is for standing and bathroom safety support **ONLY**.
- **FAILURE TO USE COMMON SENSE AND HEED THE ABOVE WARNINGS FURTHER INCREASES RISK OF SERIOUS INJURY. USE AT YOUR OWN RISK AND WITH APPROPRIATE AND SERIOUS ATTENTION TO SAFE OPERATION. USE CAUTION.**

CLEANING AND MAINTENANCE

- **NEVER** use organic solvents, abrasive cleaners or scouring pads on any part of the Bathroom Safety Rail.
- Use a mild soap and water solution, rinse with clean water, then dry with a soft cloth. Ensure that the Bathroom Safety Rail is well rinsed before using.
- Check fit and tightness of all wing nuts, hex bolts and E-clips **WEEKLY** to ensure that the Bathroom Safety Rail is stable and safe to use.

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(4) Hex bolts / Pernos
hexagonales

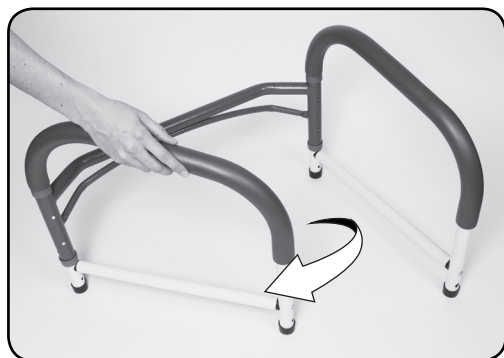
(4) E-clips / Perillas
de las alas

Note: Actual items may vary from what is shown.

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2 Assembly Ensamblaje

Remove all packaging material, unfold and open top frame.

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
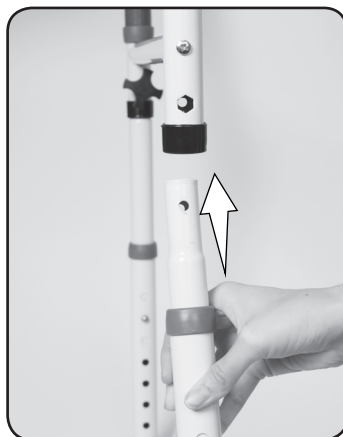
Secure frame by inserting all E-clips.

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Assemble legs.

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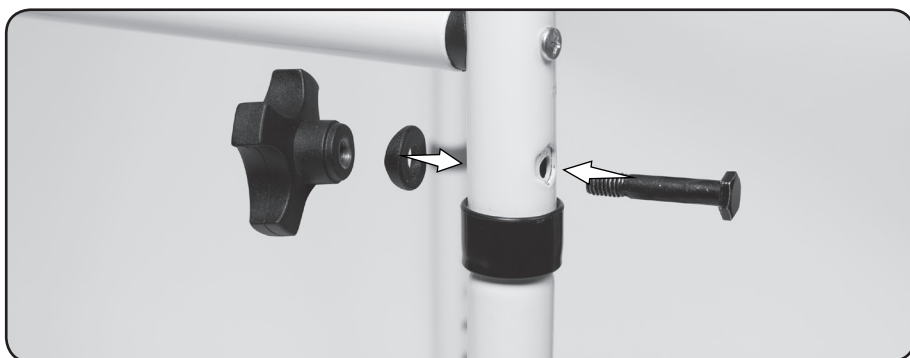Secure legs using hex bolt, plastic washer and tighten with black wing nut. Punch out will hold hex bolt in place as you firmly tighten wing nut.

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Ensure all bolts are fixed in place and wing nuts have been firmly tightened before each use.

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3 Adjusting the height Ajuste de la altura

⚠ WARNING: Make sure detent buttons are securely locked into place.

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Adjust leg extensions to appropriate height using detent buttons.

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Set bathroom safety rail around toilet (as shown) or use for balance at sink. Ensure floor is dry when using.

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42-B36900_04 © 2019
Manufactured for / Fabricado por: Compass Health Brands Corp.