

CAREX[®]

ITEM# P56100 0000

Easy-Up Bed Rail



Questions, problems or missing parts?

Before returning, contact Compass Health Brands customer service:


800-526-8051 | CAREX.COM

Product Assembly

1 Check all parts

Part	Quantity
A	1
B	2
C	1
D	2
E	1
F	1
G	1
H	2
I	1

Weight Capacity: 300 lbs

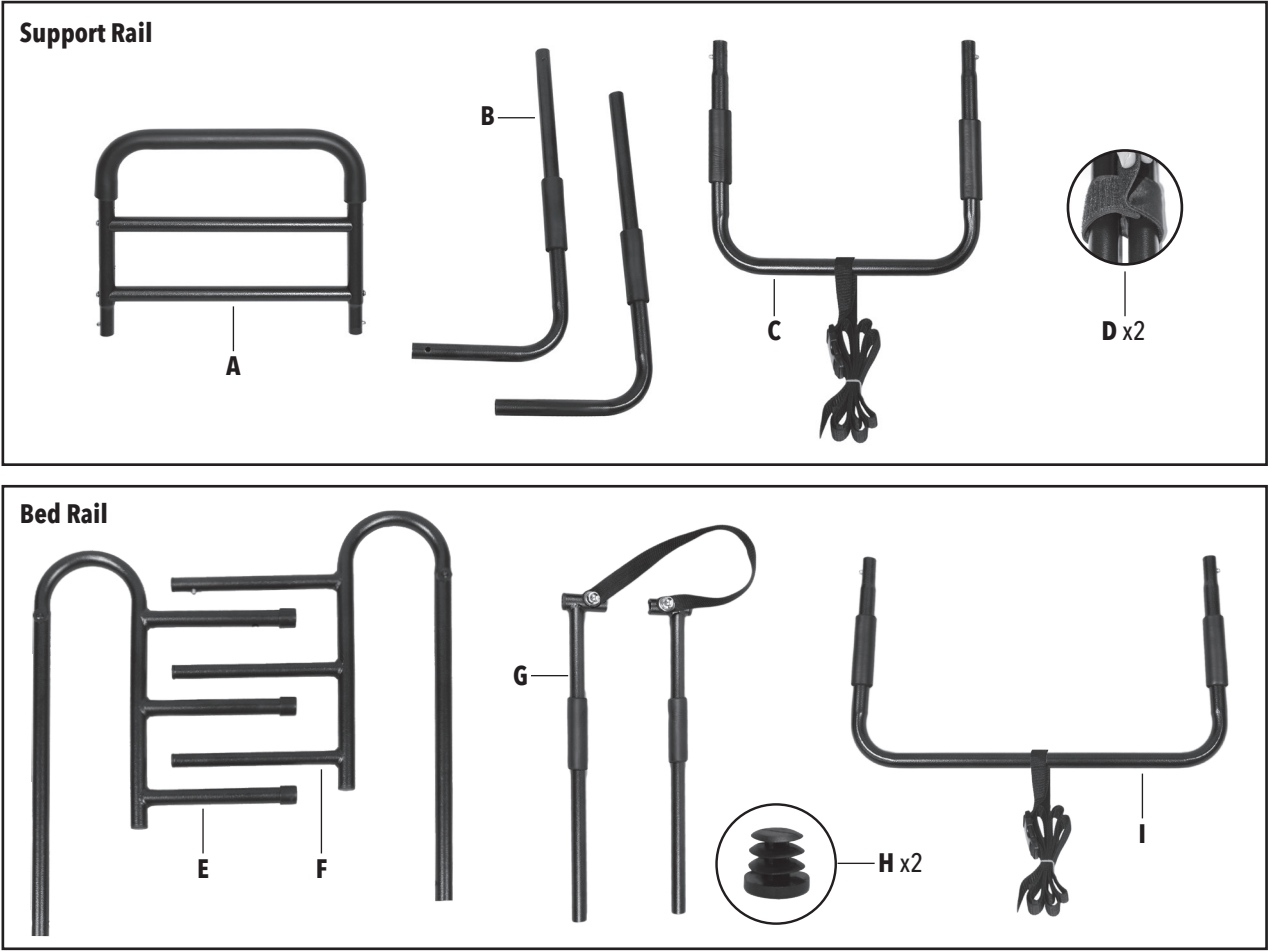

⚠ WARNING

To avoid serious injury, ensure that the snap buttons are securely in place.


YES

CLICK


NO

2 SUPPORT RAIL: Connect

- Depress button and align to the adjustment hole.
- Repeat for the other side.

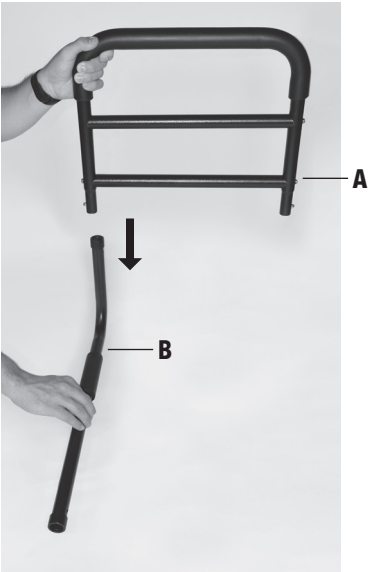

3 Connect

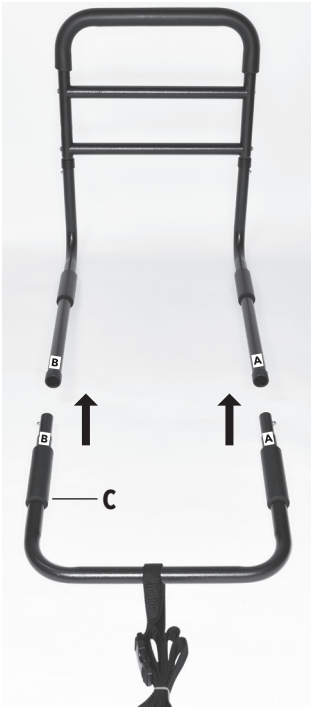


Support Rail assembled



Product Assembly

4 BED RAIL: Connect

- Depress button and align to the adjustment hole.

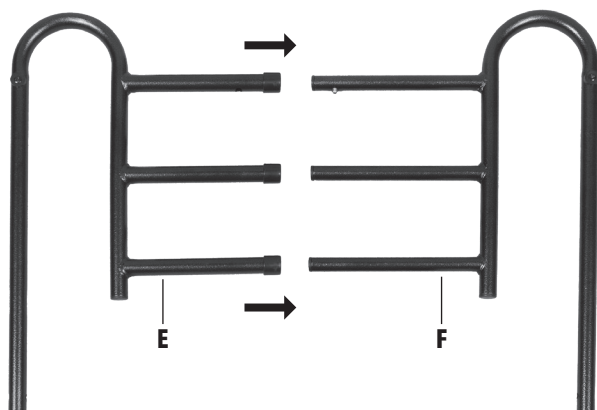


5 Connect


- Depress button and align to the adjustment hole.
- Repeat for the other side.



6 Connect

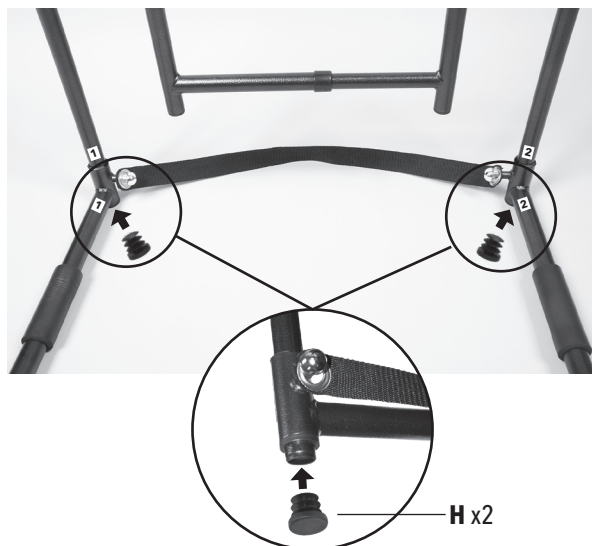

7 Connect

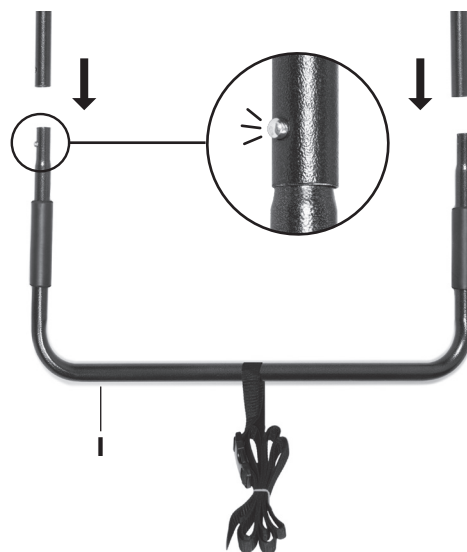

Bed Rail assembled



Product Installation and Adjustment

1 Installation to the bed


⚠ WARNING

SUFFOCATION/STRANGULATION/ENTRAPMENT HAZARD

When adjusting the foam handle height, the top of the handle must be a minimum of 4" above the top of the mattress.



2 Secure straps to bed frame and tighten

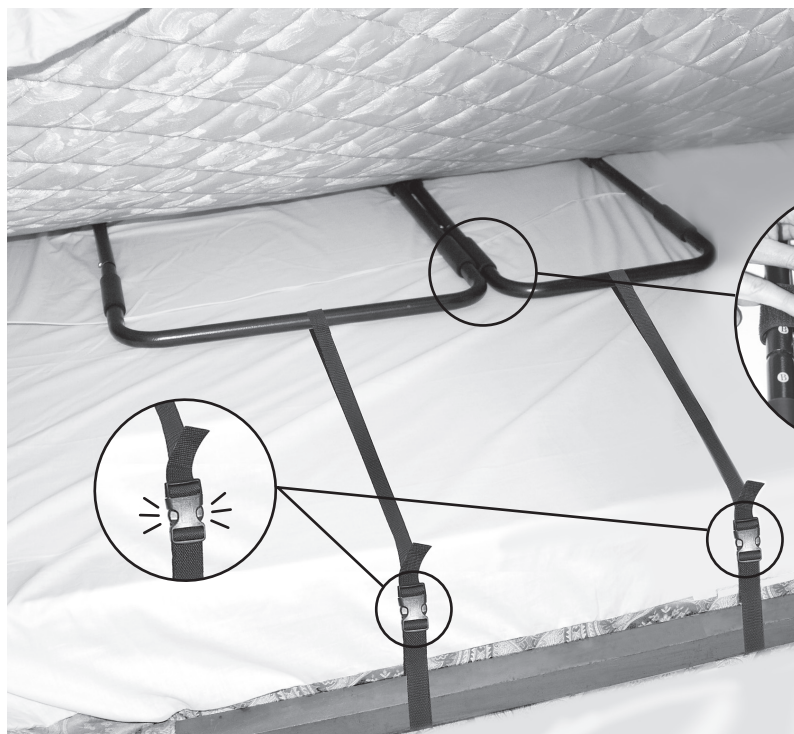

⚠ CAUTION

Secure bars together with straps provided as shown.



D x2

✗ Incorrect positioning



⚠ WARNING

SUFFOCATION/STRANGULATION/ENTRAPMENT HAZARD

If product is installed incorrectly or moves from its initial position, gaps can occur which can entrap and kill. People with Alzheimer's disease, dementia or other neurological conditions, or those who are sedated, confused, or frail, are at increased risk of entrapment, suffocation and strangulation.

- **NEVER** use unless product is tight against mattress, without gaps, and at least 12 1/2 in. from headboard and footboard.
- **NEVER** use with children.
- **NEVER** use on toddler, bunk, water, or inflatable beds, or on beds with mattress toppers or soft compressible pads.
- **NEVER** use with a mattress height taller than 16 in. or shorter than 10 in.
- **NOT** to be used with articulating beds.



To lower (with foot):

1. Hold top of bed rail as shown. **DO NOT** put limbs through the rails.
2. Minding your knee, press down on the strap with your foot and gently lower rail.



To lower (with hand):

1. Hold top of bed rail as shown. **DO NOT** put limbs through the rails.
2. While holding top of bedrail, pull strap towards you and gently lower rail.



To raise:

1. Hold top of bedrail as shown.
2. Slowly lift until both pins snap into holes.
3. Test to assure the bedrail is secure.

⚠ CAUTION

To avoid injury, assure that your extremities and those of others are away from the bed rail bars when raising or lowering.

⚠ WARNING

ENTRAPMENT/STRANGULATION/SUFFOCATION AND FALL HAZARDS

Gaps in and around this product can entrap and kill. People with Alzheimer's disease or dementia, or those who are sedated, confused, or frail, and are at increased risk of entrapment and strangulation. People attempting to climb over this product are at increased risk of injury or death from falls. Always make sure this product is properly secured to bed. If product can move away from bed or mattress, it can lead to entrapment and death.

Cleaning and Maintenance Instructions

- ❑ Use warm damp cloth to clean Bed Rail. **ALWAYS** ensure product is completely dry before using.
- ❑ **ALWAYS INSPECT BED RAIL PRIOR TO USE.** Properly inspecting and maintaining your Bed Rail can reduce risk of injury. Make sure all parts are secure and are in good working order.

⚠ WARNING

SUFFOCATION/STRANGULATION/ENTRAPMENT HAZARD

If product is installed incorrectly or moves from its initial position, gaps can occur which can entrap and kill. People with Alzheimer’s disease, dementia or other neurological conditions, or those who are sedated, confused, or frail, are at increased risk of entrapment, suffocation and strangulation.

⚠ WARNING

SUFFOCATION/STRANGULATION/ENTRAPMENT HAZARD

If the strap provided is not properly secured, the product may move into an unsafe position which increases the danger of entrapment. See instructions for proper use of the straps.

⚠ WARNING


ENTRAPMENT HAZARD

NEVER use product without properly securing it to bed. Incorrect installation can allow product to move away from mattress, bed frame and/or head or foot boards, which can lead to entrapment and death.

- Bedside rail and straps work with most twin, queen and king size beds.
- Stop using immediately if damaged or broken, or if parts are missing.
- Stop using immediately if product shifts out of its original position until it is readjusted into the correct position.
- In addition to contacting the manufacturer directly, consumers should report problems to the CPSC at its website [SaferProducts.gov](https://www.saferproducts.gov) or call 1-800-638-2772, or to the FDA at 1-800-332-1088.
- For further information, see: [cpsc.gov/en/Safety-Education/Neighborhood-Safety-Network/Posters/Adult-Bed-Rails/](https://www.cpsc.gov/en/Safety-Education/Neighborhood-Safety-Network/Posters/Adult-Bed-Rails/) and www.fda.gov/bedsafety.




Zone 1

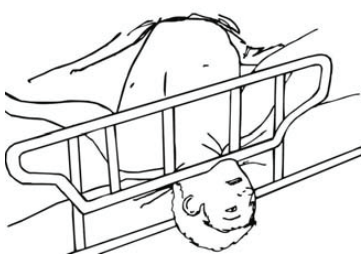

Zone 2



Zone 3

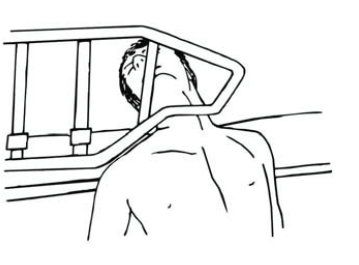

Zone 4

CAREX®

Questions, problems or missing parts?
Before returning, contact Compass Health Brands customer service:

800-526-8051 | CAREX.COM

Retain this manual for future use.

COMPASSHEALTH

Manufactured for:
Compass Health Brands Corp.
Middleburg Heights, OH 44130
www.compasshealthbrands.com
Phone 800.376.7263
Made in Taiwan
42-P56100_0000_00 © 2021