

CAREX[®]

ITEM# E815C0 0000

Stand-Alone Adjustable Toilet Safety Frame



Please read all instructions before assembly or use.

Questions, problems or missing parts?

Before returning, contact Compass Health Brands customer service:

800-526-8051 | CAREX.COM

WARNINGS

INDICATIONS FOR USE:

This device is intended to provide stability assistance for end users with moderate stability or balance issues when sitting down and rising from a commode or toilet seat.

This device is intended for a single user **ONLY**.

CONTRAINDICATIONS:

For safety reasons, people with balance disorders should **NOT** remain without supervision when using this device. User **MUST** be able to maintain even weight distribution while lowering and rising from commode and while seated on the commode.

DO NOT use the device if you have a history of dizziness, balance problems or a need for a device that can support heavy leaning. Consult a healthcare provider to determine if this device is right for you.

This device is **NOT** intended to be used as a walking aid. If the need for a walking aid is required, a Walker or Rolling Walker (Rollator) should be considered. This device is to be used as a temporary stationary stability aid **ONLY** at the commode or the front of a sink.

SAVE THESE INSTRUCTIONS. This sheet MUST accompany the device with each use.

CAUTIONS:

Read ALL instructions and warnings prior to assembly and use. Failure to use this device as intended may increase the risk of injury to the user or damage to the device.

- This device is intended for a single user **ONLY**. In the event this device is passed to a new user, Compass Health Brands assumes **NO** responsibility for any damage or injury resulting from use of the device.
- Inspect all parts for shipping damages before assembly. If there is shipping damage – **DO NOT USE**. Contact your dealer/carrier for further instructions.
- If you do not understand these instructions, contact a healthcare professional, dealer or technical professional before attempting to install or use device.
- Ensure the floor is dry when using.

 **CAUTION:** Grip the handrail securely and push directly down, not outward, on the rails when raising or lowering yourself onto the seat.

 **CAUTION:** Please inspect all parts carefully for cracks or breaks before use.

 **CAUTION:** **DO NOT** use on bathmats. Feet **MUST** be flat on the ground at all times.

WARNINGS:

- **DO NOT ATTEMPT TO ADJUST OR OPERATE THE TOILET SAFETY FRAME WITHOUT READING ALL INSTRUCTIONS CAREFULLY.**
- Users that are unable to understand or follow these instructions and warnings or have limited physical capabilities should be supervised or assisted while using this device.
- **DO NOT** use device if it is **NOT** securely assembled and/or installed.
- This toilet safety frame has a weight limit of 300 lbs. (136kg), evenly distributed. **ALWAYS** observe the weight limit on the labeling of this device as serious injury may result.
- The device is to be used as a bathroom safety frame **ONLY**. It is not designed to support the entire weight of the individual or devices that attach to the commode directly.
- **ALWAYS INSPECT SAFETY RAIL PRIOR TO USE.** Properly inspecting and maintaining your device can reduce the risk of injury. Make sure that all parts are secure and are in good working order.
- **ALWAYS** check frame screws and fasteners for tightness prior to use. Make sure that if device is not in good working order or fasteners cannot be tightened, **DISCONTINUE USE OF THE DEVICE.**
- **DO NOT** use this device if you are unable to evenly balance your weight between the two handles.
- **ALWAYS** make sure the device is fully opened and the folding legs are locked in position before use (design dependent).
- This is not a toy. **DO NOT** let children play on or with safety rail.
- **DO NOT** stand, kneel or climb on the toilet safety frame.
- **BE AWARE** of your surroundings like loose cords or unsecure rugs or other hazards when operating the safety rail as this can cause a loss of balance. Look for hazards and avoid them. This device is to be used indoors **ONLY**.
- **DO NOT** use while under the influence of mind-altering drugs, alcohol or while suffering from dizziness.
- **DO NOT** use this device as a "walker" – it is for standing and bathroom safety support **ONLY**.
- **DO NOT** use the toilet safety frame if wobbly, unstable, or not level. **ALWAYS** be sure that the frame is secure and stable before using. Failure to comply with all product warnings may result in serious injury.
- This product type is designed for use with the most common types of commodes. It will not fit all commode types. Be sure that the unit has been installed correctly, fits properly and is stable before use as this may cause damage to the device or injury to the user. (design dependent).
- User **MUST** maintain even weight distribution while lowering and rising from device and while seated. Not doing so can result in tipping of the device and falls/serious injuries.
- The support rails are for providing minimal balance assistance when getting on and off the seat. **DO NOT** attempt to use the support rails to support body weight during transfer, as this may result in failure of the unit and serious injury (design dependent).
- The armrests are intended to be used on both sides evenly. If pressure is applied on one side only, use only half of the body weight. At overload, there is a risk that the armrest will break.
- Ensure that the toilet safety frame sits level and is secure before each use and that all legs are adjusted to equal height and that push buttons are fully engaged. This ensures an even height adjustment and that the adjustable legs are securely locked in position (design dependent).
- **DO NOT** disassemble, repair or modify any component.

FAILURE TO USE COMMON SENSE AND HEED THE ABOVE WARNINGS FURTHER INCREASES RISK OF SERIOUS INJURY. USE AT YOUR OWN RISK AND WITH APPROPRIATE AND SERIOUS ATTENTION TO SAFE OPERATION. USE CAUTION.


Assembly / Operating Instructions

1 Remove Packaging and Check Parts

- ❑ Remove the toilet safety frame components from all packaging including tape, cardboard and plastic.
- ❑ To avoid risk of suffocation to children or animals, dispose of the plastic bags correctly.
- If you **DO NOT** understand the manual, contact your dealer for a complete explanation of the instructions, cautions and warnings. Otherwise injury may result.

⚠ WARNING: Ensure all bolts and hardware have been firmly tightened before each use.

- **DO NOT** use excessive force when installing any parts.
- If any parts are missing or damaged, **DO NOT** attempt to use.

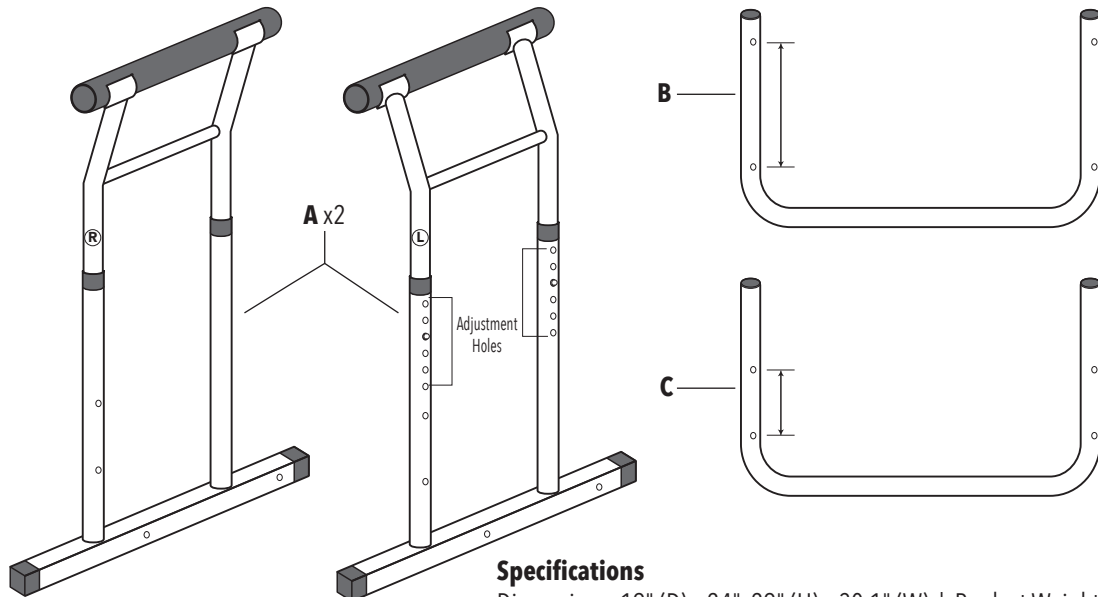


	Description	QTY
A	Side Frame	2
B	Base U-Frame	1
C	Vertical U-Frame	1
D	Screw Knob	8
E	Magazine Rack	1

Specifications
Dimensions: 19" (D) x 24"-29" (H) x 30.1" (W) | Product Weight: 11 lbs. | Weight Capacity: 300 lbs.
Width between hand grips: 22.8"

2 Attach Base U-Frame to base of Side Frame

- ❑ Place Base U-Frame (B) on a table top. Hold Side Frame (A) upright and secure to Base U-Frame with Screw Knobs (D). Repeat for other side.

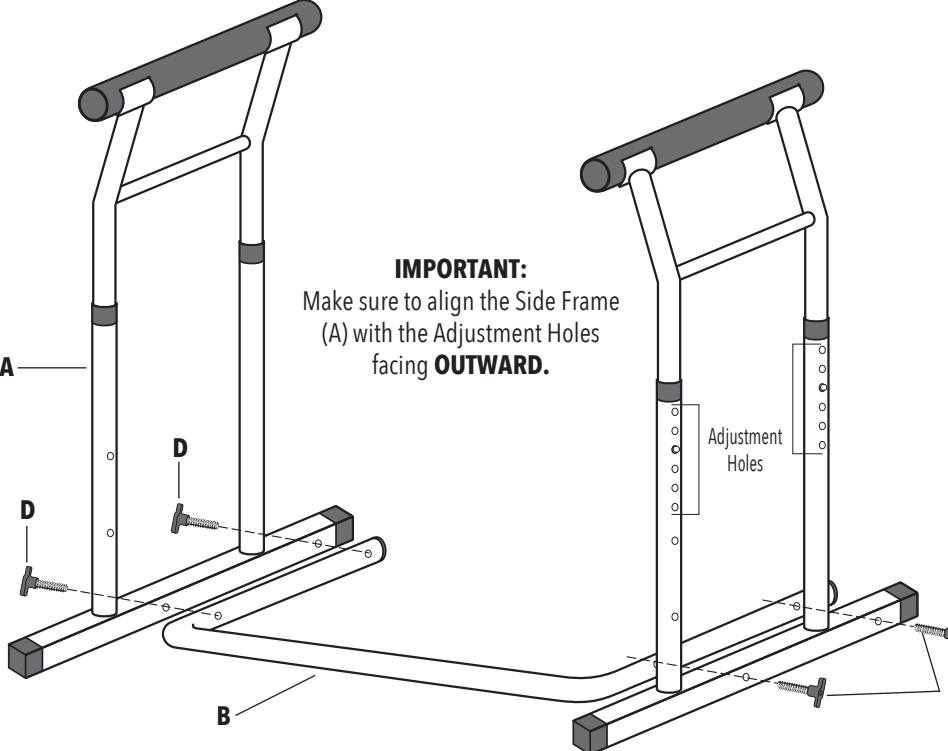

IMPORTANT:
Make sure to align the Side Frame (A) with the Adjustment Holes facing **OUTWARD**.


3 Attach Vertical U-Frame to vertical pole of Side Frame

- Hold the Vertical U-Frame (C) upright and secure to Side Frame (A) with 2 Screw Knobs (D). Repeat for other side.




CAUTION:

Hold arm rests firmly with both hands. Do not lean on any one side of the arm. This may cause the product to topple.

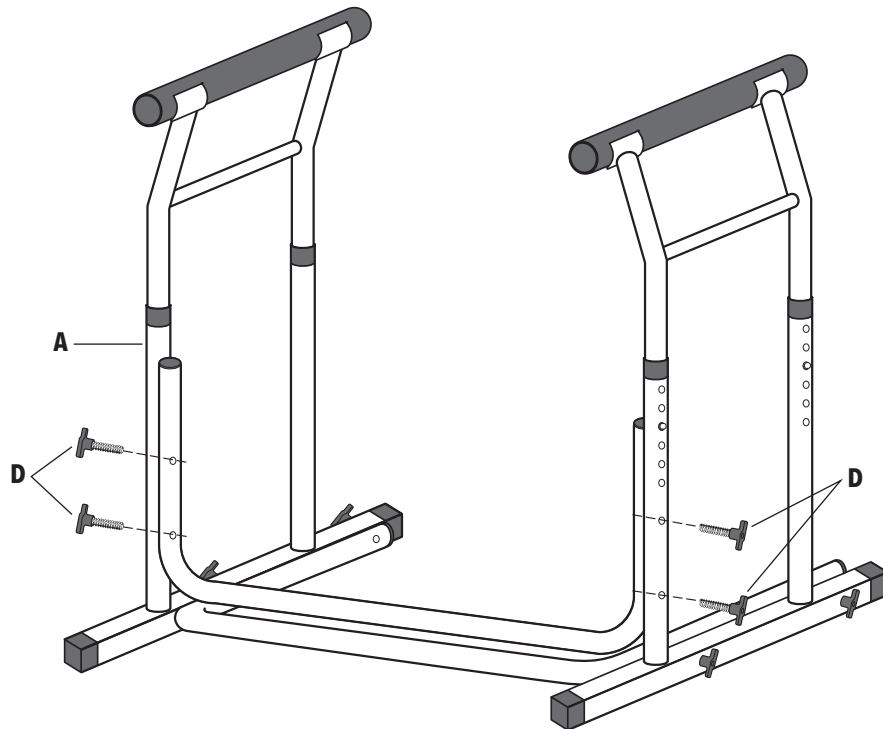

4 Hang Magazine Rack on Side Arm

- Hang the Magazine Rack (E) on the horizontal bar just below the handle. Fits on either Side Arm (A).

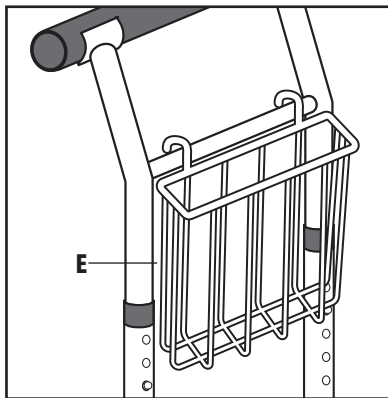


5 Height adjustment

- To adjust the height of the toilet safety frame, depress push pins on Side Frame (A) and move arms to desired height.
- Ensure pins are locked in place and an audible "click" is heard.



CAUTION:

Periodically check to ensure all Screw Knobs (D) are tightened as loosening may occur.

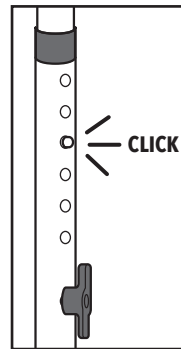

6 Position Toilet Safety Frame



WARNING: To avoid tripping, please pay attention to your surroundings when lowering or rising from a seated position.

7 Cleaning and Maintenance

- Check fit and tightness of all wing nuts, bolts, knobs and E-clips **WEEKLY** to ensure that the Bathroom Safety Rail is stable and safe to use.
- **NEVER** use organic solvents, abrasive cleaners or scouring pads on any part of toilet safety frame.
- Use a mild soap and water solution, rinse with clean water, then dry with a soft cloth.

CAREX®

Questions, problems or missing parts?
Before returning, contact Compass Health Brands customer service:

800-526-8051 | CAREX.COM

Retain this manual for future use.

COMPASSHEALTH

Manufactured for: Compass Health Brands Corp.
6753 Engle Rd., Middleburg Heights, OH 44130
www.compasshealthbrands.com
800.376.7263 • Made in China
42-FGE815C0 0000_04 © 2024