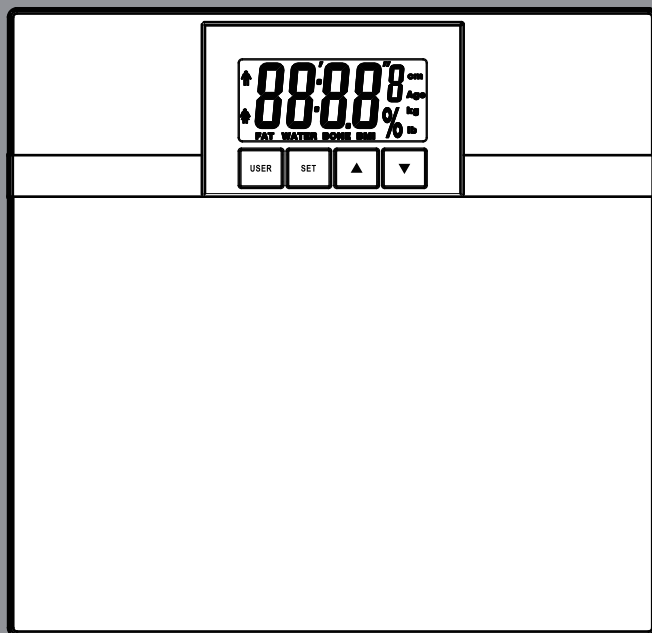
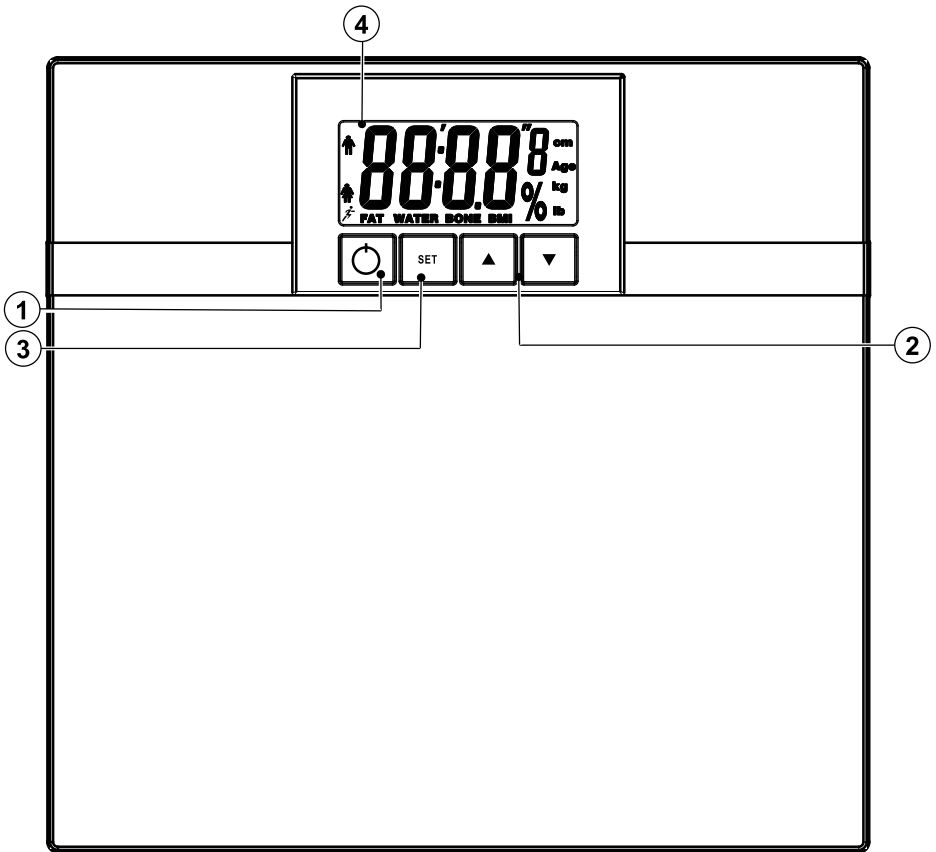
BK60W
BK60B
BK60S

English (Original Instructions)

1. On / Off Button
2. Up / Down selection buttons
3. Set Button
4. Display

Intended use

Your Black & Decker Scale has been designed for weighing the human body.

This product is intended for household use only.

Read this entire manual carefully before using the appliance.

Safety instructions

- Warning! When using battery-powered appliances, basic safety precautions, including the following, should always be followed to reduce the risk of fire, leaking batteries, personal injury and material damage.
- Read this entire manual carefully before using the appliance.
- The intended use is described in this manual. The use of any accessory or attachment or the performance of any operation with this appliance other than those recommended in this instruction manual may present a risk of material damage.
- Retain this manual for future reference.

Using your appliance

- Do not place the appliance on or near heat sources.
- Do not use the appliance near flammable materials.
- Do not use the appliance outdoors.
- Always protect the appliance from water or excessive humidity.
- Do not overload the appliance.
- Do not use the appliance at ambient temperatures below 10 °C or above 30 °C.
- Do not place the scale on an uneven surface.
- Do not immerse the scale in liquids.
- Do not allow the scale to get wet.
- Do not step on the edge of the scale.
- Do not step on the scale with wet feet.

- Ensure scale is dry before stepping onto it.

Safety of others

- This appliance is not intended for use by persons (including children) with reduced physical, sensory or mental capabilities, or lack of experience and knowledge, unless they have been given supervision or instruction concerning use of the appliance by a person responsible for their safety.
- Children should be supervised to ensure that they do not play with the appliance.
- Do not allow children or any person unfamiliar with these instructions to use the appliance.

After use

- Remove the batteries when the appliance is not going to be used for a longer period of time.
- When not in use, the appliance should be stored in a dry place. Children should not have access to stored appliances.

Inspection and repairs

- Before use, check the appliance for damaged or defective parts. Check for breakage of parts and any other conditions that may affect its operation.
- Do not use the appliance if any part is damaged or defective.
- Have any damaged or defective parts repaired or replaced by an authorised repair agent.
- Never attempt to remove or replace any parts other than those specified in this manual.

Electrical safety

To avoid the risk of short circuiting, do not immerse the appliance or the batteries in water or other liquid.

English (Original Instructions)

- Use only the correct type of battery for the appliance (see technical data)
- Never attempt to charge non-rechargeable batteries.
- The electronic components inside the appliance may be affected by other devices used in close proximity, such as mobile phones or microwave ovens. If this happens, the display will show erroneous data. To remedy this, either switch the interfering device off, or move the appliance away from it.

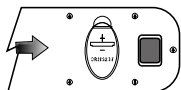
Additional safety instructions for non-rechargeable batteries.

- Never attempt to open for any reason.
- Do not store in locations where the temperature may exceed 40 °C.
- When disposing of batteries, follow the instructions given in the section "Protecting the environment".
- Do not incinerate batteries.
- Under extreme conditions, battery leakage may occur. When you notice liquid on the batteries, proceed as follows:
 - Carefully wipe with the liquid off with a damp cloth.
 - Avoid skin contact.
- The batteries and battery compartment should be checked periodically for any signs of corrosion.


Assembly

Fitting the batteries


- Remove the battery cover from the appliance.

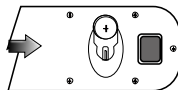- Insert 2 batteries (CR2032) into the appliance.

- Refit the battery cover.

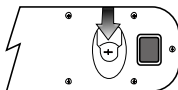Note! The appliance display will show " Lo" if there is insufficient power in the batteries. If this occurs then change the batteries.

Use

- Place the appliance on a flat surface.
- Ensure the appliance is dry before stepping on to it.

Setting personal data

The memory can store personal data for up to 12 people.

- Switch the appliance on.
- " 8888 " will appear in the display (4).
- Press the set button (3).
- " -01- " will appear blinking in the display (4).
- Select the user number by pressing the up or down button (2).
- Confirm the selection by pressing the set button (3)
- Enter the data in the menus appearing in the order as follows.

Selecting the gender

The screen will display the gender by a female ♀ or a male ♂ symbol.

- Select the gender by pressing up or down button (2).
- Store the data by pressing the set button (3).

Athlete mode

People of 15 – 50 years of age who train intensively for more than 12 hours a week and who have a resting heart rate approximately 60 beats per minute or less, have these measurements performed more accurately in athlete mode.

- Select between athlete mode  and normal mode by pressing up or down button (2).
- Store the data by pressing the set button (3).

Setting the age

The screen will show the data in years of age.

- Select the age by pressing up or down button (2).
- Store the data by pressing the set button (3).

Setting the height

The screen will show the data in centimeters.

- Select the height by pressing up or down button (2).
- Store the data by pressing the set button (3).

Selecting the measuring system

The appliance can show the results in both metric and the imperial system.

- Turn the appliance over.
- Select the required measuring system by sliding the switch into the desired position.

Kg/lb/St

Note!

- The appliance display will show “ 0.0 “ if there is no button press within 5 seconds whilst setting personal data.
- If the measuring system is set to kg then the height will display in centimeters.

- If the measuring system is set to lb or st then the height will display in Inches.

Switching on and off**Switching on**

- Press the on/off button (1). “ 8888 “ will appear in the display (4) followed by “ 0.0 “.

Switching off

- After approximately 15 seconds of inactivity the screen will go blank.

Measuring weight only

- Switch the appliance on.
- “ 8888 “ will appear in the display (4).
- Step on to the appliance when “ 0.0 “ appears in the display. (The appliance will turn off automatically if you do not step on to it within 10 seconds)
- The weight will appear on the screen for approximately 3 seconds.

Measuring weight body fat and water percentage

These measurements are an indication of your level of fitness and your physical health. Consult your physician or a qualified dietitian for appropriate advice regarding further action.

The measurements are performed using the technique of B.I.A. (Bioelectrical impedance analysis), a highly reliable method for analyzing the composition of the human body. B.I.A. measures the resistance to the flow of a small, harmless low-level electrical current using the built in electrodes. Personal data like gender, height and age, for the key factors of the analysis results processed by the software program.

When using this appliance, we recommend that you take regular measurements, always around the same time of day and under the same conditions.

English (Original Instructions)

Refer to the tables below for an indicative interpretation of the analysis results.

Note!

- This appliance only works if you are bare foot.
- Excessive drinking or eating, exercising, medical conditions, use of medication, the female menstrual cycle etc., will affect the measurement results.

Warning! This appliance must not be used by pregnant women and people with a cardiac pacemaker or any other medical device implanted in the body.

- Switch the appliance on.
- “8888” will appear in the display (4).
- Select the program number of your personal data stored in the memory using the up and down arrows.
- Your personal data will now be shown in the display (4).
- Step on to the appliance when “0.0” appears in the display. (The appliance will turn off automatically if you do not step on to it within 10 seconds)
- The weight will appear on the screen for approximately 3 seconds.
- The result of the body fat percentage will appear on the screen (4).
- The result of the body water percentage will appear on the screen (4).

Body fat percentage

Age	Under fat	Normal	Over fat	Obese
Female				
10-17	<20	20-31	31-37	>37
18-39	<21	21-32	32-38	>38
40-59	<23	23-34	34-40	>40
60-79	<25	25-36	36-42	>42
Athlete		12-25		

Age	Under fat	Normal	Over fat	Obese
Male				
10-17	<8	8-19	19-25	>25
18-39	<9	9-20	20-26	>26
40-59	<11	11-22	22-28	>28
60-79	<13	13-24	24-30	>30
Athlete		3-15		

Body water percentage

	Female	Male
Adult	38-75	36-75

Error indicator

The screen will display an error warning “Err” if an error during measurement has occurred.

Cleaning and maintenance

Before cleaning and maintenance, remove the plug from the socket and allow the appliance to cool.

Warning! Before cleaning and maintenance, switch the appliance off.

Warning! Do not immerse the appliance, the cord set or the plug in water or any other liquid.

- Wipe the product exterior with a damp cloth.
- Do not use scouring pads.
- If necessary, use a mild detergent to remove spots.

- Do not use any abrasive or solvent based cleaner.
- Do not immerse the appliance in water or any other liquid.

Technical data

Voltage	6V _{DC}
Batteries	2 X CR2032
Max weight capacity	180kg

Protecting the environment

Separate collection. This product must not be disposed of with normal household waste.

Should you find one day that your Black & Decker product needs replacement, or if it is of no further use to you, do not dispose of it with household waste. Make this product available for separate collection.

- Black & Decker provides a facility for recycling Black & Decker products once they have reached the end of their working life. This service is provided free of charge. To take advantage of this service, please return your product to any authorised repair agent who will collect them on our behalf.
- You can check the location of your nearest authorised repair agent by contacting your local Black & Decker office at the address indicated in this manual. Alternatively, a list of authorized Black & Decker repair agents and full details of our after-sales service and contacts are available on the Internet at: www.2helpU.com.

Guarantee

Black & Decker is confident of the quality of its products and offers an outstanding guarantee. This guarantee statement is in addition to and in no way prejudices your statutory rights. The guarantee is valid within the territories of the Member States of the European Union and the European Free Trade Area.

If a Black & Decker product becomes defective due to faulty materials, workmanship or lack of conformity, within 24 months from the date of purchase, Black & Decker guarantees to replace defective parts, repair products subjected to fair wear and tear or replace such products to ensure minimum inconvenience to the customer unless:

- The product has been used for trade, professional or hire purposes;
- The product has been subjected to misuse or neglect;
- The product has sustained damage through foreign objects, substances or accidents;
- Repairs have been attempted by persons other than authorised repair agents or Black & Decker service staff.

To claim on the guarantee, you will need to submit proof of purchase to the seller or an authorised repair agent. You can check the location of your nearest authorised repair agent by contacting your local Black & Decker office at the address indicated in this manual. Alternatively, a list of authorised Black & Decker repair agents and full details of our after-sales service and contacts are available on the Internet at: www.2helpU.com.

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