
EN Nipple Shields with Sterilizer Case

Size 1 — up to 24 mm | Size 2 — 25 mm and up


Cleaning: Always use nipple shields under the care of a medical professional or lactation consultant. Before the first use and after every use, wash nipple shields with warm soapy water, rinse clean, sterilize, and air dry.

Sterilizer Instructions: Rest the nipple shields on top of the holders inside the case. Add 5 teaspoons / 1 oz / 25 mL of water to the bottom reservoir. Close case securely and microwave for 3 minutes using a 700-1000 watt microwave. Use caution when removing the case from the microwave, **it will be hot!** Drain water from the case and open lid to allow nipple shields to dry and cool for at least 5 minutes. Once cool, wipe out any remaining water drops to ensure that case is dry before closing lid to store.

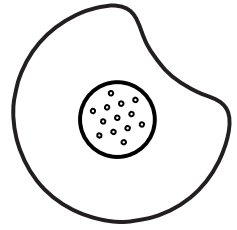
Instructions for Use:


- ① With clean hands, hold the nipple shield in both hands. The shiny side should be facing your skin and the matte side facing out.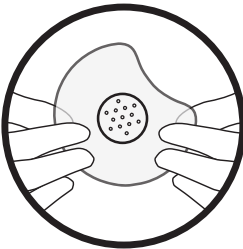


- ② Position the cut out where baby's nose will be.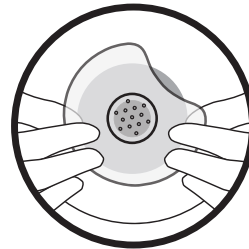


- ③ Gently stretch the shield over your breast, ensuring your nipple fully fits.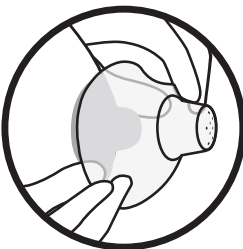

- ④ If needed, a small amount of nipple cream may be first applied to your skin to help with shield adhesion. Do not apply to your nipple.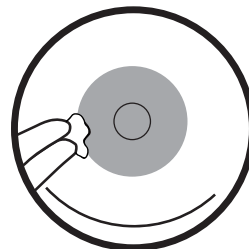

- ⑤ Smooth the sides down for full adhesion.

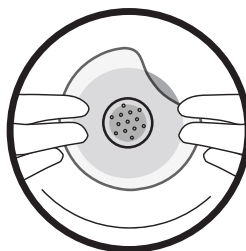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