EN Pacidose™


①

Fill the syringe with the correct amount of medicine. Check the amount again.


②

Attach Pacidose™ to the syringe and offer it to your child.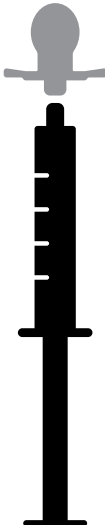

③

As your child begins to suck, slowly press in the plunger and dispense in small increments.

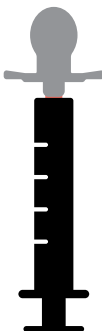Helpful Tips

- Let your child swallow small amounts before giving more.
- Give medication when your child is hungry.
- Before using Pacidose™ with medicine, dip into milk, formula, or other liquid and let your child taste it.
- Give other liquids through Pacidose™ so your child trusts it.
- Pacidose™ is compatible with most other syringes.

WARNING!

Do not tie Pacidose™ around a child’s neck as it presents a strangulation danger. Pacidose™ is not a pacifier or a toy and is not intended for such use. Never leave a child unattended while using Pacidose™. Discard if you see any sign of damage or weakness. Clean before each use. Pacidose™ is safe for use in dishwasher (top rack only), electric and microwave sterilizers, or boiling water. Syringe hand wash only. Squeeze out any trapped water from bulb.

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