EN Dr. Brown's® Specialty Feeding System

Includes the Infant-Paced Feeding Valve

Dr. Brown's® Specialty Feeding System was designed in collaboration with medical professionals seeking an oral feeding solution for infants identified with complex feeding challenges such as those often observed in infants with cleft lip and palate, ankyloglossia, high-arched palate, oro-neuromotor dysfunction, common or rare syndromic sequences, and/or craniofacial anomalies.

The Specialty Feeding System assists infants who are unable to generate adequate suction to retrieve fluid from a bottle. This bottle system allows the infant to retrieve fluid by compressing the nipple. The Infant-Paced Feeding Valve (blue one-way valve) can be inserted into any Dr. Brown's nipple to create a "compression" nipple.

Benefits of the Specialty Feeding System:

- Consistent and reliable nipple flow rates
- Infant directed feeding vs. manually expressed boluses by feeder
- Requires minimal assembly
- Helps decrease gassiness/air intake and burping with non-squeeze method
- Preserves Vitamins A, C, and E in breast milk
- Easy to clean and safe for dishwasher and sterilizer. Refer to bottle instructions for more information on bottle cleaning and use.
- The full bottle system and nipples are BPA free and do not contain natural rubber latex.
- Exhibits a typical feeding bottle design

NOTE: With the Infant-Paced Feeding Valve (blue one-way valve) in place, the nipple will fill as infant elevates tongue to the underside of the nipple during feeding. No manual priming of the nipple is necessary.

Technical Notes:


NOTE: The Specialty Feeding System WILL NOT FUNCTION as intended without the INSERT, RESERVOIR, and INFANT-PACED FEEDING VALVE.

- The Specialty Feeding System is composed of an Options+ bottle system with the internal vent system, a silicone nipple, and the Infant-Paced Feeding Valve.
- If the reservoir remains over 1/3 full when the bottle is placed upright, check that the nipple/teat hole is clear and that the insert and reservoir are firmly pressed together. The reservoir bulb can be emptied by momentarily loosening the nipple/teat collar. Be sure to re-tighten before feeding.
- If leaking occurs, the collar may have been screwed on to the bottle too tightly or too loosely. Unscrew the collar from the bottle and re-screw the collar back onto the bottle to ensure optimal functioning.
- If leaking occurs, the insert and reservoir may not be properly assembled. Be sure the insert and reservoir are connected securely.
- If you are having difficulty using the Specialty Feeding System, call **Dr. Brown's Customer Service at 1-800-778-9001** or contact us at: **help@drbrownsbaby.com**. For purchases made outside the United States and Canada, please contact your local distributor.

WARNING! Valves are small parts. Keep out of reach of children.

IMPORTANT – Consult your health care provider before using the Dr. Brown's® Specialty Feeding System to confirm this bottle system is recommended for your child's feeding challenge.

ASSEMBLY AND FEEDING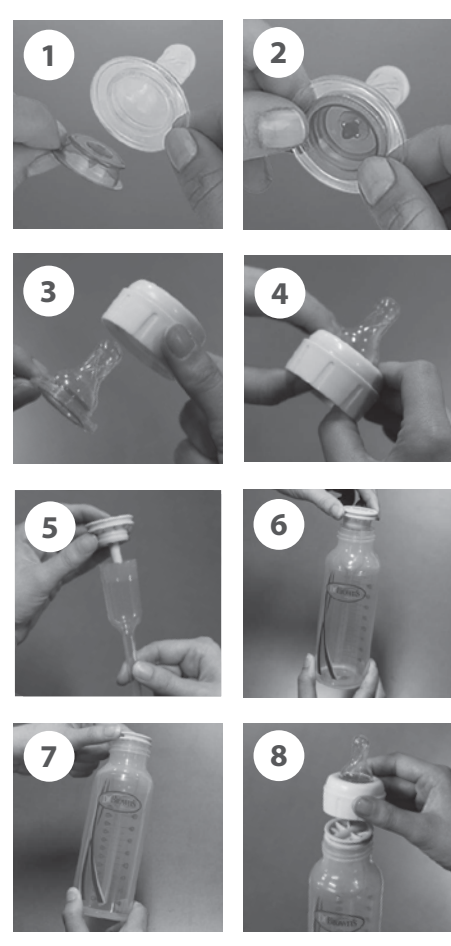

1. Insert the Infant-Paced Valve (blue one-way valve) into the nipple/teat.

For Wide-Neck version, it may be preferable to insert nipple into collar before inserting Infant-Paced Feeding Valve.

2. Make sure the valve is fully secured, flush with the nipple/teat.
3. Insert the nipple/teat into the collar.
4. Make sure the nipple/teat is fully seated in the collar.
5. Assemble the Internal Vent System by placing the insert into the reservoir to snugly fit together.
6. Place the assembled Internal Vent System into the bottle.
7. Ensure the insert makes full contact with the top of the bottle.
8. Affix the collar to the bottle and turn until closed (do not over-tighten).

If warming the bottle – do not complete steps 6-8 until after the warming is complete to prevent leakage. Use a storage/travel cap. Always test the temperature of milk before feeding baby.

Do not shake the bottle. Shaking will cause air bubbles. Formula should be stirred gently to fully dissolve lumps as lumps can clog nipple/teat holes. Fill to the desired amount.

To prevent potential leaks, do not fill the bottle above the FILL LINE WARNING.

BEFORE FIRST AND EACH USE

Before first use, fully disassemble all parts, wash the product. To prevent a possible choking hazard, pull on the nipple/teat in all directions before each use to ensure it does not tear. Inspect carefully, especially when the child develops teeth. Discontinue use and replace the nipple/teat immediately if any signs of wear or damage are visible, such as cracking or change of shape. Small pieces can accidentally be inhaled and cause serious injury.

CLEANING

CAUTION: Do not use the wire cleaning brush to clean the nipple/teat as this will cause it to tear.

Be sure to remove wire cleaning brush from bottle before cleaning. Fully disassemble all parts and wash prior to each use. All parts are safe for use in dishwasher (top rack only), electric sterilizer, microwave sterilizer, or boiling water. Do not microwave wire cleaning brush.

REMINDERS

For proper bottle and venting function, only use Dr. Brown's® nipples/teats with this bottle; other brands may leak or flow too fast or too slow. When feeding your baby, hold the bottle so that the reservoir tube hole in the bottom of the bottle is not covered by liquid (approximately at a 45° angle). Keep your baby propped up during feeding. Remove from your baby's mouth when they are not feeding.

When transporting bottles, always keep them upright and use storage/travel cap or disk to help prevent leaking. During air travel, remove internal parts and secure with travel cap or disk to prevent leaking from air pressure.


For your child's safety and health

WARNING!

- Continuous and prolonged sucking of fluids will cause tooth decay.
- Always check food temperature before feeding.
- Throw away at the first signs of damage or weakness.
- Keep components not in use out of the reach of children.
- Never attach to cords, ribbons, laces or loose parts of clothing. The child can be strangled.
- Never use feeding nipples/teats as a pacifier/soother.
- Always use this product with adult supervision.
- Glass containers may break.
- Always check the nipples/teats for signs of wear and replace when damaged.
- Do not use a microwave oven to heat the bottle as microwave ovens can create hotspots in milk/formula and/or bottle components.
- Do not leave the nipple/teat in direct sunlight, heat, or in sterilizing solution for longer than recommended as this may weaken the nipple/teat.
- Do not allow a child to walk, or run while using any bottle.
- Do not let a child take a bottle to bed or self-feed for long periods.

GLASS WARNING

Dr. Brown's glass bottles are made of borosilicate glass. Glass bottles may break. Bottles should always be inspected before each use for sharp edges, fragments, or flakes of glass that may have become dislodged and loose inside the bottle. Do not use if there are any cracks or chips on the bottle. Sudden temperature changes can cause glass bottles to crack or break. Do not freeze glass bottles. **Never allow the child to stand, walk, or run while using a glass bottle. Do not allow baby to self feed.**

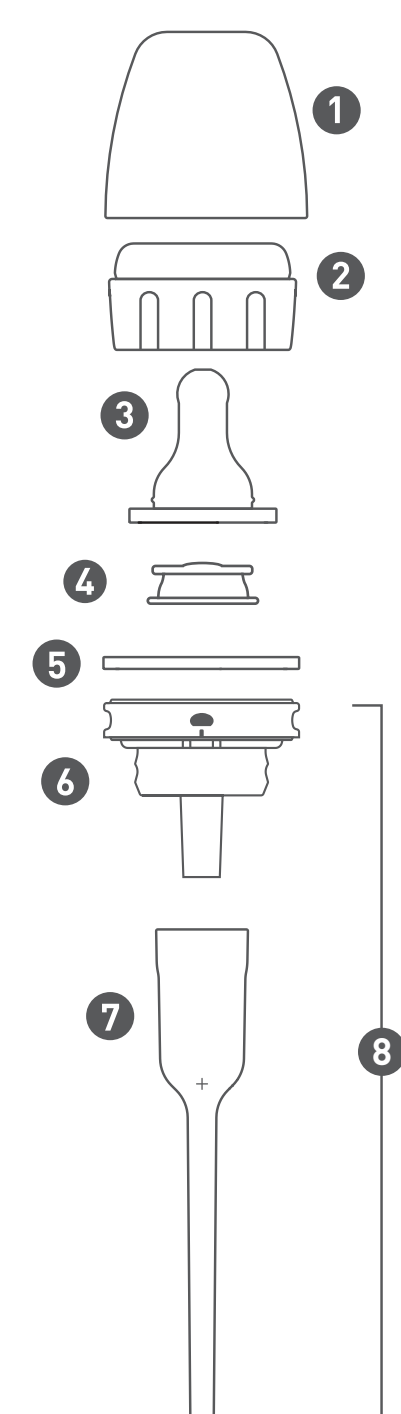Bottle Components

- 1 Cover/Cap
- 2 Collar
- 3 Nipple/Teat
- 4 Infant-Paced Feeding Valve (blue one-way valve)
- 5 Travel Disk
- 6 Insert
- 7 Reservoir Tube
- 8 **Vent Assembly:** Press firmly together to prevent leakage

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