




Quick Start Guide

eufy Breast Pump Charging Case



Product Overview

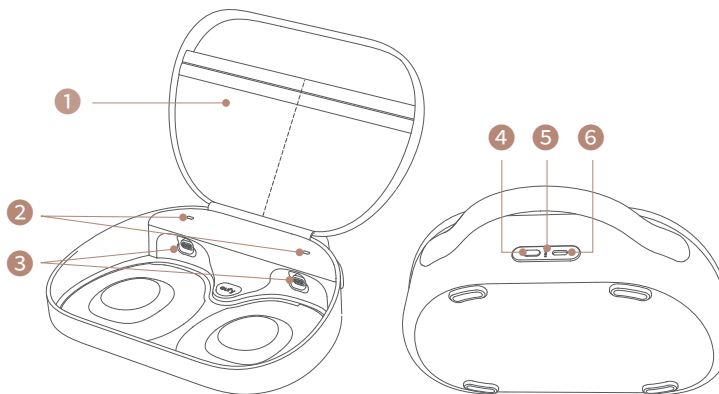

1	Storage Pockets
2	Breast Pump Charging Indicators - Flashing White: Charging the Pump Hub - Steady White and Turn Off in 10 min: Charging Complete
3	Magnetic Charging Ports (for Breast Pumps)
4	Battery Status Button Press to Show the Battery Level of the Charging Case
5	Case Battery Indicators 1 Indicator Steady White: Battery Level < 25% 2 Indicators Steady White: Battery Level 25%-50% 3 Indicators Steady White: Battery Level 50%-75% 4 Indicators Steady White: Battery Level > 75%
6	USB-C Charging Port (for Charging Case)

Charging

Charging the Case

Charge your case with a standard USB-C charging cable (included). To check the battery level of the charging case, press the case battery status button.

Fully charge the case only	3.2 h
Fully charge the case and two breast pumps inside the case	5 h



With a fully charged case and breast pumps, the battery can last up to 5 days.


Charging Breast Pumps



Before charging the breast pumps:

- Make sure all assembled components are clean and dry.
- Remove dust covers from the breast pumps.

Align the charging pins on a pump hub with the magnetic charging port inside the case. Then let go to allow the entire pump to be placed in the slot. You should hear a click from the magnetic connector and see the charging indicator light flash white. When the light stays steady (then turns off in 10 min), your pumps are fully charged. It takes 1.7 hours to fully charge the pumps with the case.

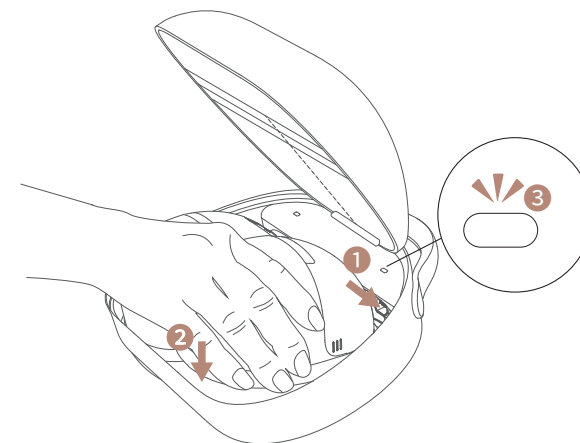


Scan to watch the tutorial video.




Scan to read the detailed User Guide.

If the washable components have not been washed or are not completely dry, you can still charge only the pump hubs in the case. Lay the case flat to ensure that the charging pins of the pumps are in close contact with the charging ports inside the case.

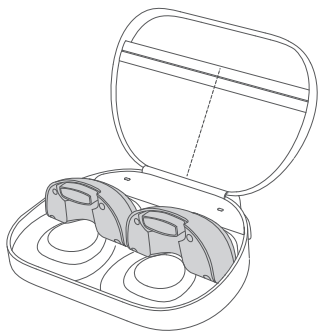

Organizing

When carrying the case with only the hubs inside, place the hubs in the storage pockets to prevent damage from collisions and friction inside the case.

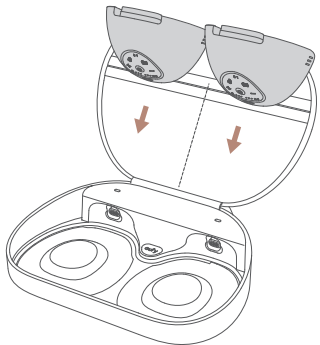

You can also place breast pump accessories, for example, the power charging cable and milk storage bags, in the storage pockets for easy access.

Cleaning

The charging case is not washable. Clean the outside and inside of the case, including the magnetic charging ports, with a damp cloth. Air dry before putting the pumps in the case.

Customer Service



12-month limited warranty



(US/CA) +1 (800) 988 7973
(UK) +44 (0) 1604 936200
(DE) +49 (0) 69 9579 7960
(中国) +86 400 0550 036
(日本) 03 4455 7823
(AU) +61 3 8331 4800
(Egypt) +20 8000000826
(UAE) +971 8000320817
(Kuwait) +965 22069086
(KSA) +966 8008500030



support@eufy.com

Anker Innovations Limited
Room 1318-19, Hollywood Plaza, 610 Nathan Road, Mongkok, Kowloon,
Hong Kong



@EufyOfficial



@EufyOfficial



@EufyOfficial