


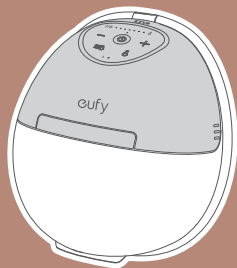

Quick Start Guide

eufy Wearable Breast Pump S1




Scan to watch the tutorial video.



Scan to read the detailed User Guide.

Breast Pump Overview

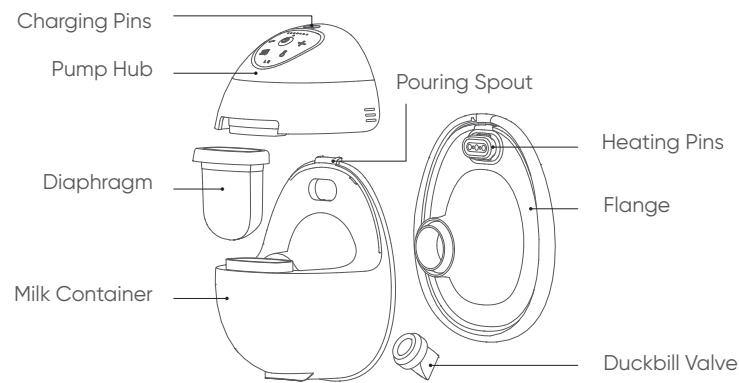

Accessories also include:



Dust Covers



Cleaning Brush



Charging Cables



Sizing Inserts (17/19/21 mm)




Nipple Measurement Card



Storage Bag


Button Controls & Light Guide

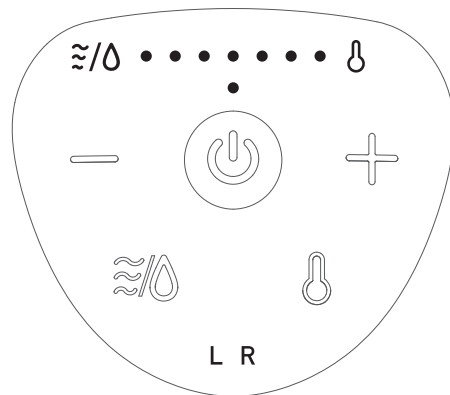

Button Controls		
	Press and Hold for 2 Seconds	Turn Breast Pump On / Off (The breast pump will turn off automatically after pumping for 30 minutes or pausing for 10 minutes.)
	Press	Pause / Continue
	Press	Switch between Stimulation Mode and Expression Mode
	Press	Turn Heating On / Off
	Press	Increase / Decrease the Suction / Heat Level (To adjust the heat level, press the heating button first.)
	Press and Hold Simultaneously for 2 Seconds	L/R Pairing in the App

Light Guide		
	Flashing Green	Charging in Progress
	Steady Green	Charging Complete
	Steady Red	Battery Level <20%
	Flashing Red	Battery Level <10%
	White	In Stimulation / Expression Mode
	Steady Orange	In Heating Mode
	Flashing Orange	Heating Exception (Reassemble the flange and pump hub to ensure the heating pins are properly seated.)
	1 to 7 Indicators White / Orange	Suction / Heat Level: 1 to 7
	L / R Indicator White	Left / Right Side Paired in the App

Disassembling

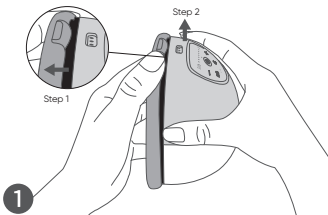


1

Step 1: Hold the pump hub with one hand and use the thumb of the other hand to pull the container edge near the spout outward to release the pump hub from the spout. (Avoid pulling the flange.)


Step 2: Lift the hub.

The locking mechanism between the pump hub and the pouring spout ensures reliable suction and heating.

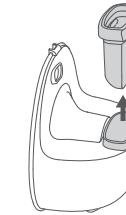


2

Remove the diaphragm from the groove.

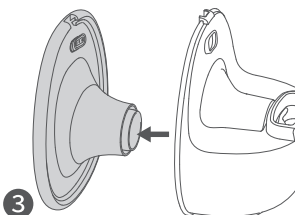

3

Unwrap the outer ring of the flange, pull the heating pins out of the oval notch, and then pull the entire flange out.

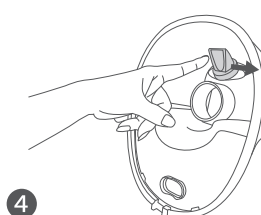

4

Flick the duckbill valve with your finger to remove it.


Cleaning and Sterilizing

- All components except the pump hub are washable.
- Do not use a microwave to sterilize.

- Before Initial Use:** Sterilize all washable parts by boiling for up to 5 minutes, stirring constantly to avoid the parts sticking to the container and melting. Alternatively, steam sterilize for a maximum of 5 minutes.
- Daily Cleaning:** Rinse all washable parts with warm, soapy water, then air dry. Sterilize if needed, using the same methods as the initial cleaning.


Assembling

1

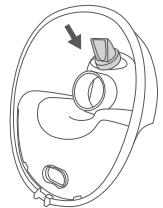


Hold the bottom of the container up, insert the duckbill valve into the small hole inside the container with your thumb and index finger and flick it to ensure it does not fall out.

2

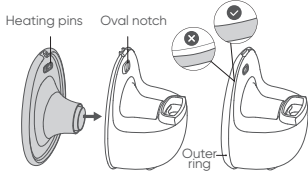


Push the flange onto the container, making sure the heating pins are firmly pushed into the oval notch. Wrap the outer ring to ensure it is properly sealed to prevent milk leakage.

3

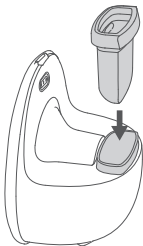

Insert the diaphragm into the groove of the container, with the concave side of the diaphragm facing the flange side.


4

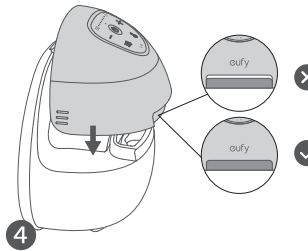


Install the pump hub on the top of the container, pressing firmly to seal the gap between the pump hub and the diaphragm.

Optional Step


To ensure a proper fit, use the Nipple Measurement Card to select an appropriate sizing insert and place the sizing insert in the flange tunnel.

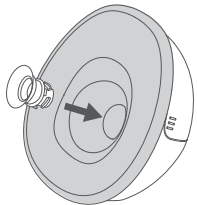

Wearing

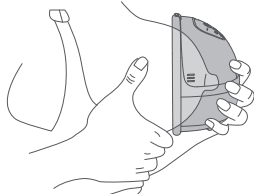

Slightly lean forward to make sure your nipple is centered in the flange tunnel. The breast pump should be firmly against your breast.

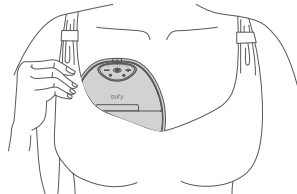

Use a full-coverage nursing bra with a flap to create a tight seal between your breasts and the flange.



Two criteria will help you determine if the breast pump is positioned correctly:


- Whether you feel a noticeable suction that can pull out milk;
- Whether it is comfortable to wear.

If the wearing position doesn't meet either of these criteria, you need to adjust the position of the pump.

It takes a bit of time for your body to adapt to a new pump, so don't be discouraged if the results aren't good at the very beginning.


Transferring Milk

1

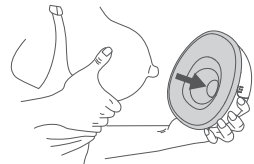

Pause or turn off the breast pump. Lean forward when taking out the pump to prevent leakage.

2

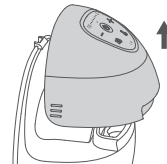


Remove the pump hub from the container. Refer to "Disassembling" step 1.




Avoid disassembling any parts (including the diaphragm) other than the pump hub before transferring milk to prevent leakage.

3

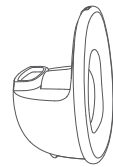

Leave the container on the table ready for transfer.

4

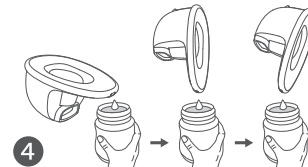

Tilt the container with the flange up to pour milk. When the pouring spout faces down, tilt further to empty completely. Lastly, remove the diaphragm if needed to collect any trapped milk under it.

Charging

Use a 2-in-1 magnetic charging cable (included) and a 5V/1A or higher adapter (not included) to charge your breast pumps. Charge it fully before initial use; it won't operate while charging.

Downloading the App

Download and install the eufy Baby app, then sign in or create an account. Follow the in-app instructions to add this device and complete the setup.



Best to Know

1. To prevent leakage:

- When you complete assembling, check again that the outer ring of the flange is perfectly sealed.
- Make sure your nipple is at the center of the flange tunnel, and use a full-coverage nursing bra to create a tight seal between your breasts and the flange.
- Do not lean back while pumping. At the end of a session, lean forward to remove the pump.

2. Can't feel suction or have zero or very little output?

- Inspect the duckbill valve and diaphragm for damage, including cracks, chips, holes, or tears. If a part is damaged, discontinue use and purchase a replacement.
- Check that the assembly is correct and that the four pieces (duckbill valve, flange tunnel, diaphragm, and pump hub) are perfectly sealed.
- Adjust the wearing position, make sure the nipple is at the center of the flange tunnel, the pump is sealed to your breast, and you can feel noticeable suction.
- Experiment with different flange or insert sizes.
- Try different suctions and rhythms.

3. To ensure proper heating:

- Press the heating pins firmly into the container oval notch.
- Seal the gap between the pump hub and the diaphragm.
- Identify the heat settings and duration that suit you best without irritating your skin.

4. Lifespan of washable parts:

The estimated life of the washable parts is 1-3 months, and actual life will vary with use and sterilization frequency. Please replace them if you feel the suction and performance are decreasing (particularly the duckbill valve), or if you have stored them for too long when planning to pump for your next child.

Customer Service



12-month limited warranty on pump hub
3-month limited warranty on washable parts



(US/CA) +1 (800) 988 7973	(AU) +61 3 8331 4800
(UK) +44 (0) 1604 936200	(Egypt) +20 8000000826
(DE) +49 (0) 69 9579 7960	(UAE) +971 8000320817
(中国) +86 400 0550 036	(Kuwait) +965 22069086
(日本) 03 4455 7823	(KSA) +966 8008500030



support@eufy.com

Anker Innovations Limited
Room 1318-19, Hollywood Plaza, 610 Nathan Road, Mongkok, Kowloon, Hong Kong



@EufyOfficial



@EufyOfficial



@EufyOfficial