


Colostrum Protector Quick Start Guide

Step by step guide on how to assemble and use the Colostrum Protector


How to assemble your Colostrum Protector

Parts & Features

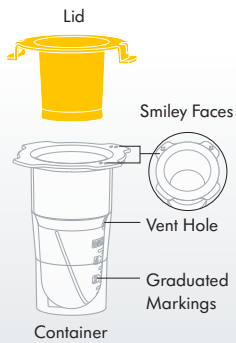


1

Attach the collector to Symphony® breast pump kit connector.

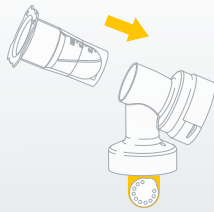

2

Ensure correct orientation* to allow for venting and prevent colostrum from spilling from the vent hole into the container below.

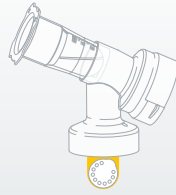


3

Attach the breast shield to the top of the Colostrum Protector.



4


Assemble the breast pump kit.

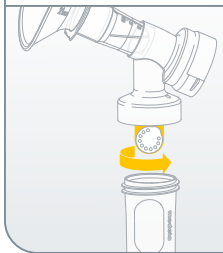


*Correct Orientation Tips:

Ensure the side with the smiley faces, vent hole, and graduated markings are facing up ↑.


NOTE:
The side with nothing on it should be facing down ↓.

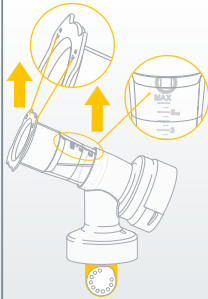


Low visibility – feel for the bumps on the back of the smiley faces.

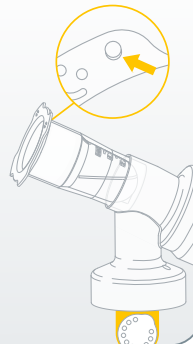

Correct Assembly

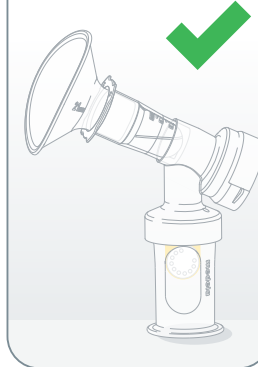

Incorrect Assembly

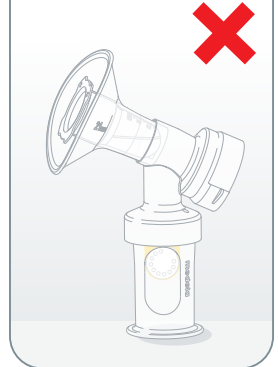

How to disassemble your Colostrum Protector

1

Remove the Colostrum Protector from the connector, leaving the breast shield attached.



2

Place on a flat surface.



3

Remove the breast shield.



4

Secure the lid.





More information

Scan the QR code for further information about your Colostrum Protector



Wash after each use

Wash in a dedicated wash basin

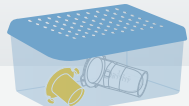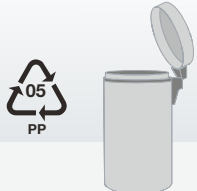1 Rinse with clear water. 	2 In a dedicated wash basin for breast pump parts, soak in warm, soapy water. • Do not place directly in the sink. 	3 Wash with a clean dishcloth or soft brush. 	4 Rinse with clear water. 	5 Place on a clean surface and/or towel and allow to air dry. 
6 Store in a clean, cool place when not in use. Do NOT store wet or damp parts. 	7 Discard or recycle after 24-hours of use according to local regulations or hospital protocol. 			

Content appearance may vary.

Refer to product instructions for use for complete instructions.