


Waterless Milk Warmer™ Quick Start Instructions

Review instruction manual for important safeguards prior to first use.

Set-up



Expand liner prior to installing in warmer.

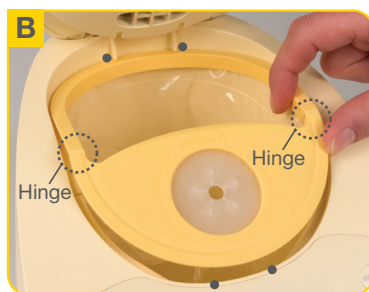

Bend insert at hinge and place into position.



Evenly spread liner within warmer cavity.



Push tray, flat-side down, into bottom of warmer.



Load feed.

NOTE: Always use insert tray to warm container or syringe.




Press power button to turn on the device.




Program


Select Mode

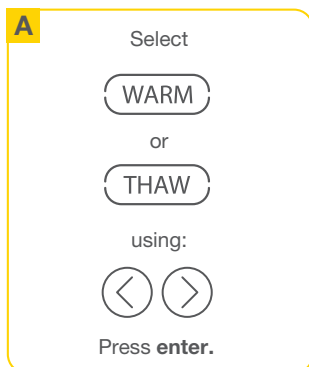


Select Starting Temperature

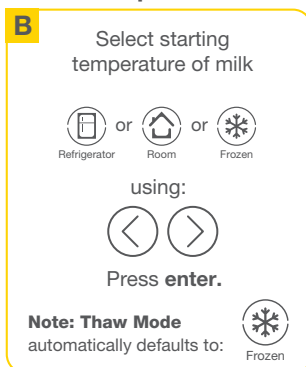

Select Volume

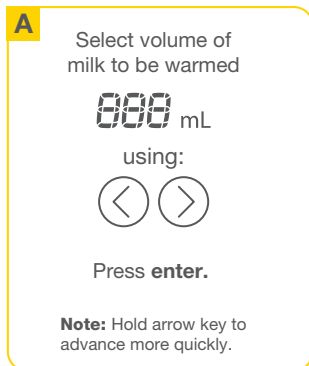


Confirm & Begin

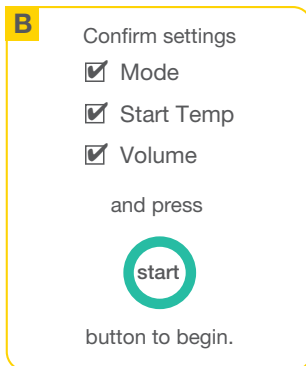

Ready

Flashing green light indicates milk is ready.

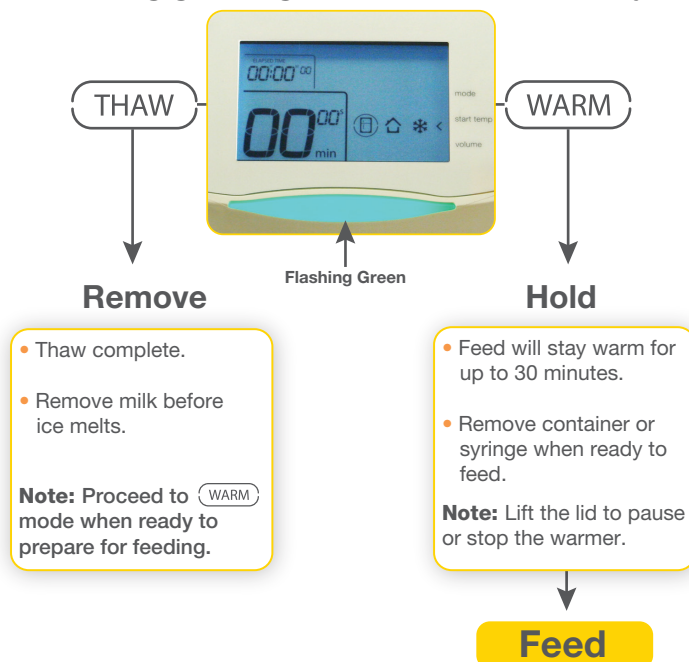

TIP: If warming or thawing milk in the Colostrum Protector, refer to the How to Warm and Thaw using Colostrum Protector Section.

Waterless Milk Warmer™ Quick Start Instructions

Review instruction manual for important safeguards prior to first use.

Colostrum Protector: Warming and Thawing in the Waterless Milk Warmer



Waterless Milk Warmer Input per mL Volume				
Actual Volume (mL)	Thawing	Warm from Room Temp	Warm from Refrigerator	Warm from Frozen
1	1	3	2	1
2	3	3	4	Thaw 3* Warm 4*
3	6	5	7	3
4	13	12	14	7
5	30	30	30	11
6	45	40	40	18

Note: The double wall of the Medela Colostrum Protector acts as an insulator, requiring additional time in the Waterless Milk Warmer to effectively thaw and warm. Follow the chart above specifically for the Medela Colostrum Protector.

These instructions do not apply to any other container.

Refer to the Instruction for Use document for complete instructions.

* When warming 2 mL from a frozen state, first thaw and then warm following the chart recommended inputs.



Scan the QR code to access additional helpful instructional resources