
momcozy

KNEADING LACTATION MASSAGER

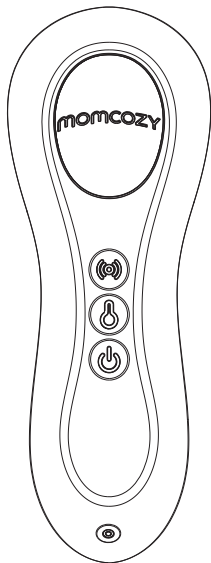LM02

User Manual

IMPORTANT! READ AND FOLLOW THESE INSTRUCTIONS CAREFULLY AND KEEP THEM FOR FUTURE REFERENCE.

→ ENGLISH


 WARNING

- This Kneading Lactation Massager is intended for a single user. It is not recommended to rent or share.
- Do NOT allow anyone unfamiliar with this product to use it.
- Do NOT use if the charging cable or product is damaged. Contact our customer service team for an exchange.
- If the charging cable casing or wiring becomes loose, separated, or frayed, stop using the charging cable immediately and contact the device manufacturer.
- Parental or guardian supervision is required at all times if users have physical or intellectual disabilities
- Do not clean or sterilize the product with a washing machine, microwave, or dishwashing machine.
- This kneading lactation massager is intended to help women overcome some breastfeeding challenges, like engorged breasts or clogged milk ducts. Please use it in strict accordance with the described purpose in this manual.
- Only use the charging cable that comes with the unit.

INDICATIONS FOR USE

Momcozy kneading lactation massager is intended to alleviate the pain and discomfort associated with breastfeeding, clogged ducts, engorgement, and mastitis. Moreover, it speeds up the let-down and milk flow.

PRODUCT OVERVIEW

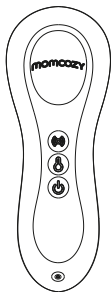Kneading Lactation
Massager * 1

Charging Cable
* 1


Storage Bag
* 1

User Manual
* 1

PRODUCT DESCRIPTION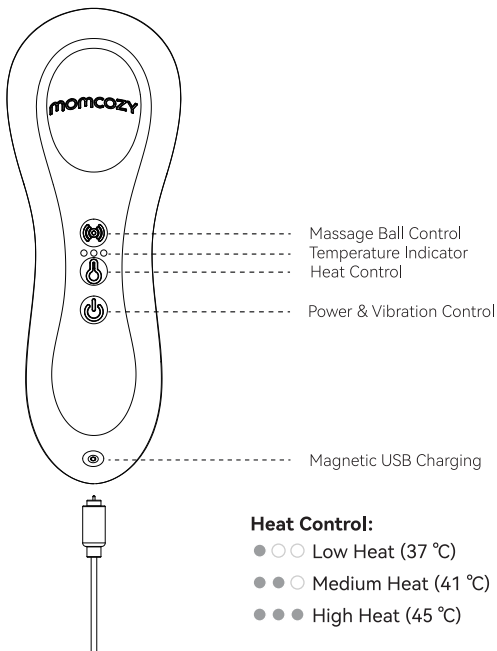

	Power/Vibration Control Press and hold for 1.5 seconds to enter standby mode, and the indicator remains solid white. This massager will change from 6 vibration patterns by pressing the power button.
	Massage Ball Control In standby mode, short press to activate the massage balls. This massager will change from low speed/medium speed/high speed/pause by pressing this button.
	Heat Control In standby mode, short press to change from low heat (37 °C)/medium heat (41 °C)/high heat (45 °C)/pause.
	Battery Indicator In charging - the indicator will be blinking white. Fully charged - the indicator will be solid white. In working - the indicator will be solid white. Low battery - the indicator will be blinking white and go out when running out of battery.

CHARGING YOUR MASSAGER


1. Connect the charging cable provided to the massager
2. Plug the USB into a computer or USB adaptor(DC 5V 1.5A).
3. The indicator will be blinking white while the massager is charging.
4. The massager is fully charged when the indicator remains solid white.
5. It takes approximately 1.5 hours to get the device fully charged. Recharge the massager as needed.

HOW TO USE


1. Use the massage balls to reach deeper into the breast ducts and gently massage the clogged area on the breast.


Deep-kneading
massage balls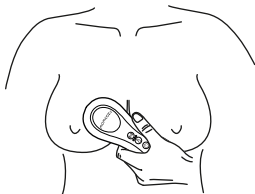

Massage the clogged area on the breast with a comfortable vibration pattern.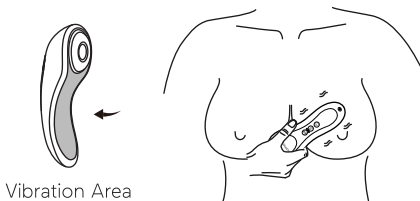


3. Allow a few minutes for the massager to reach full heat and warm the breast to stimulate milk flow.

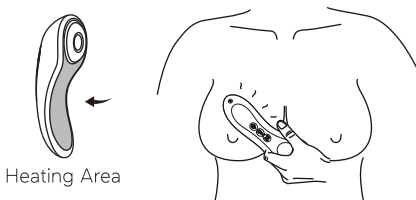4. Massage the clogged area on the breast with the curved side, while moving towards the nipple.

Massage clogged area
with curved side

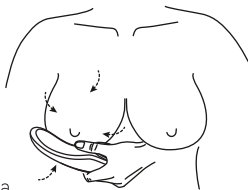Note:

1. When heating, the device will change from high heat to low heat after 20 minutes. It will shut off automatically after heating for 30 minutes.
2. It cannot be turned on when charging.
3. Fully charge the battery before the first use.
4. Allow a few minutes for the massager to reach full heat.

CARE AND STORAGE

- Always clean your massager before and after every use.
- Unplug from power source before cleaning.
- Clean with warm water.
- Do NOT submerge the massager in water.
- Allow the massager to air dry.
- Avoid direct sunlight.
- Store the massager in a clean and dust-free place.

PRODUCT SPECIFICATION

Product Name: Kneading Lactation Massager

Model: LM02

Net Weight: 150g

Input Voltage: DC5V 1.5A

Working Temperature: 0 °C to 47 °C

Storage Temperature: -10 °C to 60 °C

Product Size: 152 mm * 55 mm * 51 mm

Battery Capacity: 1200mAh

Packaging Size: 176*86*58 mm

FCC COMPLIANCE

Any Changes or modifications not expressly approved by the party responsible for compliance could void the user's authority to operate the equipment.

This device complies with part 15 of the FCC Rules.

Operation is subject to the following two conditions:

- (1) This device may not cause harmful interference, and
- (2) this device must accept any interference received, including interference that may cause undesired operation.

Note:

This equipment has been tested and found to comply with the limits for a Class B digital device, pursuant to part 15 of the FCC Rules. These limits are designed to provide reasonable protection against harmful interference in a residential installation. This equipment generates uses and can radiate radio frequency energy and, if not installed and used in accordance with the instructions, may cause harmful interference to radio communications. However, there is no guarantee that interference will not occur in a particular installation.

If this equipment does cause harmful interference to radio or television reception, which can be determined by turning the equipment off and on, the user is encouraged to try to correct the interference by one or more of the following measures:

- Reorient or relocate the receiving antenna.
- Increase the separation between the equipment and receiver.
- Connect the equipment into an outlet on a circuit different from that to which the receiver is connected.
- Consult the dealer or an experienced radio/TV technician for help.

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