



momcozy Mobile Flow™ Quick Start Guide





Step 1:

To get started, search for the Momcozy app on your smartphone's app store, either Google Play Store or App Store. The app is compatible with iOS 13 and Android 8.0 or higher versions. Please note that the app's Bluetooth connection may depend on the smartphone connected to the device.

1


Step 2:

After installation, please register your account. If you have a Momcozy account, log in.

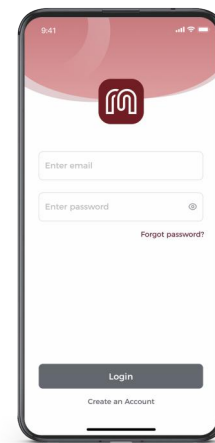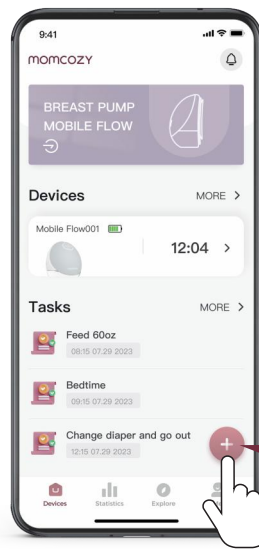


Step 3:

Please ensure you have turned on Bluetooth on your phone, and the breast pump should remain powered. Add devices through the "+" and "Devices" buttons.

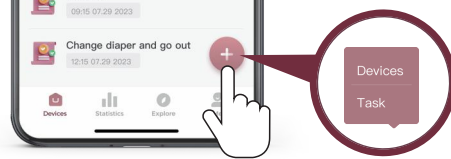


2

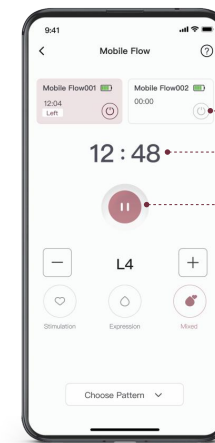

Off
Running Time
Pause/Continue

Suction Level


Mode Selection

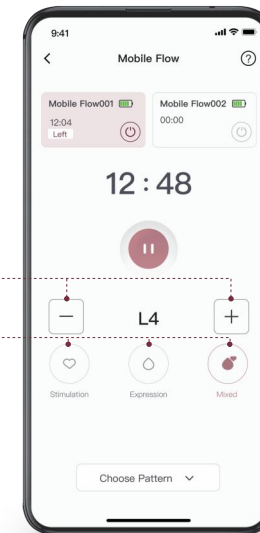


Step 4:
After connecting, use the app to control the breast pump.

3


Installation Guidelines

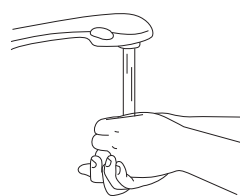


1
Wash your hands thoroughly with soap and water.

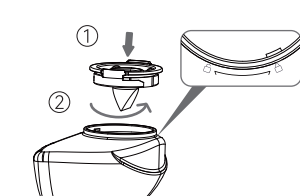

2
Assemble the milk collector by rotating it in the direction of the lock icon at the mouth of the bottle.

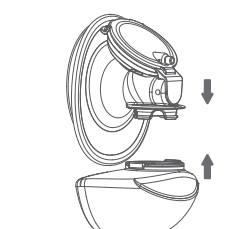

3
Fasten the flange to the mouth of the bottle and assemble with the milk collector.

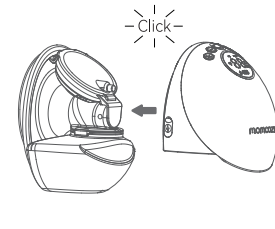

4
Assemble the motor and flange.

4


Cleaning & Sterilization

*Non-washable parts

Pump Motor




*Washable parts

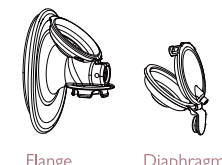

Flange

Diaphragm

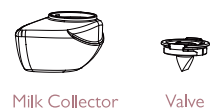

Milk Collector

Valve

Cleaning and Sterilizing Washable Parts
1. Clean with soapy water(40-60 °C) for 3-5 minutes.
2. Use boiling water/steam/sterilization pot for sterilization. 100 °C (212 °F) for 5-10 minutes is required.

* It is not recommended to sterilize pump parts in a microwave or UV sterilizer. High temperature(> 110°C) will shorten their service life and cause safety hazards.

For any product problems, please refer to the operation guide video for further solutions. Scan the QR code below to access the operation guide video.



5

Battery Indicator



- | | |
|--|--------------------------|
| 1 Battery Display | 2 Milk Overflow Reminder |
| 3 Long Press: Turn On / Off | 4 Mode Selection |
| 5 Short Press: Pause/ Continue | 6 Bluetooth Indicator |
| 7 Switching Direction | 8 Mode Indicator |
| 9 Running Time / Suction Level Display | |
| 10 Increase / Decrease | |

Tips

- 1) The expected service life of the washable parts is 1-3 months. It is recommended to replace the washable parts after long-term storage or before pumping for your next child.
- 2) The breast pump will shut off automatically after pumping for 30 minutes.
- 3) If irritation or discomfort occurs, use your finger to break the seal between the flange and the breast tissue, discontinue use and see a healthcare provider.

If you have any questions, please contact us by email.
support@momcozy.com

6