

momcozy

Wearable Breast Pump

Mobile Flow™

User Guide



EN

Read this user manual carefully before using this device and keep this manual for future reference.



TABLE OF CONTENTS

1. Product Introduction	03
2. Warning Instructions and Precautions	03
3. Safety Information	05
4. App Quick User Guide	07
5. Product Description	08
6. Operating Instructions	11
7. Storage Conditions	31
8. Specification	32
9. Maintenance and Replacement Parts	33
10. Declaration of Conformity	33
11. Troubleshooting	34
12. Disposal	35
13. Warranty	36
14. Travel or International Use Statement	37
15. Manufacturer Information	37
16. Important safety information	38

1. Product Introduction

The Momcozy Wearable Breast Pump is intended to be used by lactating women to express and collect milk from their breasts. The Momcozy Wearable Breast Pump forms a vacuum around the nipple to express breast milk. The milk is collected in a milk collector.

The device includes stimulation mode, expression mode and mixed mode. In stimulation mode, the breast pump begins with a quick and short sucking pattern to get your milk to start flowing. In expression mode, the breast pump begins with a slow sucking pattern for milk expression, sucking more deeply and more slowly. In mixed mode, the breast pump begins with a quick and short sucking pattern and then with a slow and long sucking pattern. To prevent milk from flowing into the vacuum system, the milk collection sets include a silicone diaphragm that physically separates the milk-contacting pathway from the vacuum system.

The device is intended for a single user.

2. Warning Instruction and Precautions

The warning signs and graphic symbols in the manual are intended to enable you to use the product safely and correctly and to prevent harm to you and others. Warning marks and graphic symbols are described as follows:

Warning/precaution symbols	
 Warning	A warning alerts the reader about a situation that, if not avoided, could result in death or serious injury. It may also describe potential serious adverse reactions and safety hazards.
 Precautions	Indicating a hazard alert that warns the reader of a potentially hazardous situation which, if not avoided, may result in minor or moderate injury to the user or patient or damage to the equipment or other property. It may also be used to alert against unsafe practices. This includes the special care necessary for the safe and effective use of the device and the care necessary to avoid damage to a device that may occur as a result of use or misuse.
Contraindications	Contraindications are conditions under which the device should not be used because the risk of use clearly outweighs any possible benefit. There may be persons for whom the device should not be used because of their health status.
Notes	Indicates the need for attention, if not attention may lead to incorrect use of the product or property device damage.

Symbol Description

	CE LOGO		UKCA LOGO
	Keep Dry		Pile Limit 5 layers
	Recycling LOGO		Food Contact Products
	Batch code		Serial Number
	Distribution packages shall not be exposed to sunlight		FCC LOGO
IP22	The first number 2: Protected against solid foreign objects of 12.5mm diameter and greater. The second number 2: Protected against vertically falling water drops when enclosure tilted up to 15°.		
	Disposal in accordance with Directive 2012/19/EU (WEEE)		
	This device has not been tested for use in an MR environment and should not be used exposed to MR environments while patients are wearing the device. Keep it outside the MRI scanner room.		
	UK Authorized Representative		French ERP Packaging
	EU Authorized Representative		

Do not use this Pump if you have an active implanted device that would be affected by a magnetic field (e.g., pacemaker; implantable cardioverter-defibrillator).

3. Safety Information

When using electrical products, especially if children are present, the following basic safety information should always be maintained.

3.1 Warning

- 1) Do not use this device if you have a cardiac pacemaker, implanted defibrillator, or other implanted metallic or electronic device. Such use could cause electrical interference, or death.
- 2) This breast pump is not intended for use by persons (including children) with reduced physical, sensory or mental capabilities, or lack of experience and knowledge. Such persons can only use this breast pump if they are supervised by or have received proper instruction concerning use of the appliance by a person responsible for their safety.
- 3) Danger: To reduce the risk of electrocution:
 - Always unplug electrical devices immediately after charging.
 - Do not use while bathing, showering, driving or swimming.
 - Do not reach for electrical product that has fallen into water, please unplug the power adapter from wall outlet immediately.
- 4) The Momcozy Wearable Breast Pump is a personal care item and for hygienic reasons should never be shared between mothers or resold from one mother to another. Either of these actions will void the warranty.
- 5) If you are a mother who is infected with Hepatitis B, Hepatitis C or Human Immunodeficiency Virus (HIV), pumping breast milk using the Momcozy Wearable Breast Pump will not reduce or remove the risk of transmitting the virus to your baby through your breast milk.
- 6) Close supervision is necessary when product is used near children or invalids. Do not allow children or pets to play with the motor unit, packaging materials or accessories.
- 7) Use product only for intended use as described in this manual.
- 8) Do not use attachments or other milk collection components other than those recommended by manufacturer.
- 9) Check the power adapter before charging, never use the damaged power adapter.

- 10) Never leave it product unattended while charging.
- 11) Keep the product and all attachments away from heated surfaces.
- 12) Never use it while sleeping or drowsy.
- 13) Do not operate where aerosol spray products are being used or oxygen is being administered.
- 14) Do not use the device around explosive or flammable material, in an MRI environment, in a hyperbaric chamber, or near a fireplace or radiant heater.
- 15) Avoid strangulation resulting from Charging cable.
- 16) Users are not allowed to repair the product. If you do so, your warranty becomes invalid. For repairs, sent to the manufacturer or a designated franchise distributor.
- 17) Always switch off the breast pump before you remove the pump body from your breast, in order to release the vacuum.
- 18) Powered breast pumps that are designed for single users should never be rented or shared.
- 19) Never use the breast pump while you are pregnant, as pumping can induce labor.
- 20) Please make sure that your skin of breast must be intact and healthy before use. If you feel overheated, please turn off the device. In order to avoid Low temperature burn, the maximum continuous use shall not exceed 30min each time, the interval between two uses shall be at least 5min to allow the device to cool down.
- 21) **For product service life (i.e. working life)**
The device has no shelf life, the service life of the product is 500 hours. The expected operating life of the washable parts is one to three months. Please check whether the product can run normally before use. Use beyond the service life may lead to degradation of product performance. Such as, the product cannot control the vacuum correctly, excessive vacuum may cause injuries.

3.2 Precaution

- Do not place or store where product can fall or be pulled into bathtub, sink or pool.
- Do not place or drop product into water or other liquid.
- Do not immerse in any liquid for any reason or subject the unit to extreme shocks, such as dropping the pump.
- Do not continue pumping for more than five minutes at a time if you do not succeed in expressing any milk. Try to express at another time during the day.
- If discomfort is felt, then suction can be broken by inserting a finger between the breast and the flange.
- If irritation or discomfort occurs, discontinue use and see a health care provider.
- If the power adapter casing or wiring becomes loose, separated, or frayed, stop use of the power adapter immediately and contact the device manufacturer. Stop using the device immediately if you note any smoke or burning from the pump unit or power adapter.
Statement: If you feel excessive warmth from the Wearable breast pump while pumping, stop pumping immediately and remove pump.

Notes: Avoid direct sunlight to the main unit.

4. App Quick User Guide

Before using a new breast pump, please refer to the following user instructions on the App, which can help you to use the product more effectively.

Step 1:

To get started, search for the Momcozy app on your smartphone's app store, either Google Play Store or App Store. The app is compatible with iOS 13 and Android 8.0 or higher versions. Please note that the app's Bluetooth connection may depend on the smartphone connected to the device.

Step 2:

After installation, please register your account. If you have a Momcozy account, log in.

Step 3:

Please ensure you have turned on Bluetooth on your phone, and the breast pump should remain powered.

Add devices through the "+" and "Devices" buttons.

After connecting, use the app to control the breast pump.

5. Product Description

The product is mainly composed of main unit and milk collection set etc.

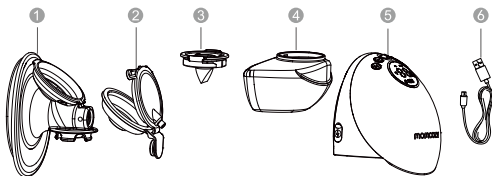


Fig.1 unit description

Description of breast pump unit		
NO.	Item	Description
①	Double-sealed flange(24mm)	All of these components make up the milk collection set
②	Diaphragm	
③	Valve	
④	Milk collector (150ml)	
⑤	Main unit	Pump motor
⑥	USB Type-C cable	Input end is USB 2.0 interface, the output end is Type-C interface

Other Accessories Included:

Flange Insert
Valve
Flange Cover
Storage Lid
Storage box

Quick Start Guide
Flange Size Ruler
User Manual
Milk collector (150ml)

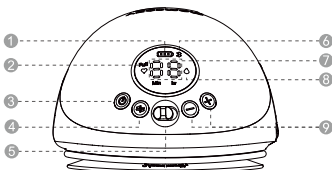


Fig.2 control panel

- ① Battery Display
- ② Milk Overflow Reminder
- ③ On / Off
- ④ Mode Selection
- ⑤ Switching Direction
- ⑥ Bluetooth Indicator
- ⑦ Running Time / Suction Level Display
- ⑧ Mode Indicator
- ⑨ Increase / Decrease

Description of control panel

No.	Buttons and LEDs	Functions
1		Long press and hold to turn on / off; Short press to pause, then press any button to continue your pumping session.
2		Mode selection
3		Decrease suction strength
4		Increase suction strength
5		Toggle button to select breast pump direction, L or R.

6		Stimulation Mode: In this mode, the breast pump begins with a quick and short sucking pattern to get your milk to start flowing.
7		Expression Mode: The breast pump begins with a slow sucking pattern for milk expression, sucking more deeply and more slowly.
8		Mixed Mode: In this mode, the breast pump begins with a quick and short sucking pattern followed by a slow and long sucking pattern.
9		Suction Level Display: The breast pump has 15 suction levels to choose from. Running Time Display: Seconds displayed if shorter than 1 minute.
10		Battery Display

Battery Level Indicator		
Low Battery: Blinking	In Charging: Lights up one by one	Fully Charged: Stays on solid
		

After the battery is depleted and the device automatically shuts down, it can not be work again and can only display a low battery status.

6. Operating Instructions

6.1 Charging battery

Please charge the product before using it for the first time. The breast pump is powered using an internal rechargeable lithium battery.

Our products are packaged with a Charging cable (USB2.0/Type-C line) but not with a power adapter. Please purchase the power adapter by yourself before charging. Make sure to buy the power adapter which matches the USB 2.0 interface, and please make sure the input power of the power adapter is AC 100~240V 50/60 Hz and the output power is DC 5V==2A.

- When the first battery indicator bar flashes, it means the battery is low, and there are 30 minutes of use time left. Please charge the device in time.
- After being fully charged, it can be used for 4-5 sessions (about 150 minutes) of pumping, 30 minutes per session.

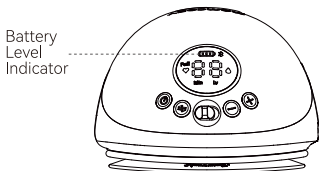


Fig. 3

Charge the battery as below:

- Insert the USB end of Charging cable into the USB interface of power adapter;
- Insert power adapter plug into AC socket;
- Insert the Type-C end of Charging cable into the charging port.

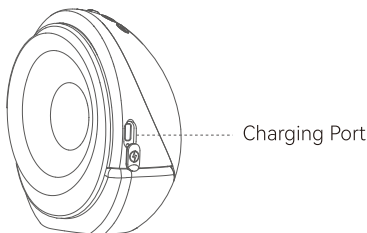


Fig. 4 Position of Charging port

The device will take approximately 150 minutes to fully charge.

**Warning**

The breast pump cannot be used during charging!

The Charging cable is only used to connect between the power adapter and the Momcozy Wearable Breast Pump unit for charging, please do not connect to other equipment, otherwise, it may cause electric shock accident!

- If the power adapter casing becomes loose, separated, or frayed, stop use of the power adapter immediately and contact the manufacturer of power adapter.
- If the Charging cable is damaged, broken or cracked, stop the use of the Charging cable immediately and contact the manufacturer to replace a new one (See the last page of this manual for contact information).
- Stop using the device immediately if you note any smoke or burning from the pump unit or power adapter.



Caution:

Using the wrong type of adapter or purchasing a poor-quality adapter may be reduced the device's safety, and may damage your property and health. Please purchase the adapter that meets the IEC 60601-1 and IEC 60601-1-2 standards in professional markets, and make sure the output of the power adapter is DC 5V=2A.

Notes:

- After fully charging the battery, if the working time of the battery is shorter than before and you want to replace it, don't try to replace the rechargeable battery by yourself. Please contact the manufacturer for replacement. Self-disassembly and replacement of the battery may cause damage to the main unit and battery.
- When the product is not used for a long time, the battery will discharge slowly. In order to avoid battery damage due to low voltage for a long time, please charge the device every three months.

6.2 Cleaning and disinfection

6.2.1 General description/requirement

Providing breast milk is one of the best things you can do for your baby's health and development. Pumping your milk is one way to provide breast milk to your baby. Keeping the parts of your pump clean is critical, because germs can grow quickly in breast milk or breast milk residue that remains on pump parts. Following these steps can help prevent contamination and protect your baby from infection. If your baby was born prematurely or has other health concerns, your baby's health care providers may have more recommendations for pumping breast milk safely.

1) BEFORE EVERY USE

Inspect and assemble a clean pump kit.

Clean pump motor with disinfectant wipes.

2) AFTER EVERY USE

Store milk safely: Use breast milk storage bags or clean, food-grade containers to store expressed breast milk. Make sure the containers are made of glass or plastic and have tight-fitting lids, and label them with the date and time, and immediately place them in a refrigerator, freezer, or cooler bag with ice packs.

Clean pumping area: Clean the dials, power switch, and countertop with disinfectant wipes.

3) Parts that need to be cleaned and disinfected




Clean and disinfect all parts that come into contact with breast and breast milk before you use the breast pump for the first time and after every use.

- Milk contact part: parts of milk collection set, including Flange, Diaphragm, Valve, and Milk Collector.
- Parts that only contact the breast skin: Flange.

4) Before cleaning/disinfection:

a. Please Wash your hands well with soap and water for 20 seconds.

b. Please disassemble all the parts as follows:

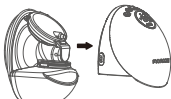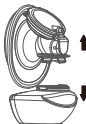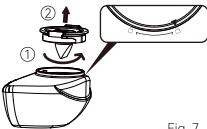 Fig. 5	 Fig. 6
1). Disassemble the main unit and flange assembly.	2). Pull the flange upwards to separate it from the milk collector.
 Fig. 7	 Fig. 8
3). Open the cap by rotating it in the direction of the unlock icon at the mouth of the bottle.	4). Remove the diaphragm assembly from the flange.



Caution:

Never put the main unit, charging cable or the adapter (If you have purchased an adapter) in water or a dishwasher, as this causes permanent damage to these parts.

6.2.2 Cleaning and disinfecting parts contacting with the milk

Step 1: Cleaning

Please manually clean the device! The safety of using a dishwasher to clean and disinfect this product has not been verified.



Warning

Do not use antibacterial or abrasive cleaning agents to clean the breast pump parts as this may cause damage.

1) Prepare the below items:

- Soap and hot water
- Drinking-quality water
- Clean wash basin used only for infant feeding items
- Clean brush used only for infant feeding items
- Unused dish towel or paper towel

2) Rinse: Rinse parts of milk collection set and any other feeding items by holding them under running water to remove the remaining milk or dirt. The water can be warm or cold, whatever you prefer.

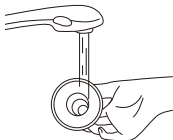


Fig. 9

3) Wash milk collection parts:

- Place parts of the milk collection set in a clean wash basin used only for infant feeding items. Do not wash directly in the sink because it may contain germs that could contaminate these items.
- Soak parts of the milk collection set in warm 40–60 °C (104–140 °F), soapy water for 3–5 minutes.

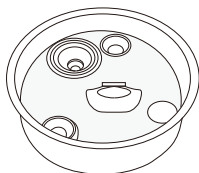


Fig. 10

- Using a clean brush used only for infant feeding items to scrub items.

4) Rinse again: Rinse by holding items under running water, or by submerging them in fresh water in a separate basin that is used only for cleaning infant feeding items. If there is still dirt, repeat step 3 and re-rinse until there is no visible dirt.

5) Air-dry thoroughly: Place parts of the milk collection set, wash basin, and bottle brush on a clean, unused dish towel or paper towel in an area protected from dirt and dust. Do not use a dish towel to rub or pat items dry because doing so may transfer germs to the items!

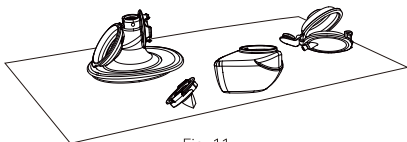


Fig. 11

6) Clean wash basin and bottle brush: Rinse the wash basins and brush well and allow them to air-dry after each use. Wash them every few days by hand with soap and warm water. If your baby is less than 3 months old, was born prematurely, or has a weakened immune system due to illness (such as HIV) or treatment (such as chemotherapy for cancer), wash basin and bottle brush after every use.

After Cleaning:

For extra germ removal, disinfect the parts of milk collection set, wash basin, and bottle brush at least once daily after they have been cleaned. Items can be disinfected using boiling water with a disinfection setting. Disinfection is especially important if your baby is less than 3 months old, was born prematurely, or has a weakened immune system due to illness or treatment.

For the disinfection of milk contact parts, please refer to step 2.

Step 2: Disinfection

Caution: Do not sterilize pump parts in a microwave or UV sterilizer. High temperature (> 110 °C) will shorten their service life and cause safety hazards.

Before disinfection, make sure you have a cleaned milk collection set, bottle brushes, and wash basins using the methods above. Disinfect all items in contact with milk (even the bottle brush and wash basin!) by using one of the following methods.

1) Prepare the below items:

- A household pot
- Drinking-quality water
- Clean tongs



Caution:

During disinfection with boiling water, prevent the bottle or other parts from touching the sides of the pot. This can cause irreversible product deformation or damage.

2) Place the disassembled milk collection set in the pot and fill the pot with enough water to cover the components to disinfect. Let the water boil for 5-10 minutes, making sure the parts do not touch the sides of the pot.

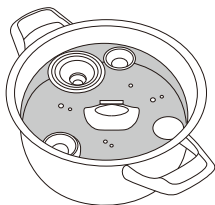


Fig. 12

3) Remove items with clean tongs.

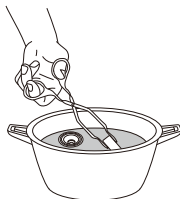


Fig. 13

4) Air-dry thoroughly.

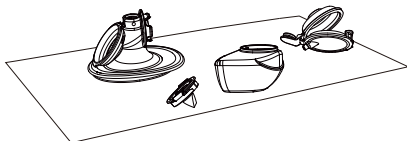


Fig. 14

Step 3: Store safely

Store dry items safely until needed. Ensure the clean pump parts, bottle brushes, and wash basins have air-dried thoroughly before storing. Items must be completely dry to help prevent germs and mold from growing. Store dry items in a clean, protected area to prevent contamination.

1. Wash hands: Wash hands well with soap and water.
2. Put back together: Put together the clean, dry pump parts.
3. Store safely: Place reassembled pump parts and other feeding items, wash basin, and dry bottle brush in a clean, protected area such as inside a closed kitchen cabinet that is used only to store clean dishes.

6.2.3 Clean / Disinfect the other parts

The shell of the main unit need to be cleaned and disinfected for the first time and after each use. The other parts, do not come into contact with the human body nor the milk during pumping, and only need to be cleaned when it is dirty.

Cleaning/disinfecting the items at room temperature 5 °C~ 40 °C as below:

Prepare the below items:

- Disinfection
- 70% isopropyl alcohol wipes
- Clean soft cloth

Cleaning method:

1. Before performing this cleaning procedure, wash your hands with hand disinfection first.
2. Remove all residual foreign matter from the main unit and/or Charging cable using a clean soft cloth.
3. Take a piece of 70% isopropyl alcohol wipes to swab on the parts until no visible contaminants remain.
4. Dry the device by placing it at room temperature for 10min to evaporate the residual cleaning reagent solution on the product.

Disinfection method:

Cleaning the shell of the main unit as the above method before disinfection.

1. Disinfect the shell of the main unit with a soft cloth dampened with the 70% isopropanol for 3 minutes.
2. Dry it by placing the main unit at room temperature for 10min to evaporate the residual disinfection solution.

6.3 Direction for use

6.3.1 Preparation before use



Caution:

1. Wash hands with soap and water, and inspect and assemble a clean pump kit.

2. Duration of pumping session:

This product is powered by a rechargeable lithium battery, after being fully charged, it can be used for 4~5 sessions (about 150 minutes) of pumping, 30 minutes per session. A typical pumping session lasts about 10-15 minutes per breast, but you should only pump as long as it is comfortable and productive for you.

1. Select Flange Size

The Momcozy wearable breast pump is provided with 24mm flange and 17/19/21mm flange insert. If the provided flange and flange insert are not the right size, you can purchase various sizes of flanges and flange insert on Amazon or Momcozy's official website. Type "momcozy" in the search bar and find the flange size and flange insert that fits you.

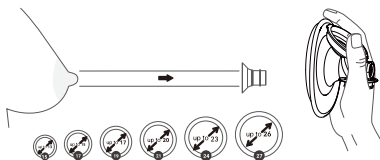


Fig.15 select right size

2. Stimulate the Nipple

- Stimulate your nipple to get a full size (do not skip this step!).
- Place the nipple into the smallest hole you can comfortably fit into without forcing it.
- Select the hole that can comfortably fit the very base of your nipple.
- Get your measurement based on the sizing chart.

3. Test the Flange Fit

- To find the suitable size flange, first, you need to make sure you place the flange in the right place on your breast.
- Please press the flange on your breast until there is no gap. Make sure your nipple is right in the middle of the flange.

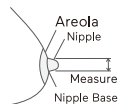


Fig.16



Fig.17

Too Small

Too small: During pumping, if your nipple continues to rub against the inner wall of the flange, then this flange is too small for you.

Good Fit

Suitable fitting: During pumping, if your nipple moves smoothly without rubbing or touching the inner wall of the flange, and almost no areola is sucked inside, then this flange is right for you.

Too Large

Too large: During pumping, if a large area of areola is sucked inside, then this flange is too big for you.

4. Get the Right Bra



Fig.18

- ① Made of stretchy fabric
- ② No underwire or padding
- ③ Has a flap
- ④ Full coverage

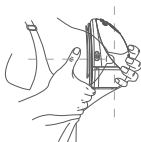


Fig.19

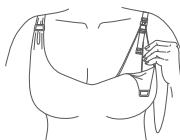


Fig.20

NO GAP

When you put the pump inside your bra, the silicone flange of this pump should be firmly pressed against your skin without a gap.

IF IT MOVES

If you feel the pump moves during use, please tighten your bra strap to adjust the pump again, or choose another bra.

5. Recommend Posture



Caution:

Always sit or stand upright while pumping to prevent the milk from leaking. This pump is not to be used when lying down or sleeping to ensure the proper orientation is maintained during use.

POSTURE

Make sure the pump and the milk collector always stay at a vertical angle. Otherwise, the milk will spill out.



Fig.21 right posture

6.3.2 Assembling the Pump



Caution:

Make sure you have cleaned and disinfected the parts of the breast pump that come into contact with milk or breast.







Warning

- Be careful, when you have disinfected the breast pump parts by boiling them, they can be very hot. To prevent burns, only start assembling the breast pump when disinfected parts have cooled down.
- Wash your hands thoroughly with soap and water before you touch breast pump parts and breasts to prevent contamination. Avoid touching the inside of milk collection set.

Notes:

- Correct placement of the Flange, Diaphragm, Milk collector, Valve, etc. is essential for the breast pump to form a proper vacuum.
- Before each use visually inspect the individual components for cracks, chips, tears, discoloration, or deterioration. In the event that damage to the device is observed, please discontinue use until the parts have been replaced.

Wash your hands thoroughly with soap and water, then assemble the pump as follows:

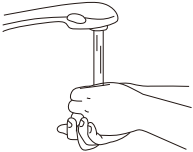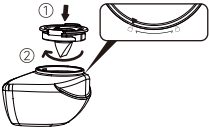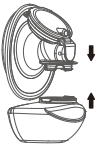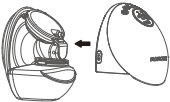 Fig. 22	 Fig. 23
1. Wash your hands thoroughly with soap and water.	2. Assemble the milk bowl by rotating it in the direction of the lock icon at the mouth of the bottle.
 Fig. 24	 Fig. 25
3. Fasten the flange to the mouth of the bottle and assemble with the milk collector.	4. Assemble the main unit and flange .

The breast pump is now ready for use.

6.3.3 Pumping instructions



Warning

- Before you remove the pump body from your breast, always switch off the breast pump to release the vacuum.
- Do not continue pumping for more than 5 minutes at a time if you do not succeed in expressing any milk. Try to express at another time during the day.
- If the process becomes very uncomfortable or painful, stop using the pump and consult your healthcare professional.



Caution:

- Monitoring milk level during pumping to ensure overfill does not occur.

1. Alignment

a. Before pumping, please make sure the pump is pressed tightly on your breast without a gap (Fig. 26).


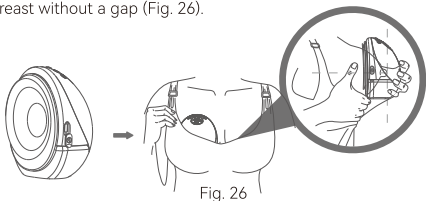



Fig. 26

- b. To align, hold the flange against your breast so you can see if your nipple is centered in the flange tunnel. You may need to bend forward.
- c. After you've aligned, look through the flange to confirm your nipple is centered before pumping. The tip of your nipple should NOT touch the top, sides or bottom of the flange tunnel. If it does, try again.

First Perspective

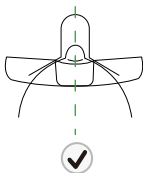

Correct Alignment
Milk expression

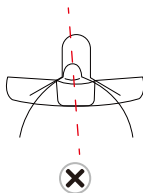

Incorrect Alignment
Try again

Fig. 27

Note:

Correct nipple alignment is key to comfort expression and performance.

Nipple must be positioned correctly in flange tunnel before pumping to avoid discomfort.

See Section 8.3.1 Preparation before use-Select Flange Size.

2. Secure Pump In Bra

Secure Pump in place by bra, taking care to keep Pump pressed firmly against breast. If necessary, tighten bra to keep pump securely in place and start pumping.



Fig. 28

3. Start Pumping

Do not slide the pump directly into your bra, follow step-by-step instructions.

Disconnect from Charger before use.

A. Turn on the pump

Consult your health care practitioner to understand how long you should pump. Momcozy pump shuts off automatically after 30 minutes of pumping time.

Long press the On / Off / Pause button to turn on or turn off the pump.

Short press the On / Off / Pause button to pause, then press any button to continue your pumping session.

- The default mode is stimulation mode when you first use the pump.

- The default vacuum level is level 1.

B. Select the mode

Modes: Stimulation Mode & Expression Mode & Mixed Mode

When you first use the pump, the default setting is suction level 1 of stimulation mode.

C. Select the vacuum level

Momcozy pump has 15 vacuum pressure settings for each mode, giving you control over what feels comfortable and works most efficiently in all modes.

Once you can feel the vacuum pressure on your nipple, use the Increase/Decrease button to adjust the vacuum pressure to find your preferred setting in all modes. The LED display will be off after no operation for 60 seconds, press any button can activate the LED display again.

Tip: A higher vacuum pressure does not always mean a higher volume of milk. Putting your nipple under excess stress can reduce milk production. Always ensure you are pumping at a comfortable level. If you don't feel suction, please check the pump is assembled correctly.



Caution:

- * Momcozy pump does not support use while charging. Always disconnect from the charger and close the charging cap before use.
- * Momcozy pump should not be used while lying down, engaging in strenuous activities or performing activities with risk of harm.
- * Using a breast pump should not cause pain. Do not try to express with a vacuum pressure setting that is too high or uncomfortable. If excessive discomfort or pain is felt during pumping, stop pumping, break the seal around your nipple and discontinue the session.

4. Pause / Start Pumping

To pause while pumping, short press the On / Off / Pause button and the mode indicator will be blinking. Press any button to continue your pumping session.

5. Turn off the Pump

Long press the On / Off / Pause button to turn off the pump. In addition, the product will automatically shut down after the 30min of pumping time.

6. Remove the Pump

Use your finger to break the seal between the flange and the breast tissue.

Lean slightly forward to remove the pump from the breast after your pumping session, then keep the flange opening side up to prevent milk from pouring over.

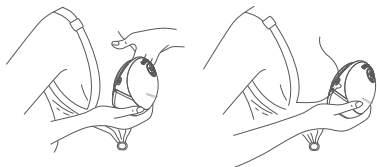


Fig. 29

We recommend the user stop the pumping session once the expressed milk volume of the milk collector approaches 120 ml to avoid overflow.



Warning

- The Milk collectors provided are for collecting the milk during suction. If you need to feed the baby or store the expressed breast milk, please use another Milk collector with a cap.



Warning

- Refrigerate or freeze expressed milk immediately or keep it at a temperature of 16 °C~ 25 °C for 2-4 hours before you feed your baby.

6.4 Storing breast milk

6.4.1 Proper storage of breast milk

Following recommended storage and preparation techniques can maintain the safety and quality of expressed breast milk for the baby's health.

These are general guidelines for storing human milk at different temperatures. Various factors affect how long human milk can be stored safely. Such factors include milk volume, room temperature when milk is expressed, temperature fluctuations in the refrigerator and freezer, and cleanliness of the environment.

Storing breast milk after expressing:

- Refrigerate or chill milk right after it is pumped, if possible. You can put it in the refrigerator, place it in a cooler or insulated cooler pack, or freeze it in small (2 to 4 ounce) batches for later feedings.
- Use breast milk storage bags or clean, food-grade containers to store expressed breast milk. Make sure the containers are made of glass or plastic and have tight-fitting lids.
 - Avoid bottles with the recycle symbol number 7, which indicates that the container may be made of BPA-containing plastic.

- Remove air before sealing.
- Lay bags flat to freeze. To prevent sticking between bags, please separate the bags with paper towels.
- Never store breast milk in disposable bottle liners or plastic bags that are not intended for storing breast milk.
- Gently swirl the container to mix the cream part of the breast milk that may rise to the top back into the rest of the milk. Do not shake the milk. This can make some of the milk's valuable parts break down.
- Freshly expressed or pumped milk can be stored:

Type of Breast Milk	Storage Locations and Temperatures		
	Countertop 77 °F (25°C) or colder (room temperature)	Refrigerator 40 °F (4°C)	Freezer 23 °F (-5°C) or colder
Freshly Expressed or Pumped	Within 2–4 hours	Within 24 hours	Within 3 months
Thawed, Previously Frozen	Use within 2 hours after the baby is finished feeding		

These guidelines are for healthy full-term babies only and may vary for premature or sick babies. Check with your health care provider.

Storage tips:

- Clearly label on the storage container with the date it was expressed, include your child's name if you are giving the milk to a childcare provider.
- Do not store breast milk in the door of the refrigerator or freezer. This will help protect the breast milk from temperature changes from the door opening and closing.
- If you don't think you will use freshly expressed breast milk within 4 days, freeze it right away. This will help to protect the quality of the breast milk.
- When freezing breast milk:
 - Store small amounts to avoid wasting milk that might not be finished. Store in 2 to 4 ounces or the amount offered at one feeding.
 - Leave about one inch of space at the top of the container because breast milk expands as it freezes.
- Breast milk can be stored in an insulated cooler with frozen ice packs for up to 24 hours when you are traveling. At your destination, use the milk right away, store it in the refrigerator, or freeze it.

6.4.2 Safe thawing of breast milk

- Always thaw the oldest breast milk first. Remember first in, first out. Over time, the quality of breast milk can decrease.
- There are several ways to thaw your breast milk:
 - In the refrigerator overnight.
 - Set in a container of warm or lukewarm water.
 - Under lukewarm running water.
- Never thaw or heat breast milk in a microwave. Microwaving can destroy nutrients in breast milk and create hot spots, which can burn a baby's mouth.
- If you thaw breast milk in the refrigerator, use it within 24 hours. Start counting the 24 hours when the breast milk is completely thawed, not from the time when you took it out of the freezer.
- Once breast milk is brought to room temperature or warmed, use it within 2 hours.
- Never refreeze breast milk after it has thawed.

7. Storage Condition

How to store the pump

- * Short-term storage
- * Please keep the breast pump out of direct sunlight. Store the breast pump and its accessories in a safe, clean, and dry place, and away from children.
- * Long-term storage

First, please charge the pump before long-term storage in which way the service life of this pump can be prolonged. Second, keep it in a place away from direct sunlight in avoidance of discoloration. Third, clean and dry the washable parts before storing them.
- * Temperature: -20 to 60 °C / -4 to 140 °F
- * Relative Humidity: 15% - 90%
- * Ambient Pressure: 90 to 106 kPa

8. Specification

Model No.	BP223
Name	Wearable breast pump
Power requirements	Input: 100~240Vac, 50/60Hz; Output: 5V $\equiv$ 2A (Charge through provided USB 2.0/Type-C Charging cable and self-purchased power adapter)
Input	5V $\equiv$ 2A
Power supply	3.7V / 1800mAh rechargeable lithium battery
Rated power	10W
Suction modes	Stimulation mode, expression mode, and mixed mode
Vacuum range	Stimulation mode: -65mmHg~-174mmHg Expression mode: -109mmHg~-305mmHg Mixed mode: -65mmHg~-305mmHg
Cycle speed	Stimulation mode: 54~90 cycle/min Expression mode: 25~60cycle/min Mixed mode: 46~84cycle/min
Pump dimension	127mm X 109mm X 77mm
Weight(with all accessories)	302g
Product use life	500 hours
Noise level	≤ 48 dB
Type of protection against electric shock	Internally powered equipment
Operating conditions	Temperature: 0°C to 40°C; Humidity: 15% to 90%, Atmospheric Pressure: 70 kPa to 106 kPa

Transportation & Storage environment	Temperature: -20 °C to 60 °C; Humidity: 15% to 90%, Atmospheric Pressure: 70 kPa to 106 kPa
Battery charging time	150 minutes
Battery usage Time	150 minutes
Operation Frequency	2402-2480MHz, 40 channels for BLE
Max. RF output power	5.65dBm EIRP

9. Maintenance and Replacement Parts

The device contains no user serviceable parts inside: Opening or tampering with this device will void the warranty. In the event the device requires repair, it should be returned to the equipment company or to retailer directly. Modification of any kind is prohibited. The MANUFACTURER will make available on request circuit diagrams, component part lists, descriptions, calibration instructions, or other information that will assist SERVICE PERSONNEL to repair those parts of the device that are designated by the MANUFACTURER as repairable by the SERVICE PERSONNEL. When the product is not used for a long time, the battery will discharge slowly. In order to avoid battery damage due to low voltage for a long time, please charge the device every three months.



Warning

Warning: the replacement of lithium batteries by inadequately trained personnel could result in a hazard.

Do not clean or maintain the device while the device is in use or while charging.

10. Declaration of Conformity

EU Compliance Statement: Shenzhen Root Innovation Technology Co.,Ltd. Hereby declares that this device is in compliance with the essential requirements and other relevant provisions of RED Directive 2014/53/EU.

For the complete CE DOC, please check on this link:

<https://momcozy.com/pages/declaration-of-conformity-momcozy>.

11. Troubleshooting

Problems	Solutions
Pump does not work	• If the let-down has not yet occurred, consider leaving the pump on breast for a longer period of time to observe for possible delayed let-down. Note: Initial milk let-down time is unique for each person and varies by many factors. • Check nipple alignment, pause the pump, and remove it from the breast. Realign the pump as described below. • You may not have the correct Flange size
Milk does not flow	• Pause the pump and remove it from breast. Realign it correctly. • You may also separate the pump from the flange to realign. Then, center your nipple in the flange tunnel and bring the pump to your breast.
Need to realign	• Pause the pump and remove it from breast. Realign it correctly. • You may also separate the Pump from the Flange to realign. Then, center your nipple in the flange tunnel and bring the pump to your breast
Feel discomfort while pumping	• If you're experiencing excessive discomfort, please try the following. • Make sure to press the Momcozy pump firmly against breast during Stimulation phase. • Tighten the bra to make sure the Momcozy pump is held firmly against the breast. • Decrease suction strength. • Realign the Momcozy pump. • You may be pumping for too long. This pump shuts off automatically after running for 30 minutes. • You may not have the correct flange size

Decreased (low) pump suction	• The Pump has 15 suction settings. Press the increase button to increase suction strength. • Check the condition of your washable parts. They should be replaced after three months of use. • If that does not work, try the following: 1. Check connections between all washable parts to ensure all are secure. 2. Visually inspect all washable parts and replace them if damaged. 3. Check that power is on. 4. Press the Pump securely against the breast.
Stop pumping	• Pump stops pumping automatically after 30 minutes. • Short press On / Off / Pause button will pause the pump, press again will resume.
Pump does not stop pumping	• Press the On / Off / Pause button. • If the issue persists, then break the seal by inserting a finger between the breast tissue and the flange. Then Press the On / Off / Pause button to turn the pump off.
Pump or Charger gets wet (immersed in water).	• Dry the pump off immediately. Prop Pump upright with the charger port down and suction controls up, and let it air dry overnight. Do not use the pump or charger within 24 hours. • Contact Customer Care.
Pump is not charging.	• Ensure the USB Type-C cable with standard Charger (not included) is fully plugged into the pump charging port.

Remark: If the above methods still do not solve the problem, please contact the manufacturer, see the last page for contact information.

12. Disposal



Correct Disposal of This Product (Waste Electrical & Electronic Equipment)

(Applicable in countries with separate collection systems)

This marking on the product, accessories (including battery, power supply unit if have) or literature indicates that the product and its electronic accessories should not be disposed of with other household waste at the end of their working life. To prevent possible harm to the environment or human health from uncontrolled waste disposal, please separate these items from other types of waste and recycle them responsibly to

promote the sustainable reuse of material resources. Household users should contact either the retailer where they purchased this product, or their local government office, for details of where and how they can take these items for environmentally safe recycling. Business users should contact their supplier and check the terms and conditions of the purchase contract. This product and its electronic accessories should not be mixed with other commercial wastes for disposal.

13. Warranty

We offer a two-year warranty period (If the corresponding country has a longer warranty period requirement, it should be subject to local regulations. For instance, in Spain, the national requirement is a 3-year warranty period. We will abide by this rule.) , Please contact the manufacturer in case of a claim under the warranty, the contact information please refer to last page of this user manual. If you have to send the unit, enclose a copy of your receipt with clear statement of defect description.

The warranty terms are as below:

- 1) In case of a warranty claim, the date of purchase has to be proven by means of the sales receipt or invoice.
- 2) Repairs carried out during the warranty period can extend the warranty period of the equipment or replacement parts by one year.
- 3) The following cases are excluded under the warranty
 - All damages which are arisen due to improper treatment, e.g. nonobservance of the user instruction.
 - All damages which are arisen due to repairs or tampering by the customer or unauthorized third parties.
 - Damages which are arisen during transport from the manufacturer to the consumer or during transport to the service center.
 - Accessories which are subject to normal wear and tear.
 - Device damage due to privately disassemble devices.
 - Liability for direct or indirect consequential losses caused by the unit is excluded even if the damage to the unit is accepted as a warranty claim.

14. Travel or International Use Statement

Since the performance of the Momcozy Wearable Breast Pump may be affected by the external environment, in view of the uncertainty and instability of the travel environment, please consult the airline for advice about carrying or using this pump on the plane.

The Momcozy Wearable Breast Pump can be internationally used, but it must be used and stored in the environment specified in this user manual, and please make sure the input power of your power adapter is AC 100~240V 50/60 Hz and output power is DC 5V==2A, and please make sure you have a converter to convert to the proper voltage of the target country. To ensure that the Momcozy Wearable Breast Pump is not affected during carrying, please check the following items before use to ensure it can operate normally:

- Check the Momcozy Wearable Breast Pump to ensure that it is free from damage and

- Before each usage, check the status of your Momcozy Wearable Breast Pump to ensure it can operate normally.

If there is any abnormality, please stop using it.

15. Manufacturer Information

Manufacturer: Shenzhen Root Innovation Technology Co., Ltd.

Address: Room 3301, Aerospace Innovation Building, No. 7013 Liuxian Avenue, Yuncheng Community, Xili Subdistrict, Nanshan District, Shenzhen, Guangdong, China, 518071

E-mail: support@momcozy.com

If assistance in setting up, using, or maintaining the device when needed or to report unexpected operation or events, please contact us.

Made in China



16. Important Safety Information

- This appliance can be used persons with reduced physical, sensory or mental capabilities or lack of experience and knowledge if they have been given supervision or instruction concerning use of the appliance in a safe way and understand the hazards involved;
- Children shall not play with the appliance;
- Cleaning and user maintenance shall not be made by children without supervision;
- Disposal of a battery into fire or a hot oven, or mechanically crushing or cutting of a battery, that can result in an explosion;
- Leaving a battery in an extremely high temperature surrounding environment that can result in an explosion or the leakage of flammable liquid or gas;
- A battery subjected to extremely low air pressure that may result in an explosion or the leakage of flammable liquid or gas.

momcozy

Evolving for Mom and Cozy