
momcozy

NS005

Silver Nipple Protective Cups

User Manual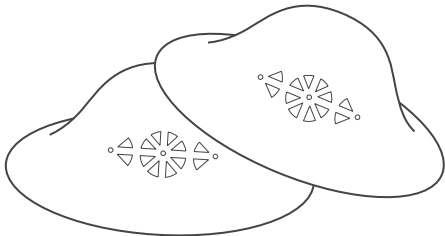

EN

DE

FR

IT

ES

BL

CS

DA

FI

NL

NO

PL

PT

SV

Before using this product, please read this user manual carefully and keep it safe for future reference.

Contents


EN	02
DE	06
FR	10
IT	14
ES	18
BL	22
CS	26
DA	30
FI	34
NL	38
NO	42
PL	46
PT	50
SV	52

Product Name	Silver Nipple Protective Cups
Product Material	999 Fine Silver
Product Size	Three sizes to choose from: Regular, XL, XXL
Suitable for	The Silver Nipple Protective Cup is primarily designed for addressing discomfort in breastfeeding women experiencing discomfort such as dryness, cracking, or pain in the nipples.
Intended use	The silver protective cup helps alleviate friction between clothing and dry, cracked, or painful nipples.


Product Size	Regular	XL	XXL
Width	1.77inches/ 45mm	2.16inches/ 55mm	2.56inches/ 65mm
Height	0.63inches/ 16mm	0.71inches/ 18mm	0.79inches/ 20mm
Applicable Areola Sizes	Smaller than 1.8 inches	Larger than 1.8 inches and smaller than 2.2 inches	Larger than 2.2 inches

How to use

- Wash and dry your hands thoroughly before using the shield.
- Clean the nipple area and the silver cups with wipes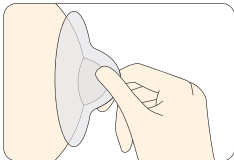

1. Gently place the silver nipple protective cup just on top of the nipple;

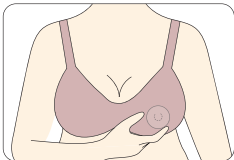2. After adjusting the position, wear your bra comfortably.

Cleaning and Storage

1. After daily use, rinse with water and dry thoroughly.

2. Since the cups are made of pure silver, long-term exposure to air, moisture, sweat, etc. will cause the surfaces of the cups to turn slightly black and yellow, which is normal.
3. If the silver nipple protective cup shows signs of discoloration, make a paste using baking soda with a small amount of water, rub it gently onto the caps, rinse thoroughly, and dry.
4. When not in use for a long time, clean and store them in a dry place.
5. Be careful when wearing it.
6. The inner walls of the cups are very thin.
7. Strong impact may cause slight deformation of the products.

If you have any questions, please contact us by email.

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Share Info GmbH
Shenzhen Root Innovation Technology Co., Ltd.