

正面

130 mm

对折线

Cleaning & Sterilization

***Non-washable parts**

Pump Motor
Tubing

***Washable parts**

Flange
Valve
Milk Collector

Cleaning and Sterilizing Washable Parts

1. Clean with soapy water(40-60 °C) for 3-5 minutes.

2. Use boiling water/steam/sterilization pot for sterilization. 100 °C (212 °F) for 5-10 minutes is required.

* It is not recommended to sterilize pump parts in a microwave or UV sterilizer. High temperature(> 110 °C) will shorten their service life and cause safety hazards.

Tips

1) The expected service life of the washable parts is 1-3 months. It is recommended to replace the washable parts after long-term storage or before pumping for your next child.

2) The breast pump will shut off automatically after pumping for 30 minutes.

3) If irritation or discomfort occurs, use your finger to break the seal between the flange and the breast tissue, discontinue use and see a healthcare provider.

momCOZY

V1 Pro Breast Pumps Hospital Grade

Turn on/off: Long press(2secs)
Unlock&The keypad lights off/on: Short Press
if there is no operation for 30 seconds, the lock screen will turn on. Short Press to unlock.

3 Modes Selection:
● Stimulation Mode
● Expression Mode
● Mixed Mode

Cycle Selection: H/M/L
Pause/Continue pumping

01~15 Suction Level: Short press - / + button to adjust the vacuum
Timer: Short Press & - / + button to adjust

If you have any questions, please contact us by email. support@momcozy.com

背面

Installation guidelines (please refer to the user manual for further details)

*Make sure your hands are clean before installation.

1 Attach the diaphragm with the milk collector and secure the lid to form a tight seal.

2 Gently push the valve into the milk collector. Make sure there are no gaps.

For your convenience, please refer to the operation guide video for further solutions. Scan the QR code above to access the operation guide video.

If you have any questions, please contact us by email. support@momcozy.com

3 First align the milk collector with the flange top, then press around to ensure a tight connection.

4 Connect the long end of the tubing to the pump motor.

Pouring Milk

Make sure to hold the milk collector in the correct direction when pouring milk. Then pour the milk out of the flange tunnel.

180 mm