


正面

130 mm

对折线

How to prevent milk leakage & Pumping tips for more breast milk

- 1 Make sure the pump is well assembled.
- 2 Bend forward to make sure your nipple is centered in the flange tunnel. The breast pump should be firmly against your breast.

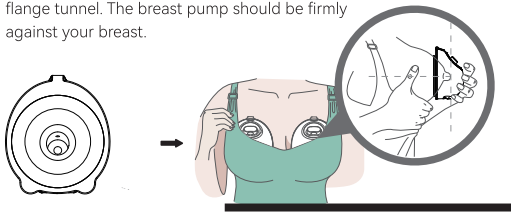

- 3 We recommend users wear a full-coverage nursing bra to form a tight seal between their breasts and flange.
- 4 Lean slightly forward to remove the pump from the breast after your pumping session.

Tips


- 1) The expected service life of the washable parts is 1-3 months. It is recommended to replace the washable parts after long-term storage or before pumping for your next child.
- 2) The breast pump will shut off automatically after pumping for 30 minutes.
- 3) If irritation or discomfort occurs, use your finger to break the seal between the flange and the breast tissue, discontinue use and see a healthcare provider.

If you have any questions, please contact us by email.  
support@momcozy.com

4

momcozy  
Wearable Breast Pump


Installation guidelines (please refer to the user manual for further details)  
\*Make sure your hands are clean before installation.

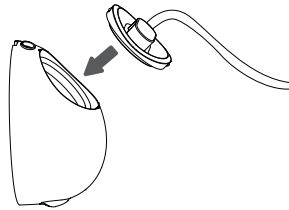

- 1 Attach the milk collector and secure the lid to form a tight seal.



- 2 Gently push the valve into the milk collector. Make sure there are no gaps.


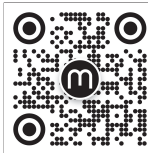


For your convenience, please refer to the operation guide video for further solutions.  
Scan the QR code above to access the operation guide video.


1

背面

- 3 First align the milk collector with the flange top, then press around to ensure a tight connection.

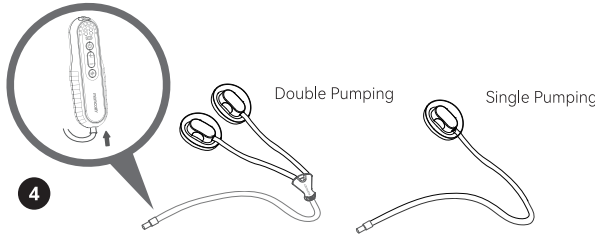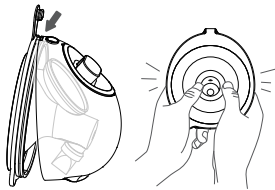

- 4 Connect the long end of the tubing to the pump motor.

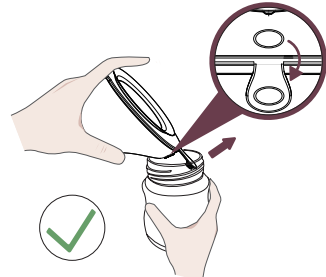

Pouring Milk

Make sure to hold the milk collector in the correct direction when pouring milk. Then pour the milk out of the flange tunnel.


If you have any questions, please contact us by email.  
support@momcozy.com

2

Cleaning & Sterilization

\*Non-washable parts

Tubing  
Pump Motor

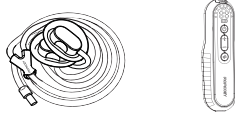

\*Washable parts



Flange



Valve




Milk Collector

Cleaning and Sterilizing Washable Parts  
1. Clean with soapy water(40-60 ° C) for 3-5 minutes.  
2. Use boiling water/steam/sterilization pot for sterilization. 100 ° C (212 ° F) for 5-10 minutes is required.

\* It is not recommended to sterilize pump parts in a microwave or UV sterilizer. High temperature( > 110 ° C) will shorten their service life and cause safety hazards.

Control panel

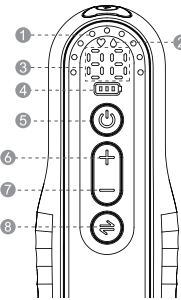

- 1 Suction Level Indicator
- 2 Mode Indicator
- 3 Running Time / Suction Level Display
- 4 Battery Display
- 5 On/Off
- 6 Increase
- 7 Decrease
- 8 Mode Selection

\* Long press and hold to turn On / Off;  
\* Short press to pause / continue pumping session.  
\* Stimulation Expression Mixed

3

180 mm