


momcozy

Lactation Massager

User manual

A1

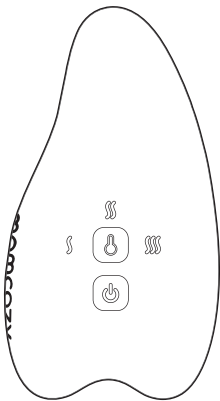

Read this user manual carefully before using this device and keep this manual for future reference.

Contents

WARNING	01
OVERVIEW	02
PRODUCT FEATURE	03
USAGE INSTRUCTIONS	05
WAYS TO USE	06
PACKAGE INCLUDED	07
SPECIFICATIONS	08
FCC COMPLIANCE STATEMENT	09
DECLARATION OF CONFORMITY	09
IMPORTANT SAFETY INFORMATION	09
DISPOSAL	10

WARNING

- DO NOT ALLOW ANYONE UNFAMILIAR WITH THIS PRODUCT TO USE IT.
- KEEP OUT OF REACH OF CHILDREN.
- THIS IS NOT A TOY.
- DO NOT DAMAGE THE CORD OR PLUG OR USE WHEN THE CORD OR PLUG IS DAMAGED AND/OR INSERTION TO A POWER SOURCE IS LOOSE. FAILURE TO OBSERVE THIS PRECAUTION CAN RESULT IN ELECTRIC SHOCK, SHORT CIRCUIT OR FIRE.
- DO NOT USE THIS UNIT FOR ANY PURPOSE OTHER THAN DESCRIBED IN THIS MANUAL OR PACKAGING.
- DO NOT OVERLOAD THE POWER SOURCE.
- DO NOT USE IF UNIT IS DAMAGED.
- ONLY USE THE USB CABLE THAT COMES WITH THE UNIT.
- ALWAYS INSERT PLUG SECURELY INTO THE POWER SOURCE.
- ENSURE THE MASSAGER IS DRY BEFORE CHARGING.
- KEEP AWAY FROM WATER WHEN CHARGING.

- REGULARLY CLEAN THE PLUG SO DUST OR SEDIMENT DOESN'T FORM ON IT.
- UNPLUG THE CORD FROM POWER SOURCE BEFORE CLEANING.
- DO NOT TOUCH PLUG WITH WET HANDS.
- DO NOT FULLY IMMERSE THE UNIT IN WATER.
- IF YOU ARE USING ANY ACTIVE IMPLANTABLE DEVICES THAT MAY BE AFFECTED BY MAGNETIC FIELDS (SUCH AS CARDIAC PACEMAKER AND IMPLANTABLE CARDIOVERTER DEFIBRILLATOR), IT IS RECOMMENDED THAT YOU CONSULT A MEDICAL PROFESSIONAL BEFORE USING MOMCOZY LACTATION MASSAGER.
- DO NOT BOIL, BAKE, MICROWAVE THE MASSAGER .
- DO NOT SHARE THIS PERSONAL OBJECT WITH OTHERS DUE TO HYGIENE CONCERNS.

OVERVIEW

Breastfeeding can be a daunting challenge for nursing moms, so we created the momcozy lactation massager to give you the most comfortable experience. This kneading lactation massager is designed to provide comfort and relaxation during breastfeeding, helping to

ease tension and promote a soothing experience. Moreover, it speeds up milk letdown and effectively empties the breast.

PRODUCT FEATURE

- **MULTIPLE MODES & INTENSITY SETTINGS:**

Equipped with multiple vibration modes and intensity the momcozy lactation massager helps nursing mothers ease discomfort and support healthy milk flow. Additionally, our massager will effectively ease discomfort from breast pressure and heaviness.


- **Adjustable Heating Levels:** Relieve tension and support healthy milk flow, while encouraging milk let-down during breastfeeding.

- **MAGNETIC USB RECHARGEABLE:** The magnetic charging design allows for a faster and simpler charge. It comes with a pouch bag for on-the-go moments.

- **DESIGNED FOR COMFORT:** Our breast massager was designed with the goal of comfort in mind, therefore it is not surprising that our massager is recommended by lactation consultants to relieve nursing mothers from common breastfeeding problems.

- **IPX7 Waterproof:** Our waterproof design allows for use in the shower, giving nursing moms the freedom to use our product whenever it's needed.

- **Smooth Milk Flow:** The Momcozy breast massager successfully prepares and stimulates the breast for feeding, so that you can have a successfully milk output.
- **Faster Milk Letdown:** Gentle use of our breast massager can help create a stronger letdown reflex by simulating the nerves in the breast and the nipple.
- **Targeted Massage:** Multiple contact points for targeted treatment.

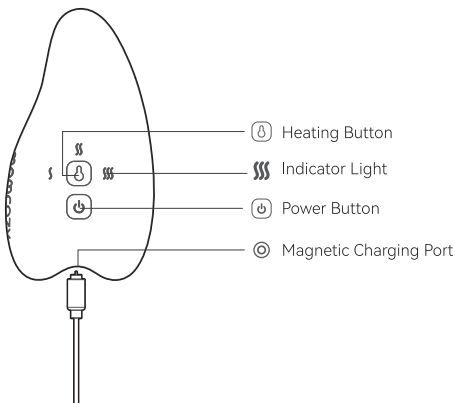

USAGE INSTRUCTIONS

DO NOT USE WHILE CHARGING

Power Button:

1. Turn the massager ON by holding down the Power Button “” for 2 seconds.
2. Short press the Power Button “” to change the vibration intensity and vibration modes.
3. Turn the massager OFF by holding down the Power Button “” for 2 seconds.

Heating Button

1. Turn the Heating function “” ON by holding down the Heating Button for 2 seconds. The heating mode will turn off automatically after a 30-minute session.
2. Short press the Heating “” Button to cycle through 3 temp settings.
3. Turn the Heating function “” OFF by holding down the Heating Button for 2 seconds.

Magnetic charging

1. One-handed operation and strong magnetic adsorption.
2. Plug the USB into a computer or USB adaptor.

3. The indicator light will slowly flash white light while the massager is charging.
4. The massager is fully charged when the white light remains solid.
5. It takes approximately 3 hours to get the device fully charged. Recharge the massager as needed.

Cleaning and disinfection:

1. Before and after each use, be sure to clean the massage device with mild soap and warm water or wipe it with 75% medical alcohol for 5 minutes. Allow it to dry completely and avoid direct sunlight.
2. When cleaning the product, do not immerse it in water.
3. Please unplug the charging cable before cleaning.
4. Store in a dry and dust-free place.

WAYS TO USE



Use the warming pad and rub breast towards the nipple.



Use the warming pad and rub breast towards the nipple.



Gently massage breast using the narrow part of massager.



Use the narrow tip of the massager and apply light pressure to the clogged area.




Gently massage breast with the curved edge.



Place massager on different parts of breast to encourage letdown and increase milk flow.


PACKAGE INCLUDED

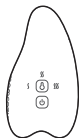

Lactation
Massager



USB Magnetic
Charging Cable

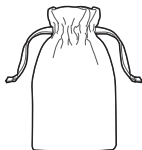

Portable
Storage Bag

SPECIFICATIONS

Operating and Storage Conditions:

Operating Conditions	Operating Temperature: -20~60°C Relative Humidity: 45~75% moisture
Storage&Transportation conditions	Storage Temperature: -20~30°C

Use time: 1.5 hours (working time will vary according to different temperature and vibration settings during use.)

Dimensions: 115mm X 61.5mm X 27.5mm

Vibration Mode: 6

Battery Capacity: 800 mAh

Battery Type: Rechargeable Lithium Polymer Battery

Input: DC 5V = 1A

Charging Time: Approx. 3 hours

Waterproof: IPX7

Levels of Temperature:

Low Temp.: 39 °C / 102.2 °F

Mid Temp.: 42 °C / 107.6 °F

High Temp.: 45 °C / 113 °F

FCC COMPLIANCE STATEMENT

This device complies with part 15 of the FCC Rules. Operation is subject to the following two conditions: (1) this device may not cause harmful interference, and (2) this device must accept any interference received, including interference that may cause undesired operation.



IMPORTANT SAFETY INFORMATION

- This appliance can be used persons with reduced physical, sensory or mental capabilities or lack of experience and knowledge if they have been given supervision or instruction concerning use of the appliance in a safe way and understand the hazards involved;
- Children shall not play with the appliance;
- Cleaning and user maintenance shall not be made by children without supervision;
- Disposal of a battery into fire or a hot oven, or mechanically crushing or cutting of a battery, that can result in an explosion;
- Leaving a battery in an extremely high temperature surrounding environment that can result in an explosion or the leakage of flammable liquid or gas;
- A battery subjected to extremely low air pressure that may result in an explosion or the leakage of flammable liquid or gas.

DISPOSAL



At the end of the product life cycle, do not throw this product into the normal household garbage, but bring it to a collection point for the recycling of electronic equipment and battery. Waste Electrical and Electronic Equipment may have potentially harmful effects on the environment. Improper disposal may lead to the accumulation of harmful toxins in the air, water, and soil, which is harmful to human health. you have obligation to dispose of the device correctly. Consult your municipal authority for information about disposal.

SYMBOLS	DESCRIPTION
	“WEEE(Waste Electrical and Electronic Equipment)”. The waste products should be handled legally.
 MEHRWEG	Indicates that packaging is recyclable.
IPX7	The product is rated IPX7, meaning it is protected against temporary immersion in water up to 1 meter for 30 minutes.
	Caution
FC	Indicates that the product complies with the electromagnetic interference standards set by the U.S. Federal Communications Commission (FCC).
	TYPE BF APPLIED PART
	CLAss II equipment
	Consult instructions for use or consult electronic

Manufactured for: Hong Kong Lute Technology Co., Limited
Address: Room 02, 21/F, Hip Kwan Commercial Building, 38 Pitt
Street, Yau Ma Tei, Kowloon, Hongkong

Email: support@momcozy.com

MADE IN CHINA.



momcozy

Evolving for Mom and Cozy