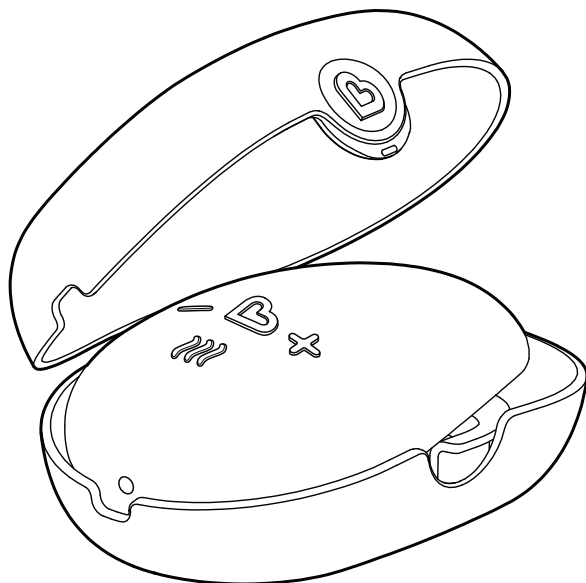
Owner's Guide

Manual del propietario

Warm Touch Lactation Massager

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---

KEEP FOR FUTURE REFERENCE

Please read the instructions carefully before using this product.

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CCD-0322-000 REV 1

|                                                                                                             |    |
|-------------------------------------------------------------------------------------------------------------|----|
|  <b>IMPORTANT SAFEGUARDS</b> |    |
| Important Safeguards.....                                                                                   | 4  |
|  <b>INCLUDED PARTS</b>       |    |
| Included Parts.....                                                                                         | 5  |
|  <b>SUGGESTED USE</b>        |    |
| Suggested Use .....                                                                                         | 6  |
|  <b>HOW TO USE</b>           |    |
| Charging the Battery.....                                                                                   | 8  |
| In-Case Charging.....                                                                                       | 8  |
| Using the Vibration Function .....                                                                          | 9  |
| Using the Heat Function .....                                                                               | 10 |
| How to Store .....                                                                                          | 11 |
| How to Clean.....                                                                                           | 12 |
| Troubleshooting .....                                                                                       | 12 |
| Technical Specifications.....                                                                               | 12 |
|  <b>CUSTOMER SERVICE</b>     |    |
| One-Year Limited Warranty .....                                                                             | 13 |
| Contact Us .....                                                                                            | 13 |

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# IMPORTANT SAFEGUARDS

## WARNING:

- Use only as directed.
- For external use only.
- Ensure the product is completely dry before charging.  
Keep away from water while charging.
- Product is not a toy. Keep away from children.
- Discontinue use if skin irritation occurs and consult your physician.

# SAVE THESE INSTRUCTIONS

## FCC INFORMATION

This device complies with part 15 of the FCC Rules. Operation is subject to the following two conditions: (1) this device may not cause harmful interference, and (2) this device must accept any interference received, including interference that may cause undesired operation.

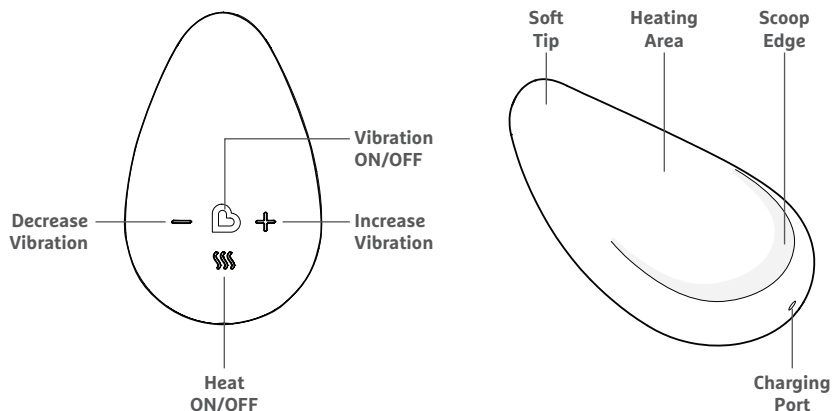
**⚠ WARNING:** To reduce the risk of electrocution: Changes or modifications to this unit not expressly approved by the party responsible for compliance could void the user's authority to operate the equipment.

**NOTE:** This equipment has been tested and found to comply with the limits for a Class B digital device, pursuant to part 15 of the FCC Rules. These limits are designed to provide reasonable protection against harmful interference in a residential installation. This equipment generates, uses and can radiate radio frequency energy and if not installed and used in accordance with the instructions, may cause harmful interference to radio communications. However, there is no guarantee that interference will not occur in a particular installation. If this equipment does cause harmful interference to radio or television reception, which can be determined by turning the equipment off and on, the user is encouraged to try to connect the interference by one or more of the following measures:


- Reorient or relocate the receiving antenna.
- Increase the separation between the equipment and receiver.
- Connect the equipment into an outlet on a circuit different from that to which the receiver is connected.
- Consult the dealer or an experienced radio/TV technician for help.

## INCLUDED PARTS

### Massager

### Travel Case
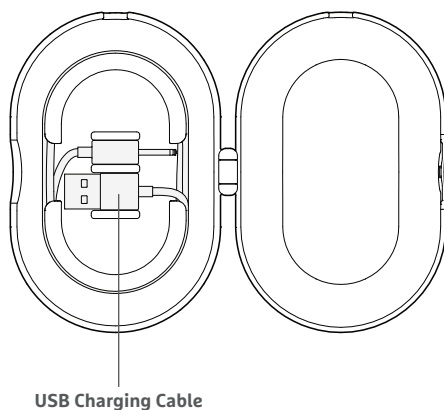
### SUGGESTED USE

Blocked milk ducts can be painful for nursing mothers. Our lactation massager lets you apply direct vibration and heat to the breast area to help loosen clogs, soothe pain, reduce engorgement and improve milk flow.

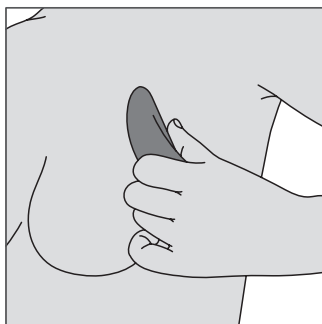
**VIBRATION:** Choose from three vibration levels to massage and relax the breast area.

**HEAT:** Use the heat feature to warm the breast and help get your milk flowing.


**TIP:** Try using during a warm shower to help soothe and warm the breast area during use.

**NOTE:** Do not submerge in water.


Apply gentle pressure using the top side to help soften the breast area.

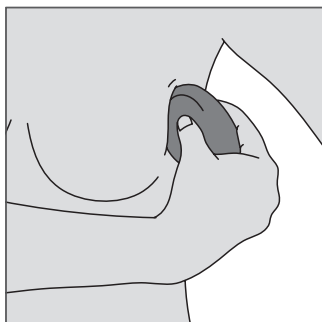
Use the scoop to get milk flowing and help empty the breast.

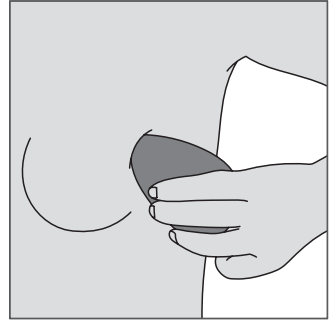
Apply direct pressure using the soft tip to help work out clogs.

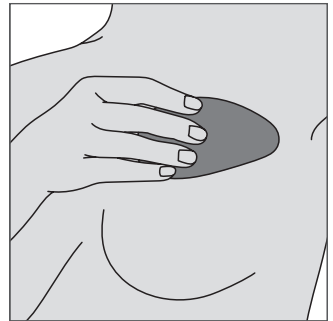Warm the breast area with the heated underside.

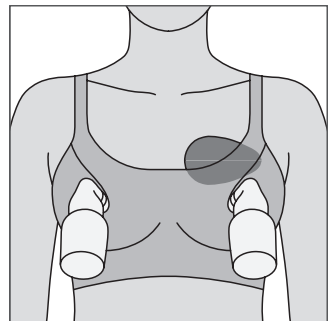Place under pumping bra for warmth and vibration while pumping.

Every mom is different. Patience and persistence are key to relieving clogged milk ducts. If the problem persists for more than 24-48 hours, please consult your healthcare provider.


## CHARGING THE BATTERY

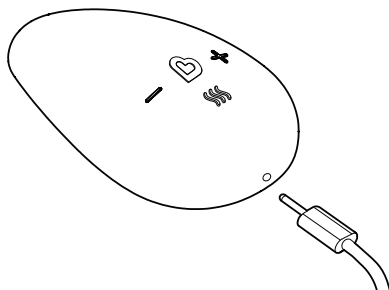Please charge the battery fully before first use.

Insert the USB charging cable into a power adapter or USB power bank (not included).

Insert the pin plug of the charging cable into the charging port at the base of the massager.

Product cannot be used while charging.


| STATUS            |  INDICATOR LIGHT |
|-------------------|---------------------------------------------------------------------------------------------------|
| Battery charging  | PULSING LIGHT                                                                                     |
| Charging complete | STEADY LIGHT                                                                                      |
| Battery low       | BLINKS RED (during use)                                                                           |
| Battery empty     | PULSES RED 3 TIMES                                                                                |

**NOTE:** The heating function consumes a lot of power. You can optimize battery life by using the vibration function alone for some sessions.

When fully charged, the product should last for approximately three to four 20-minute sessions depending on vibration setting and heat usage. We recommend charging after each use when possible to ensure a full charge for your next session.

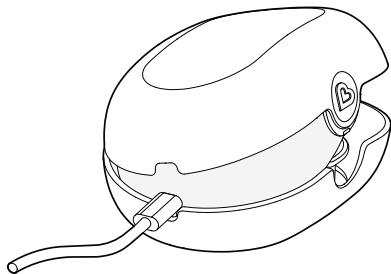**IN-CASE CHARGING:** The massager can be recharged while stored in its case. Simply route the charging cable through the opening, then close the case.

## USING THE VIBRATION FUNCTION

You can choose from three vibration levels to get the exact intensity you need.

**ACTIVATING VIBRATION:** Press and hold the  power button for 1 second to turn on vibration.

**SWITCHING MODES:** Press  again to switch between continuous and pulsing vibration.

**SWITCHING LEVELS:** Press  or  to increase or decrease the vibration level. There are 4 levels. When powered on, vibration starts at level 2.

| LEVEL   | VIBRATION                                                                                                      |
|---------|----------------------------------------------------------------------------------------------------------------|
| Level 0 | No vibration. Set to 0 if you would like to stop vibration and use only heat at any point during the session.. |
| Level 1 | Low vibration                                                                                                  |
| Level 2 | Standard vibration. When powered on, vibration always starts at level 2.                                       |
| Level 3 | High vibration                                                                                                 |

**SHUTTING OFF:** Press and hold the  button for 1 second to turn vibration off. If the heating function is also in use, both functions will turn off.

**AUTO SHUT-OFF:** All functions will shut off automatically after 20 minutes of use.

## USING THE HEAT FUNCTION

The heating element covers the bottom surface area and underside of the tip, so you can choose between general and targeted warmth.

**ACTIVATING HEAT:** The heat can be used independently from the vibration and must be turned on separately from the main power.

Press and hold the  button for 1 second to turn on heat. The button's LED will pulse during warm up.

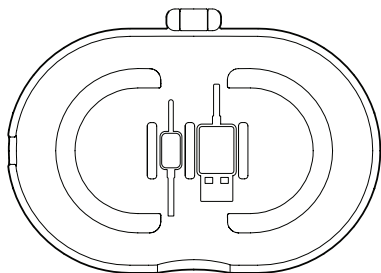
| STATUS     |  INDICATOR LIGHT |
|------------|---------------------------------------------------------------------------------------------------|
| Warming up | PULSES WHITE                                                                                      |
| Heat ready | STEADY WHITE                                                                                      |

**SHUTTING OFF:** Press and hold the  button for 1 second to turn off heat. This action will NOT turn off vibration.


## HOW TO STORE

Store the massager in the case when not in use.


Place the USB plug onto its matching icon in the storage case. Push down until the plug locks into position.

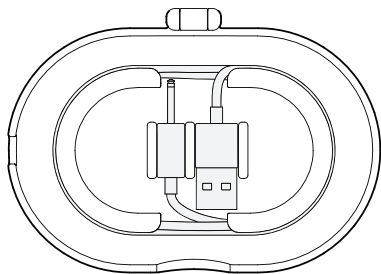Wrap the charging cable around the cable tray, then place the pin plug onto its matching icon. Push down to lock into position.

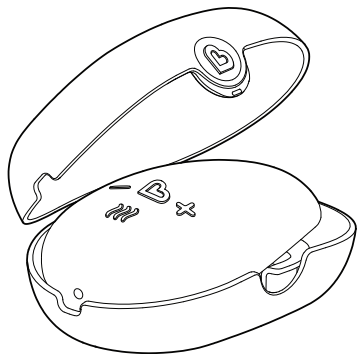Place the massager into the case and close the lid.

## HOW TO CLEAN

Clean the massager after each use. Wipe clean with a damp cloth, soapy water, or sanitizing cleansing wipe. Do not submerge in water. Air dry and avoid direct sunlight.

## TROUBLESHOOTING

### **MASSAGER WILL NOT TURN ON**

Plug in to power source via USB and fully charge the battery.

### **MASSAGER WILL NOT CHARGE**

Push the charging cable fully in the massager until the indicator light turns on. Ensure that USB end is plugged into charging source (not included).

## TECHNICAL SPECIFICATIONS

**Product Name:** Warm Touch Lactation Massager

**Model:** MK0839

**Power Input:** 5VDC

**Lithium-ion Battery Capacity:** 500mAh

**Operating Temperature:** 32~104 °F (0~40 °C)

**Storage Temperature:** -4~131 °F (-20~55 °C)

**Battery Life per Each Full Charge:** > 60 minutes

**Dimensions:** 3.9 in. (99mm) x 2.5 in. (63mm) x 1 in. (25mm)

**Net Weight:** 3 oz. (85 g)

## ONE-YEAR LIMITED WARRANTY

Warm Touch Lactation Massager includes a 1-year limited warranty against defects from the date of purchase. In the event of a malfunction or defect during the in-warranty period, Munchkin, Inc. at its option, will replace or repair the product under warranty. Upon receipt of the product and proof of purchase with a valid date of purchase, the defective part will be immediately repaired or replaced and returned to you free of charge (USA only). Please send defective unit, proof of purchase and a description of the problem you have encountered to:

Munchkin, Inc.  
7835 Gloria Avenue  
Los Angeles, CA 91406, USA  
Attn: Warm Touch Lactation Massager Warranty

**NOTE:** This warranty does not cover misuse, abuse, abnormal wear & tear, negligence, environmental exposure, alteration or accident. Any alteration or removal of the product date code invalidates all claims against the manufacturer.

## CONTACT US

If you have any questions or wish to provide comments about this product, feel free to contact our Customer Satisfaction team at **1-800-344-BABY** (2229) (M-TH 8:00am- 5:00pm PST, F 8:00am-1:00pm PST) or visit our website at **[www.munchkin.com](http://www.munchkin.com)**.

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© 2024 MUNCHKIN, INC., A WHY BRANDS COMPANY. 7835 GLORIA AVE., LOS ANGELES, CA 91406.
MUNCHKIN BABY CANADA LTD., 3-8460 MOUNT PLEASANT WAY, MILTON, ON L9T 8W7. MADE IN