
Owner's Guide

Manual del propietario

Munchkin Mini Thermometer

Minitermómetro Munchkin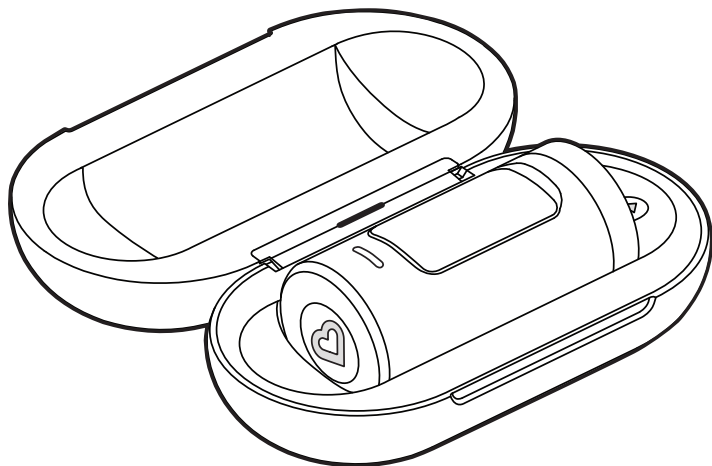

KEEP FOR FUTURE REFERENCE

Please read the instructions carefully before using this product.

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 IMPORTANT SAFEGUARDS	
Important Safeguards.....	5
 INCLUDED PARTS	
Included Parts.....	7
 HOW TO USE	
Before First Use.....	8
Before Every Use.....	8
Display Guide.....	8
To Measure Temperature.....	9
Switch Between Fahrenheit and Celsius.....	10
View Previous Readings.....	10
How to Replace the Battery.....	11
Troubleshooting.....	13
Meaning of Symbols.....	14
Regulatory Information.....	15
Product Specifications.....	19
Technical Specifications.....	20
 CUSTOMER SERVICE	
One-Year Limited Warranty.....	21
Contact Us.....	21

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IMPORTANT SAFEGUARDS

WARNING:

- For adult use only. Keep this product away from children.
- Intended to be used by consumers in household environment. Do not make medical judgements based on the thermometer reading. Seek medical attention and follow instructions from your health care provider.
- Do not touch or blow on the infrared sensor. Keep the sensor clean. A dirty sensor might lead to incorrect results.
- To clean the sensor and case regularly, wipe gently with cotton balls or a soft cloth dipped in isopropyl (rubbing) alcohol. Do not use toilet paper or tissue, which might scratch the infrared sensors.
- Do not measure temperature during or after a hot bath, long exposure to the sun, cold weather, or strenuous exercise, otherwise the reading might be affected.
- At the end of the service life of this product, do not discard it in the garbage. Please discard used batteries according to national and local regulations.
- Do not disassemble or repair your thermometer.
- Do not place or use the thermometer near water, high temperature, humidity, direct sunlight, dust, or corrosion.
- Do not take measurements during a sudden change of ambient temperature or in harsh environments.
- Do not measure temperature on any body part that has inflammation, trauma, or postoperative local lesions.
- Do not drop the instrument. Protect it from severe impact and shock. Do not twist the instrument or the measuring sensor.
- Do not store this thermometer in temperature extremes below -25 °C or over 55 °C (below -13 °F or over 131 °F) or in excessive humidity (above 95% non-condensing relative humidity).
- The accuracy of IR thermometers can differ from contact thermometers. Factors affecting accuracy include operator technique, anatomical variations and subject cooperation. Maintain proper distance and stability during measurements. Consider variations in skin color, thickness, and surface conditions. Ensure subject cooperation and steadiness.

SAVE THESE INSTRUCTIONS

⚠ **BATTERY WARNING:**

⚠ **WARNING**

INGESTION HAZARD: This product contains a button cell or coin battery. **DEATH** or serious injury can occur if ingested. A swallowed button cell or coin battery can cause **Internal Chemical Burns** in as little as **2 hours**. **KEEP** new and used batteries **OUT OF REACH OF CHILDREN**. **Seek immediate medical attention** if a battery is suspected to be swallowed or inserted inside any part of the body.




WARNING: Contains coin/button battery

This product contains a CR2032, 3VDC coin battery. A coin battery can cause serious internal chemical burns if swallowed.

- Remove and immediately recycle or dispose of used batteries according to local regulations and keep away from children. Do NOT dispose of batteries in household trash or incinerate. Even used batteries may cause severe injury or death.
- Call a local poison control center for treatment information.
- Non-rechargeable batteries are not to be recharged.
- Do not force discharge, recharge, disassemble, heat above 176°F (80°C) or incinerate. Doing so may result in injury due to venting, leakage or explosion resulting in chemical burns.
- Ensure the batteries are installed correctly according to polarity (+ and -).
- Do not mix old and new batteries, different brands or types of batteries, such as alkaline, carbon-zinc, or rechargeable batteries.
- Remove and immediately recycle or dispose of batteries from equipment not used for an extended period of time according to local regulations.
- Always completely secure the battery compartment. If the battery compartment does not close securely, stop using the product, remove the batteries, and keep them away from children.
- Always remove exhausted batteries from product.
- Rechargeable batteries are to be removed before being charged.
- Rechargeable batteries are only to be charged under adult supervision.
- Do not short circuit battery terminals.
- Do not submerge in water or other liquids.
- Only batteries of the same or equivalent type as recommended are to be used.
- Use caution when opening battery compartment as some batteries may leak, explode or catch on fire.


INCLUDED PARTS

Please check that all parts are included before first use. If any parts are missing or damaged, please contact our Customer Satisfaction Team at **1-800-344-2229**.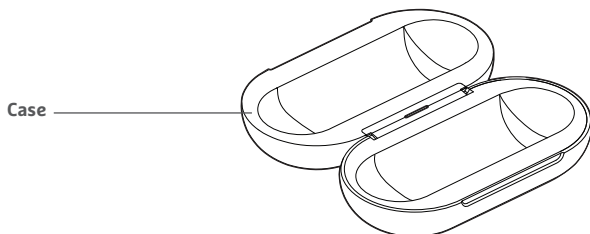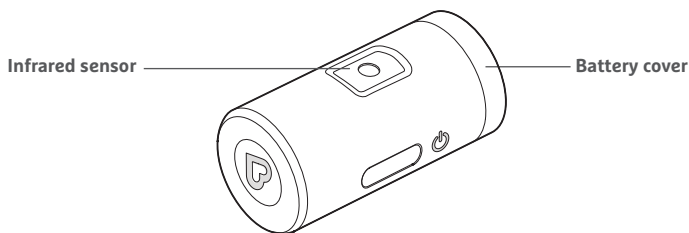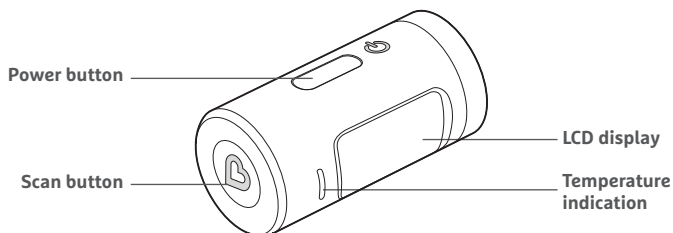

BEFORE FIRST USE

Remove and discard the LCD display protection film and battery-saver.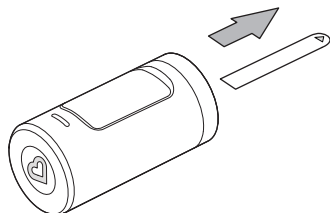


BEFORE EVERY USE

Check that the infrared sensor is clean and free from damage.

Use in an environment that's between 59°F (15°C) to 95°F (35°C). For best results, ensure that the thermometer and the person to be measured are both in this environment for at least 30 minutes before use.

DISPLAY GUIDE


- ① Temperature reading
- ② Scale (Fahrenheit/Celsius)
- ③ Low battery indicator
- ④ Measure toward forehead
- ⑤ Temperature indication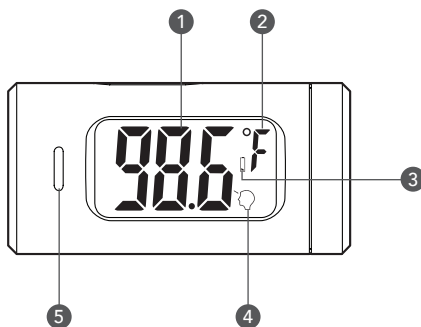


COLOR	MEASUREMENT
Green (Normal)	95.9-99.5 °F (35.5 °C - 37.5 °C)
Red (Elevated / High)	> 99.5 °F (37.5 °C)
Blue (Low)	< 95.9 °F (35.5 °C)

TO MEASURE TEMPERATURE


1. Press the power button next to .
The display screen turns on. After 3 seconds, the blue backlight flashes when the thermometer is ready.
2. Place the sensor near the forehead.
For best results, the distance from the skin should be less than 3cm (1inch).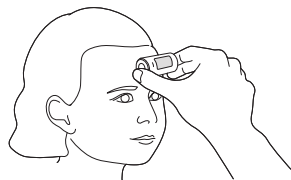

3. Press the side scan button to begin the temperature reading.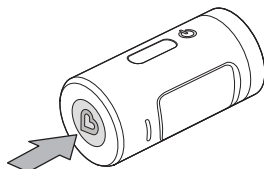

4. After 1 second, a beeping sound indicates that the reading is complete. The measured temperature appears on the display.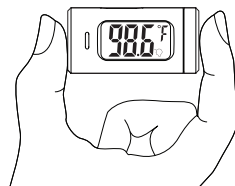


5. Press and hold the power button for 3 seconds to turn off the device. After 30 seconds of inactivity, the power shuts off automatically.

NOTE: To take another temperature reading, wait until the blue backlight flashes. To ensure accurate measurements, please avoid hair covering the forehead. If there are fringes, uncover them and wait for 3 minutes before taking the measurement. Use in an environment that's between 59°F (15°C) to 95°F (35°C).



SWITCH BETWEEN FAHRENHEIT AND CELSIUS

1. While the power is on, double press the power button to switch between °C and °F.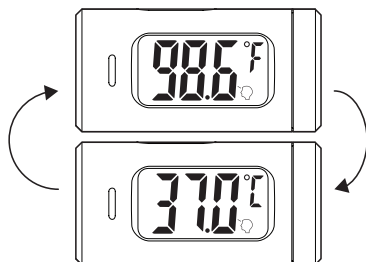


VIEW PREVIOUS READINGS

1. With the power off, press and hold the power button for 3 seconds.
2. Release the button when the LCD displays 3 dashes.
3. Press the power button to step through previous readings.
4. Press and hold the power button again for 3 seconds to turn the unit off.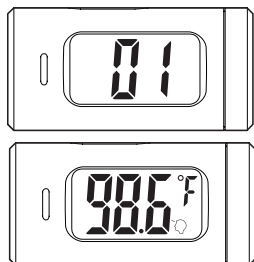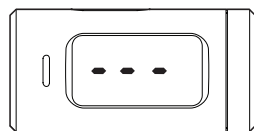


HOW TO REPLACE THE BATTERY


1. When the "low battery" symbol  appears on the display, replace the CR2032 coin battery.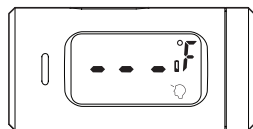


2. Use a #1 Philips-head screwdriver (included) to remove the battery cover.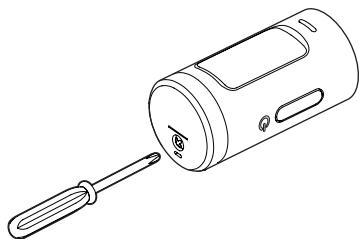


3. Pull the battery out using your fingers or with an insulated tool.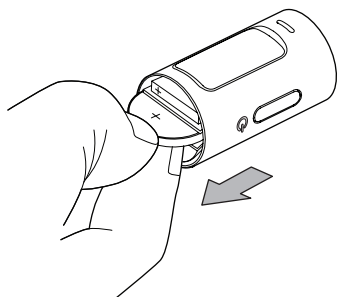

4. Insert a new battery into the compartment. Match the correct + and - polarity.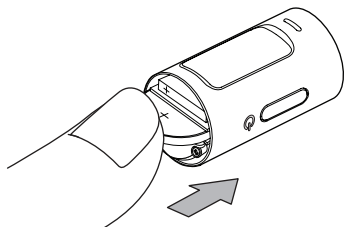

5. Screw the battery cover back on.

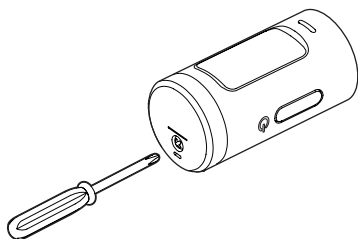6. Discard the used battery according to local environmental regulations.

TROUBLESHOOTING

1. If nothing shows on the display after turning the power on:

- a. Check the battery. If the battery is depleted, replace with a fresh CR2032 3VDC button cell battery.
- b. Check if the battery is inserted correctly. Pull the battery out and make sure battery polarity matches the battery compartment + and - polarity.
- c. If the internal circuit is damaged, the unit must be returned to Munchkin for after-sales service.

2. If the temperature reading is not functioning:

- a. Retry after 1 minute.
- b. Check if the sensor head is dirty. Use a cotton swab or soft cloth dipped in isopropyl alcohol to gently wipe the sensor.
- c. If the sensor is damaged or scratched, please contact Munchkin for repair.
- d. Measurement might be done incorrectly, please read the manual carefully.

3. If the display shows "HH" or "LL":

The system detected that the ambient temperature exceeds the normal operating range of the device. The measurement should be done in an environment between 59°F (15°C) to 95°F (35°C), and the device should be placed in such an environment for 30 minutes before use.

5. If the display shows "HI":

The target temperature is above the measurement limit of the device.

6. If the display shows "LO":

The target temperature is below the measurement limit of the device.

It might also indicate low voltage. Please replace the battery.

The functions of the thermometer may be adversely affected if the above circumstances occur.

MEANING OF SYMBOLS

	Device Manufacturer
	Follow Instruction for use
	Compliance with international requirements for protection from electric shock. (Type BF applied parts.)
	Temperature limits of device can be safely exposed.
	Atmospheric pressure limit the device can be safely exposed.
	Humidity limit the device can be safely exposed.

FCC WARNING

This device complies with part 15 of the FCC Rules. Operation is subject to the following two conditions: (1) This device may not cause harmful interference, and (2) This device must accept any interference received, including interference that may cause undesired operation.

WARNING: Changes or modifications to this unit not expressly approved by the party responsible for compliance could void the user's authority to operate the equipment.

NOTE: This equipment has been tested and found to comply with the limits for a Class B digital device, pursuant to part 15 of the FCC Rules. These limits are designed to provide reasonable protection against harmful interference in a residential installation. This equipment generates, uses and can radiate radio frequency energy and if not installed and used in accordance with the instructions, may cause harmful interference to radio communications. However, there is no guarantee that interference will not occur in a particular installation. If this equipment does cause harmful interference to radio or television reception, which can be determined by turning the equipment off and on, the user is encouraged to try to correct the interference by one or more of the following measures:

- Reorient or relocate the receiving antenna.
- Increase the separation between the equipment and receiver.
- Connect the equipment into an outlet on a circuit different from that to which the receiver is connected.
- Consult the dealer or an experienced radio/TV technician for help.

REGULATORY INFORMATION

THE THERMOTER WAS DESIGNED AND MANUFACTURED IN COMPLIANCE WITH THE FOLLOWING STANDARDS:

IEC 60601-1	Medical electrical equipment – Part 1: General requirements for basic safety and essential performance
IEC 60601-1-2	Medical electrical equipment – Part 1: General requirements for basic safety and essential performance – Collateral standard: Electromagnetic compatibility – Requirements and tests
IEC 80601-2-56	Medical electrical equipment Part 2-56: Particular requirements for basic safety and essential performance of clinical thermometers for body temperature measurement
ISO 15223-1	Medical devices--symbols to be used with medical device labels, labelling and information to be supplied -- Part1: General requirements.
IEC 60601-1-11	Medical - Home Healthcare Environment

This thermometer is an adjusted mode thermometer that provides an oral equivalent reading.

This infrared thermometer meets requirements established in ASTM Standard E 1965-98 (for the thermometer system). Full responsibility for the conformance of the product to the standard is assumed by Munchkin, Inc., 7835 Gloria Avenue, Van Nuys, CA 91406, USA.

ASTM laboratory accuracy requirements, for the thermometer only in the display range of 96.8 °F to 102.2 °F (36 °C to 39°C) for infrared thermometers is ± 0.5 °F (± 0.3 °C), whereas for mercury-in-glass and electronic thermometers, the requirement per ASTM Standards E 667-86 and E 1112-86 is ± 0.2 °F (± 0.1 °C).

WARNING:

- Use of this equipment adjacent to or stacked with other equipment should be avoided because it could result in improper operation. If such use is necessary, this equipment and other equipment should be observed to verify that they are operating normally.
- The use of accessories, transducers and cables other than those specified or provided by manufacturer of this equipment could result in increased electromagnetic emissions or decreased electromagnetic immunity of this equipment and result in improper operation.
- Portable RF communications equipment (including peripherals such as antenna cables and external antennas) should be used no closer than 30 cm (12 inches) to any part of the product. Otherwise, degradation of the performance of this equipment could result.
- If the use location is near (e.g. less than 1.5 km from) AM, FM or TV broadcast antennas, before using this equipment, it should be observed to verify that it is operating normally to assure that the equipment remains safe with regard to electromagnetic disturbances throughout the expected service life.

EMS COMPLIANCE TABLE

PHENOMENON	BASIC EMC STANDARD	IMMUNITY TEST LEVELS HOME HEALTHCARE ENVIRONMENT
Electrostatic Discharge	IEC 61000-4-2	±8kv contact ±2kv, ±4kv, ±8kv, ±15kv air
Radiated RF EM field	IEC 61000-4-3	10V/m 80MHz-2.7GHz 80%AM AT 1KHz
Proximity fields from RF wireless communications equipment	IEC 61000-4-3	Refer to table below
Rated power frequency magnetic fields	IEC 61000-4-8	30A/m 50Hz or 60Hz

ELECTROMAGNETIC COMPATIBILITY

The device complies with the IEC 60601-1-2 standard for electromagnetic compatibility. Please do not let thermometer become source of the disturbance, also try your best avoid source of the disturbance. Details and measurement and EMC data are shown on the next page. We reserve the right to make technical and design changes in the course of continuous product improvement

PHENOMENON: RF emissions


COMPLIANCE: CISPR 11 Group 1, Class B

ELECTROMAGNETIC ENVIRONMENT: Home healthcare environment



PROXIMITY FIELDS FROM RF WIRELESS COMMUNICATIONS EQUIPMENT

TEST FREQUENCY (MHz)	BAND (MHz)	IMMUNITY TEST LEVELS HOME HEALTHCARE ENVIRONMENT
385	380-390	Pulse modulation 18Hz, 27V/m
450	430-470	FM, ± 5 KHz deviation, 1KHz sine, 28V/m
710 745 780	704-787	Pulse modulation 217Hz, 9V/m
810 870 930	800-960	Pulse modulation 18Hz, 28V/m
1720 1845 1970	1700-1990	Pulse modulation 217Hz, 28V/m
2450	2400-2570	Pulse modulation 217Hz, 28V/m
5240 5500 5785	5100-5800	Pulse modulation 217Hz, 9V/m

PRODUCT SPECIFICATIONS

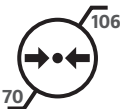Product name:	Munchkin Mini Thermometer
Model:	KFT-27(MKCA0892)
Display System:	1.02"(26mm)*0.51"(13mm) LCD display
LED Indication:	95.9-99.5°F (35.5°C - 37.5°C) display green; > 99.5°F (37.5°C) display red; < 95.9°F (35.5°C) display blue;
Apply for:	Forehead temperature
Accuracy:	95°F~107.6°F $\pm 0.4^{\circ}\text{F}$ (35°C~42°C $\pm 0.2^{\circ}\text{C}$); 89.6°F~94.8°F $\pm 0.5^{\circ}\text{F}$ (32°C~34.9°C $\pm 0.3^{\circ}\text{C}$); 107.8°F~109.2°F $\pm 0.5^{\circ}\text{F}$; (42.1°C~42.9°C) $\pm 0.3^{\circ}\text{C}$
Resolution:	0.1°F (0.1°C)
Temperature Measurement:	Infrared scan
Measurement Display:	32°C~42.9°C (89.6°F~109.2°F)
Memory Records:	20 sets
Time-out Feature:	30 seconds (No operation)
Working Temperature:	   Operating temperature range: 15°C~35°C (59°F~95°F); Operating Atmospheric pressure range: 70KPa~106KPa; Operating Humidity range: 15%RH~85%RH;

TECHNICAL SPECIFICATIONS

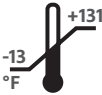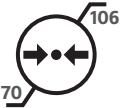Storage Temperature:	   Storage temperature range: -25°C~55°C (-13°F~131°F); Storage Atmospheric pressure range: 70KPa~106KPa; Storage Humidity range: 15%RH~95%RH;
Service Life:	5 years
Battery:	1 CR2032
Battery Life:	1200 Cycles
Weight:	Approximate 22.5g (including battery)
Dimension:	L 48.3mm x W 24.1mm x H 24.1mm (L 1.9" x W 0.95" x H 0.95")

NOTE: Included battery is for demo use only, for longer life, it is recommended to use fresh CR2032 battery.

KANGFU MEDICAL EQUIPMENT FACTORY
NO. 380 EAST NINGKANG ROAD, YUEQING CITY,
ZHEJIANG PROVINCE, CHINA

MUNCHKIN IS A REGISTERED TRADEMARK OF MUNCHKIN, INC.

ONE-YEAR LIMITED WARRANTY

Munchkin Mini Thermometer includes a 1-year limited warranty against defects from the date of purchase. In the event of a malfunction or defect during the in-warranty period, Munchkin, Inc. at its option, will replace or repair the product under warranty. Upon receipt of the product and proof of purchase with a valid date of purchase, the defective part will be immediately repaired or replaced and returned to you free of charge (USA only). Please send defective unit, proof of purchase and a description of the problem you have encountered to:

Munchkin, Inc.
7835 Gloria Avenue
Van Nuys, CA 91406, USA
Attn: Munchkin Mini Thermometer Warranty

NOTE: This warranty does not cover misuse, abuse, abnormal wear & tear, negligence, environmental exposure, alteration or accident. Any alteration or removal of the product date code invalidates all claims against the manufacturer.

CONTACT US

If you have any questions or wish to provide comments about this product, feel free to contact our Customer Satisfaction team at **1-800-344-BABY** (2229) (M-TH 8:00am- 5:00pm PST, F 8:00am-1:00pm PST) or visit our website at **www.munchkin.com**.

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