



Active XL **DUO**



INSTALLATION MANUAL | OG6400

Assembly instructions

The OG6400 Massage chair arrives in one box.



- 1

Open the box and remove all contents - one footrest, one power cord, one user manual & assembly instruction. Take the chair base out of the box.



- 2** Remove the packing materials and locate the two catch pins. Take off the catch pins with the Phillips screw driver and put them aside.



- 3** Move the footrest close to the chair. Unzip the zipper to locate the footrest wire connector and air hose connector. Align the two white dots and connect the footrest wire connector. Screw the cap tightly. Connect the air hose. Zip the zipper.



- 4** Attach the footrest onto the base pegs and install the two catch pins.



5

Zip the zipper between the seat pad and the footrest cushion. According to the requirements of the UL standard for safety (UL1647), the zipper pull must be removed and discarded.



6

Turn on the chair. Connect the chair to a power supply and turn on the power switch. Now your massage chair is ready to go!
Note: Place the remote control into the slot when not use.

