
RENPHO

USER MANUAL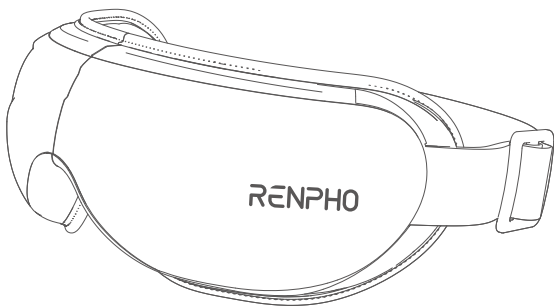

EYERIS 2

Model: R-G006

CONTENTS

• ENGLISH	01-11
-----------	-------

• DEUTSCH	12-23
-----------	-------

• FRANÇAIS	24-35
------------	-------

• ITALIANO	36-46
------------	-------

• ESPAÑOL	47-57
-----------	-------

Thank you for purchasing the RENPHO Eyeris 2! Please read this instruction manual carefully and use the product only as indicated. RENPHO accepts no responsibility for any cause of injuries due to misuse or negligence to follow the instructions outlined. Please keep this manual for future reference.

IMPORTANT SAFETY INSTRUCTIONS

READ ALL INSTRUCTIONS CAREFULLY BEFORE USE.

1. This device is not intended for use by persons (including children) with reduced physical, sensory or mental capabilities, or lack of experience and knowledge, unless they have been given supervision or instruction concerning use of the device by a person responsible for their safety.
2. Children should be supervised to ensure that they do not play with the device.
3. The device is intended for household use only and not intended for medical use. It should not be used in treatment or alleviation of disease or abnormal physical state.
4. Cleaning and maintenance should not be conducted by children.
5. Please consult your doctor before use if you fit into any of the following user groups:
 - Persons who had undergone eye surgery or eye conditions such as glaucoma, cataract and retinal detachment
 - The elderly
 - Pregnant or postpartum women
 - Persons who are allergic to leather material
 - Persons with skin infections, sensitivity, redness and/or burns on the skin
 - Persons with any swelling or wound around the eye area
 - Persons using a pacemaker or other implantable medical devices
 - Persons with a heart condition, malignant tumors, or other serious health conditions
 - Persons undergoing medical treatment
 - Persons with high or abnormal blood pressure
6. If you experience any discomfort during use, stop using the device immediately.

7. Remove contact lenses, glasses, false eyelashes, hairpins, and any other accessories before use.
8. Do not use the device while driving, walking, or running.
9. Avoid contact with sharp objects.
10. High temperatures may cause burns to occur regardless of the control setting. Do not use the device on infants, persons who are with disabilities, sleeping or unconscious. Check frequently where the skin is in contact with the heated area to reduce the risk of blistering.

DANGER - To reduce the risk of electric shock:

1. Never use pins or other metallic fasteners with this device.
2. Do not use the device with wet hands or submerge the device in water or any liquid.

CAUTION - To reduce the risk of burns:

1. Use heated surfaces carefully. May cause serious burns. Do not use over insensitive skin areas or in the presence of poor circulation. The unattended use of heat by children or incapacitated persons may be dangerous.
2. The device has a heated surface. Persons insensitive to heat must be careful when using the device.

WARNING - To reduce the risk of burns, fire, electric shock, or injury to persons:

1. Do not operate under blanket or pillow. Excessive heating can occur and cause fire, electric shock, or injury to persons.
2. Close supervision is necessary when this device is used by, on, or near children, invalids, or disabled persons.
3. Use this device only for its intended use as described in this manual. Do not use attachments not recommended by the manufacturer.
4. Never operate this device if it is not working properly, has been damaged, or dropped into water.
5. Never drop or insert any object into any opening.
6. Do not operate where aerosol (spray) products are being used or where oxygen is being administered.

About the battery:

1. Do not use while bathing or in a shower.
2. Do not place or store device where it can fall or be pulled into a tub or sink.

3. Do not place in or drop into water or other liquid.
4. Never operate on a soft surface such as a bed or couch where the air openings may be blocked.
5. Do not expose the device or battery to excessive temperatures.
6. Be aware of the risk of terminals of the battery-operated device or battery being short-circuited by metal objects.
7. This device contains batteries that are non-replaceable. When the battery is at end of life, the device shall be properly disposed of.
8. Do not dismantle, open or shred battery.
9. Do not subject cells or batteries to mechanical shock.
10. In the event of a cell leaking, do not allow the liquid to come in contact with the skin or eyes. If contact has been made, wash the affected area with copious amounts of water and seek medical advice.
11. Secondary cells and batteries need to be charged before use. Always use the correct charger and refer to the manufacturer's instructions or product manual for proper charging instructions.
12. Do not leave a battery on prolonged charge when not in use.
13. After extended periods of storage, it may be necessary to charge and discharge the cells or batteries several times to obtain maximum performance.
14. CAUTION - The battery used in this device may present a risk of fire or chemical burn if mistreated. Do not recharge, disassemble, heat above 100°C (212°F).

SAVE THESE INSTRUCTIONS

ABOUT THE PRODUCT

1. What's in the Box

1 x Eyeris 2
1 x USB-C Charging Cable
1 x Storage Bag

2. Technical Specifications

Product Name: Eye Massager
Model: R-G006
Input: 5V  1A
Rated Power: 5W
Battery Capacity: 1500mAh 5.55Wh
Product Weight: 325g / 0.7lb
Product Size: 205 x 115 x 97mm /
8.1 x 4.5 x 3.8inch

USING THE PRODUCT


1. Charging

- Fully charge the device before the first use.
- Use the provided charging cable and a DC 5V 1A charger (not included) to charge the device when the power indicator flashes red slowly, accompanied by a voice prompt "Low power, please charge".
- It takes about 3 hours to fully charge the device. Once charging is complete, the indicator will turn solid red.
- If the device is being charged at a voltage higher than 5V, the indicator will keep flashing red quickly for 1min and then turns off, indicating an improper charger is used.

Note: Do not use the device while charging.

2. Wearing

- Remove any eye accessories (e.g., eyeglasses, contact lenses, false eyelashes, etc.) that may hinder you from using the device.
- Wear the device and ensure it fits your eyes properly. Adjust the elastic headband at the back of your head to fit your head.
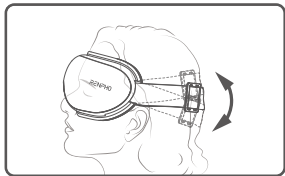


Note:

- Upon first use, it is recommended to start with soft pressure and then choose strong pressure if comfortable. If you feel any pain during use, switch back to soft pressure or loosen the headband.
- It is not recommended to use the device for more than 20 minutes at a time.

- Your vision may become blurry for a while, as expected after using the device. It is recommended to close your eyes for a few minutes until your vision goes back to normal.
- It is recommended to wipe any sweat, eye cream, or cosmetic residue left on the protein leather surface after use to prolong the lifespan of the device.

3. Operation

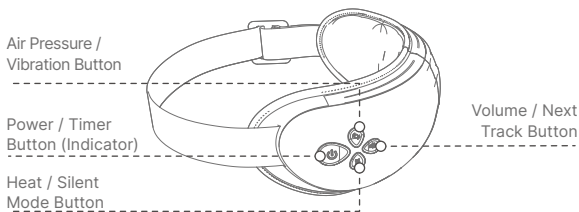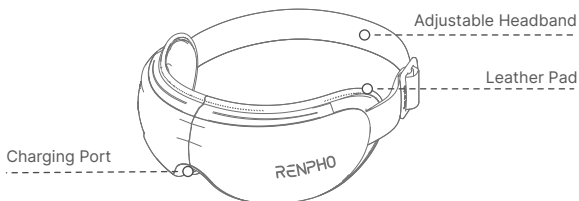Power / Timer Button

- Press and hold to turn on/off the device.

The indicator will be solid white once the device is turned on.

- Press to cycle the auto-off timer through 10/15 (default)/20min.

Heat / Silent Mode Button

- Press to cycle through low heat/high heat (default)/no temperature.
- Press and hold to switch between voice mode (default) and silent mode.

In silent mode, voice broadcast and music will be disabled.

Air Pressure / Vibration Button

- Press to cycle the air pressure through soft pressure (default)/strong pressure/no pressure.
- Press and hold to turn on/off the vibration.
The vibration is off by default.

Volume / Next Track Button

- Press to adjust the music volume level among level 0 (default)/1/2/3/4/5 (maximum volume).
- Press and hold to switch to the next music track.

Note: If no pressure and no temperature are selected, and the vibration function is disabled, the device will automatically shut off after 10 seconds of inactivity.

4. Bluetooth Connection

Turn on your smartphone's Bluetooth. Search for "R-G006" and connect to play your own music through the device. Once paired successfully, the indicator turns solid blue and the device will be automatically connected to Bluetooth next time.

Note: After Bluetooth connection, your music will play through the device at a default volume level of 5.

5. Auto-off Timer (10/15/20min)

Once the timer is set, the device will operate for the selected duration and then automatically shut off.

6. Memory Function

The device will recall the last settings of air pressure, vibration, heat and timer once turned on again.

CLEANING AND MAINTENANCE

1. Turn off the device before cleaning. Use a soft cotton cloth dampened with mild detergent to wipe it clean.
2. Do not expose the device to direct sunlight or high temperature. If the device has not been used for over 3 months, please recharge it before use.
3. Do not disassemble the device by yourself, as it may cause damage and void the manufacturer's warranty.
4. Battery life can vary by application and/or operating conditions. If the device is not in use for a long period, please keep it powered off and charge it once a month to prolong its lifespan.

TROUBLESHOOTING GUIDE

Problem	Possible Cause	Solution
There is a mechanical or air sound coming from the device	Normal sound of air pressure or air inflation	No action is necessary
Power-on failure	Out of power	Charge the device
Low massage intensity or weak air pressure	Low battery	Charge the device
The device stopped working suddenly	Out of power	Charge the device
	Built-in auto-off timer is activated	Restart the device for another massage session if needed. It is not recommended to use this device for more than 20 minutes at a time

Problem	Possible Cause	Solution
The device cannot be charged	Using other types of charging cables	Use the provided charging cable only
	Using improper charger with voltage as not indicated	Use the DC 5V 1A charger only
	Low battery protection is triggered due to prolonged non-charging	Contact after-sales service team
No heat	No temperature or low heat is selected	Switch to high heat
	Built-in heat protection is triggered due to excessive use	Turn the device off and let it cool down
	The device needs time to warm up	Wait for 1 minute after power on
No air pressure	No pressure is selected	Switch to soft pressure or strong pressure
No broadcast or music	Volume level is too low or volume level is at level 0	Adjust the volume level
	Silent mode is activated	Switch to voice mode

WARRANTY POLICY

Your RENPHO product purchase is covered by a one-year limited manufacturer warranty from the date of delivery.

For warranty terms and conditions, please visit:

<https://renpho.com/pages/warranty-terms-and-conditions>

Note: Product registration is not required for the warranty.

If you choose not to register your product, it will not diminish the product warranty.

CUSTOMER SERVICE

Please feel free to contact us if you have any questions or concerns. RENPHO Customer Service Team guarantees a quick response and hassle-free solutions to any issue you may have within business hours.

✉ **Email:** **support-eu@renpho.com (UK&EU)**
support@renpho.com (US&CA)
support-au@renpho.com (AU)

☎ **TEL:** **+44(0)7434 666088 (UK ONLY)**
Monday-Friday 9:00AM-5:00PM (GMT)
+1(844) 417 0149 (US&CA)
Monday-Friday 9:00AM-4:30PM
1800 MY RENPHO (1800 69 7367) (AU ONLY)
Monday-Friday 9:00AM-6:00PM (AEST)

*For defective products or the return of items, please contact us with your order number within the specified warranty period. DO NOT dispose of any product parts as they may be required for inspection/repair.

FCC Regulatory Compliance

Warning: changes or modifications not expressly approved by the party responsible for compliance could void the user's authority to operate the equipment.

Note: This equipment has been tested and found to comply with the limits for a Class B digital device, pursuant to part 15 of the FCC Rules. These limits are designed to provide reasonable protection against harmful interference in a residential installation. This equipment generates, uses and can radiate radio frequency energy and, if not installed and used in accordance with the instructions, may cause harmful interference to radio communications. However, there is no guarantee that interference will not occur in a particular installation. If this equipment does cause harmful interference to radio or television reception, which can be determined by turning the equipment off and on, the user is encouraged to try to correct the interference by one or more of the following measures:

- Reorient or relocate the receiving antenna.
- Increase the separation between the equipment and receiver.
- Connect the equipment into an outlet on a circuit different from that to which the receiver is connected.
- Consult the dealer or an experienced radio/TV technician for help.

ISED Regulatory Compliance

This device contains licence-exempt transmitter(s)/receiver(s) that comply with Innovation, Science and Economic Development Canada's licence-exempt RSS(s). Operation is subject to the following two conditions:

- 1: This device may not cause interference.
- 2: This device must accept any interference, including interference that may cause undesired operation of the device.

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1. Meaning of the crossed-out wheeled bin symbol

The crossed-out wheeled bin symbol means that you are legally obliged to dispose of these appliances separately from unsorted municipal waste, i.e. old appliances must be disposed of separately from household waste. Batteries or rechargeable batteries that are not permanently installed must be removed beforehand and disposed of separately.

2. Options for returning old appliances

Owners of WEEE can return or collect it within the framework of the possibilities set up and available by public waste management authorities to ensure proper disposal.

3. Personal data

All end-users of WEEE are reminded that you are responsible for deleting personal data from the WEEE to be disposed of.

1. The symbol of the crossed-out garbage can on batteries or accumulators indicates that they must not be disposed of in household waste at the end of their life.
2. You are legally obliged to return old batteries and accumulators after use. You can do this free of charge at a collection point in your area. Addresses of suitable collection points can be obtained from your city or local government.
3. Batteries may contain substances that are harmful to the environment and human health. The separate collection and recycling of old batteries and accumulators is intended to avoid negative effects on the environment and human health.
4. Please avoid the generation of waste from old batteries as far as possible, e.g. Please avoid littering public spaces by not carelessly leaving batteries or electrical and electronic equipment containing batteries.

Hereby, Shenzhen Ruiyi Business Technology Co., Ltd. declares that this equipment complies with Directive 2014/53/EU, Directive 2011/65/EU, Radio Equipment Regulations 2017, Restriction of the Use of Certain Hazardous Substances in Electrical and Electronic Equipment Regulations 2012, and other relevant provisions. The full text of the Declaration of Conformity can be obtained at the following internet address:
<https://renpho.eu/pages/compliance-data>

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