

RENPHO

USER MANUAL



Blood Pressure Monitor

Model: TMB-1872-B

Version:1.3



Guangdong Transtek Medical Electronics Co., Ltd.
Zone A, No.105, Dongli Road, Torch Development District,
528437 Zhongshan, Guangdong, China

Distributed by JOICOM CORPORATION

14129 The Merge Street, Building 3 Unit A, Eastvale, CA 92880



Made in China

Thank you very much for selecting RENPHO Blood Pressure Monitor TMB-1872-B.

Please read the user manual carefully and thoroughly to ensure the safe usage of this product, Keep the manual well for further reference in case you have problems.

Table of Contents

INTRODUCTION.....	3
• General Description	
• Indications for Use	
• Contraindications	
• Measurement Principle	
• Safety Information	
• LCD Display Signal	
• Monitor Components	
• List	
BEFORE YOU START.....	15
• The Choice of Power Supply	
• Installing and Replacing the Batteries	
• Setting Date, Time	
• Select the User	
MEASUREMENT.....	21
• Tie the Cuff	
• Start the Measurement	
DATA MANAGEMENT.....	24
• Recall the Records	
• Delete the Records	
APP OPERATION GUIDE	27
INFORMATION FOR USERS.....	35
• Tips for measurement	
• Maintenances	
ABOUT BLOOD PRESSURE.....	37
• What are systolic pressure and diastolic pressure?	
• What is the standard blood pressure classification?	
• Irregular heartbeat detector	
• Why does my blood pressure fluctuate throughout the day?	
• Why do I get a different blood pressure at home compared to the hospital?	
• Is the result the same if measuring on the right arm?	

TROUBLESHOOTING.....	42
SPECIFICATIONS.....	43
EMC GUIDANCE.....	45

FCC ID: OU9TMB1872BS

FCC Regulatory Compliance

This device complies with part 15 of the FCC Rules. Operation is subject to the following two conditions: (1) This device may not cause harmful interference, and (2) this device must accept any interference received, including interference that may cause undesired operation.

Warning: changes or modifications not expressly approved by the party responsible for compliance could void the user's authority to operate the equipment.

Note: This equipment has been tested and found to comply with the limits for a Class B digital device, pursuant to Part 15 of the FCC Rules. These limits are designed to provide reasonable protection against harmful interference in a residential installation. This equipment generates, uses and can radiate radio frequency energy and, if not installed and used in accordance with the instructions, may cause harmful interference to radio communications.

However, there is no guarantee that interference will not occur in a particular installation.

If this equipment does cause harmful interference to radio or television reception, which can be determined by turning the equipment off and on, the user is encouraged to try to correct the interference by one or more of the following measures:

- Reorient or relocate the receiving antenna.
- Increase the separation between the equipment and receiver.
- Connect the equipment into an outlet on a circuit different from that to which the receiver is connected.
- Consult the dealer or an experienced radio/TV technician for help.

RF Exposure Compliance

This equipment complies with FCC radiation exposure limits set forth for an uncontrolled environment. This transmitter must not be co-located or operating in conjunction with any other antenna or transmitter.

INTRODUCTION

♥ General Description

Thank you for selecting RENPHO arm type blood pressure monitor (TMB-1872-B). The monitor features blood pressure measurement, pulse rate measurement and the result storage. The design provides you with two years of reliable service.

Readings taken by the TMB-1872-B are equivalent to those obtained by a trained observer using the cuff and stethoscope auscultation method.

This manual contains important safety and care information, and provides step-by-step usage instructions for using the product. Read the manual thoroughly before using the product.

Features:

- 2.36*2.91inch Digital LCD display
- Maximum 120 records per each user
- Measuring during inflation technology

♥ Indications for Use

The Blood Pressure Monitor is a digital monitor intended for use in measuring blood pressure and heartbeat rate with arm circumference ranging from 22cm to 42cm (about 8¾"-16½").

It's intended for indoor use by adults only.

♥ Contraindications

1. The device should not be used by any person who may be suspected of, or is pregnant.
2. The device is not suitable for patients with implanted, electrical devices, such as cardiac pacemakers, defibrillators.

INTRODUCTION

♥ Measurement Principle

This product uses the Oscillometric Measuring method to detect blood pressure. Before every measurement, the unit establishes a "zero pressure" equivalent to the atmospheric pressure. Then it starts inflating the arm cuff, meanwhile, the unit detects pressure oscillations generated by beat-to-beat pulsatile, which is used to determine the systolic and diastolic pressure, and also pulse rate.

♥ Safety Information

The signs below might be in the user manual, labeling or other components. They are the requirement of standard and use.

	Symbol for "THE OPERATION GUIDE MUST BE READ"		Symbol for "TYPE BF APPLIED PARTS"
	Symbol for "MANUFACTURER"		Symbol for "ENVIRONMENT PROTECTION - Electrical waste products should not be disposed of with household waste. Please recycle where facilities exist. Check with your local authority or retailer for recycling advice"
	Symbol for "SERIAL NUMBER"		
	Symbol for "DIRECT CURRENT"		Symbol for "MANUFACTURE DATE"
	Symbol for "RECYCLE"		MR Unsafe To identify an item which poses unacceptable risks to the patient, medical staff or other persons with the MR environment.
	Caution: These notes must be observed to prevent any damage to the device.		



CAUTION

- * This device is intended for adult use in homes only.
- * The device is not suitable for neonatal patients, pregnant women, patients with implanted, electronic devices, patients with pre-eclampsia, premature ventricular beats, atrial fibrillation, peripheral, arterial disease and patients undergoing intravascular therapy or arterio-venous shunt or people who received a mastectomy. Please consult your doctor prior to using the unit if you suffer from illnesses.
- * The device is not suitable for measuring the blood pressure of children. Ask your doctor before using it on older children.
- * The device is not intended for patient transport outside a health-care facility.
- * The device is not intended for public use.
- * This device is intended for non-invasive measuring and monitoring of arterial blood pressure. It is not intended for use on extremities other than the arm or for functions other than obtaining a blood pressure measurement.
- * Do not confuse self-monitoring with self-diagnosis. This unit allows you to monitor your blood pressure. Do not begin or end medical treatment without asking a physician for treatment advice.
- * If you are taking medication, consult your physician to determine the most appropriate time to measure your blood pressure. Never change prescribed medication without consulting your physician.
- * Do not take any therapeutic measures on the basis of a self measurement. Never alter the dose of medicine prescribed by a doctor. Consult your doctor if you have any questions about your blood pressure.



CAUTION

- * When the device is used to measure patients who have common arrhythmias such as atrial or ventricular premature beats or atrial fibrillation, the best result may occur with deviation. Please consult your physician about the result.
- * Don't kink the connection tube during use, otherwise, the cuff pressure may continuously increase which can prevent blood flow and result in injury to the PATIENT.
- * When using this device, please pay attention to the following situation which may interrupt blood flow and influence blood circulation of the patient, thus causing injury to the patient: connection tubing kinking too frequent and consecutive multiple measurements; the application of the cuff and its pressurization on any arm where intravascular access or therapy, or an arterio-venous (A-V) shunt, is present; inflating the cuff on the side of a mastectomy.
- * Warning: Do not apply the cuff over a wound; otherwise it can cause further injury.
- * Do not inflate the cuff on the same limb which other monitoring ME equipment is applied around simultaneously, because this could cause temporary loss of function of those simultaneously-used monitoring ME equipment.
- * On the rare occasion of a fault causing the cuff to remain fully inflated during measurement, open the cuff immediately. Prolonged high pressure (cuff pressure > 300mmHg or constant pressure > 15mmHg for more than 3 minutes) applied to the arm may lead to danger.
- * Please check that operation of the device does not result in prolonged impairment of patient blood circulation.

CAUTION

- * When measurement, please avoid compression or restriction of the connection tubing.
- * The device cannot be used with HF surgical equipment at the same time.
- * The ACCOMPANYING DOCUMENT shall disclose that the SPHYGMOMANOMETER was clinically investigated according to the requirements of ISO 81060-2
- * To verify the calibration of the AUTOMATED SPHYGMOMANOMETER, please contact the manufacturer.
- * This device is contraindicated for any female who may be suspected of, or is pregnant. Besides providing inaccurate readings, the effects of this device on the fetus are unknown.
- * Too frequent and consecutive measurements could cause disturbances in blood circulation and injuries.
- * This unit is not suitable for continuous monitoring during medical emergencies or operations. Otherwise, the patient's arm and fingers will become anaesthetic, swollen and even purple due to a lack of blood.
- * When not in use, store the device in a dry room and protect it against extreme moisture, heat, lint, dust and direct sunlight. Never place any heavy objects on the storage case.
- * This device may be used only for the purpose described in this booklet. The manufacturer cannot be held liable for damage caused by incorrect application.
- * This device comprises sensitive components and must be treated with caution. Observe the storage and operating conditions described in this booklet.

CAUTION

- * The equipment is not AP/APG equipment and not suitable for use in the presence of a flammable anesthetic mixture with air or with oxygen or nitrous oxide.
- * Warning: No servicing/maintenance while the ME equipment is in use.
- * The patient is an intended operator.
- * The patient can measure data and change batteries under normal circumstances and maintain the device and its accessories according to the user manual.
- * To avoid measurement errors, please avoid the condition of strong electromagnetic field radiated interference signal or electrical fast transient/burst signal.
- * The blood pressure monitor and the cuff are suitable for use within the patient environment. If you are allergic to polyester, nylon or plastic, please don't use this device.
- * During use, the patient will be in contact with the cuff. The materials of the cuff have been tested and found to comply with requirements of ISO 10993-5 and ISO 10993-10. It will not cause any potential sensation or irritation reaction.
- * Adaptor is specified as a part of ME EQUIPMENT.
- * If you experience discomfort during measurement, such as pain in the arm or other complaints, press the START/STOP button to release the air immediately from the cuff. Loosen the cuff and remove it from your arm.
- * If the cuff pressure reaches 40 kPa (300 mmHg), the unit will automatically deflate. Should the cuff not deflate when the pressure reaches 40 kPa (300 mmHg), detach the cuff from the arm and press the START/STOP button to stop inflation.

CAUTION

- * Before use, make sure the device functions safely and is in proper working condition. Check the device, do not use the device if it is damaged in any way. The continuous use of a damaged unit may cause injury, improper results, or danger.
- * Do not wash the cuff in a washing machine or dishwasher!
- * The service life of the cuff may vary by the frequency of washing, skin condition, and storage state. The typical service life is 10000 times.
- * It is recommended that the performance should be checked after maintenance and repair, by retesting at least the requirements in limits of the error of the cuff pressure indication and air leakage (testing at least at 50mmHg and 200mmHg).
- * Please dispose of ACCESSORIES, detachable parts, and the ME EQUIPMENT according to the local guidelines.
- * Manufacturer will make available on request circuit diagrams, component part lists, descriptions, calibration instructions, etc., to assist to service personnel in parts repair.
- * The plug/adaptor plug pins insulate the device from the main supply. Do not position the device in a position where it is difficult to disconnect from the supply mains to safely terminate operation of ME equipment.
- * The operator shall not touch the output of batteries/adaptor and the patient simultaneously.
- * Cleaning: Dust environment may affect the performance of the unit. Please use a soft cloth to clean the whole unit before and after use. Don't use any abrasive or volatile cleaners.
- * The device doesn't need to be calibrated within two years of reliable service.

CAUTION


- * If you have any problems with this device, such as setting up, maintaining or using, please contact the SERVICE PERSONNEL of RENPHO. Don't open or repair the device by yourself in the event of malfunctions. The device must only be serviced, repaired and opened by individuals at authorized sales/service centers.
- * Please report to RENPHO if any unexpected operation or events occur.
- * Keep the unit out of reach of infants, young children or pets to avoid inhalation or swallowing of small parts. It is dangerous or even fatal.
- * Be careful against strangulation due to cables and hoses, particularly due to excessive length.
- * At least 30 min required for ME equipment to warm from the minimum storage temperature between uses until it is ready for the intended use. At least 30 min required for ME equipment to cool from the maximum storage temperature between uses until it is ready for the intended use.
- * This equipment needs to be installed and put into service in accordance with the information provided in the ACCOMPANYING DOCUMENTS;
- * Wireless communications equipment such as wireless home network devices, mobile phones, cordless telephones and their base stations, walkie-talkies can affect this equipment and should be kept at least a distance d away from the equipment. The distance is calculated by the MANUFACTURER from the 80MHz to 5.8 GHz column of Table 4 and Table 9 of IEC 60601-1-2 as appropriate.

INTRODUCTION

CAUTION

- * Please use ACCESSORIES and detachable parts specified/ authorised by MANUFACTURE. Otherwise, it may cause damage to the unit or danger to the user/patients.
- * There are no luer lock connectors in the construction of tubing. There is a possibility that they might be inadvertently connected to intravascular fluid systems, allowing air to be pumped into a blood vessel.
- * Please use the device under the environment which is indicated in the user manual. Otherwise, the performance and lifetime of the device will be impacted and reduced.

♥ LCD Display Signal

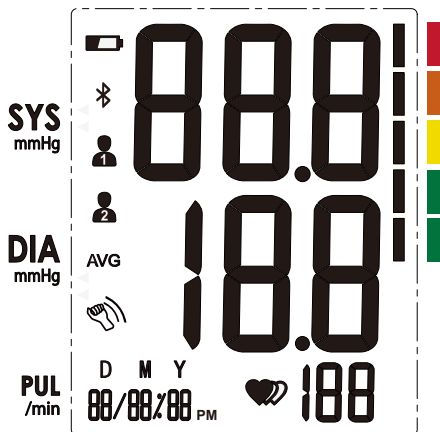


INTRODUCTION

SYMBOL	DESCRIPTION	EXPLANATION
	Bluetooth	When flashing, it means Bluetooth is pairing
		When it is always on, it means that Bluetooth is connected to Bluetooth
SYS	Systolic blood pressure	High blood pressure
DIA	Diastolic blood pressure	Low blood pressure
PUL /min	Pulse display	Pulse in beats per minute
AVG	Average value	The average value of blood pressure
	Memory	Indicate it is in memory mode and which group of memory it is.
	Motion indicator	Motion may result in an inaccurate measurement
mmHg	mmHg	Measurement Unit of the blood pressure (1mmHg=0.133kPa)
	Low battery	Batteries are low and need to be replaced
	Irregular heartbeat	Blood pressure monitor is detecting an irregular heartbeat during measurement.
	Blood pressure level indicator	Indicate the blood pressure level
D M Y 00/00/00 PM	Current Time	Day/Month/Year, Hour/Minute
	Heartbeat	Blood pressure monitor is detecting a heartbeat during measurement.
	User 1	Start measurement for User 1
	User 2	Start measurement for User 2


INTRODUCTION

♥ Monitor Components

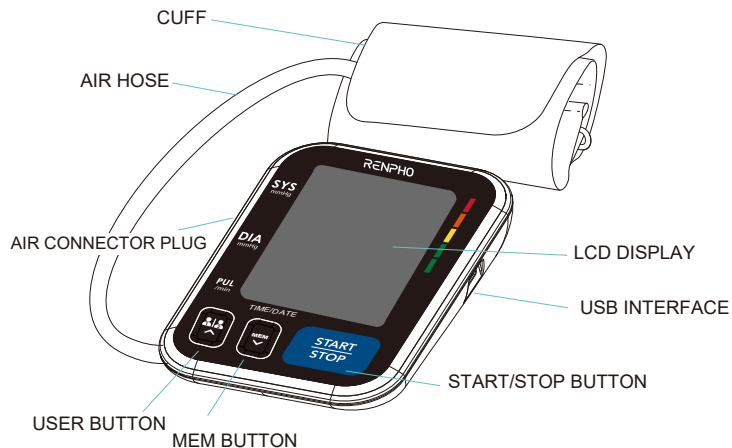

Component list of pressure measuring system

1. Cuff
2. Air pipe
3. PCBA
4. Pump
5. Valve

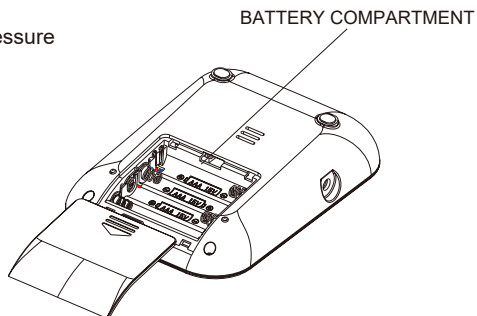


INTRODUCTION

♥ List


1. Blood Pressure Monitor (TMB-1872-B)




2. Cuff (Type BF applied part) (8.66~16.53in/22~42cm)

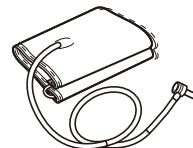

3. User Manual

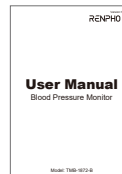

4. Quick Guide

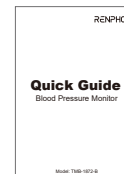


5. USB Cable



6. Carrying Case




7. 4*AAA Batteries

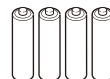

BEFORE YOU START

♥ The Choice of Power Supply

1. Battery-powered mode:
6VDC 4*AAA batteries
2. AC adaptor powered mode: 5V $\equiv$ 1A
Please use the AC adaptor (not included) and USB cable just like the following picture:

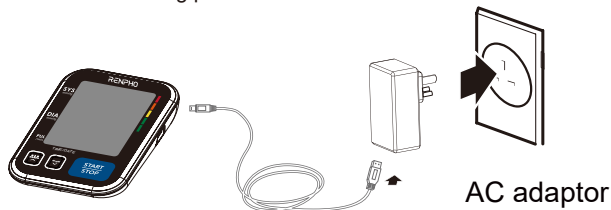


⚠ CAUTION

In order to get the best effect and protect your monitor, please use the right battery and special power adaptor which complies with local safety standard.

♥ Installing and Replacing the Batteries

- Slide off the battery cover.
- Install the batteries by matching the correct polarity, as shown.
- Place the battery cover back.

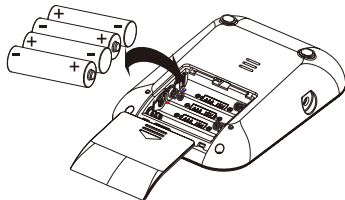

BEFORE YOU START

Replace the batteries whenever the below happens

- The **Lo+**  shows
- The display is dim
- The display does not light up

⚠ CAUTION


- Do not use new and used batteries together.
- Do not use different types of batteries together.
- Do not dispose of the batteries in fire. Batteries may explode or leak.
- Remove batteries if the device is not likely to be used for some time.
- Worn batteries are harmful to the environment. Do not dispose with daily garbage.
- Remove the old batteries from the device following your local recycling guidelines.

♥ Setting Date, Time


It is important to set the clock before using your blood pressure monitor, so that a time stamp can be assigned to each record that is stored in the memory. (The setting range of the year: 2020--2060)

BEFORE YOU START


1. When the monitor is off, press and hold the "START/STOP" button for 3 seconds to enter the mode for year setting.

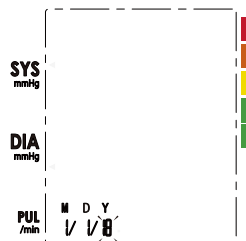

2. Press the "USER" button or "MEM" button to change the [YEAR]. Each press will increase or decrease the numeral by one in a cycling manner.

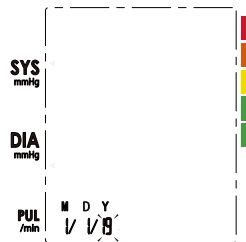


3. When you get the right year, press the "START/STOP" button to set down and turn to the next step.

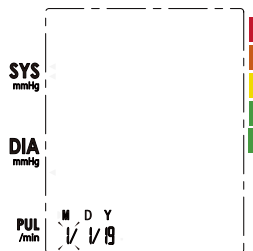


BEFORE YOU START

4. Repeat step 2 and 3 to set the [MONTH] and [DAY].

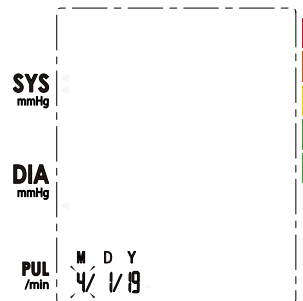

5. Repeat step 2 and 3 to set the [HOUR] and [MINUTE].

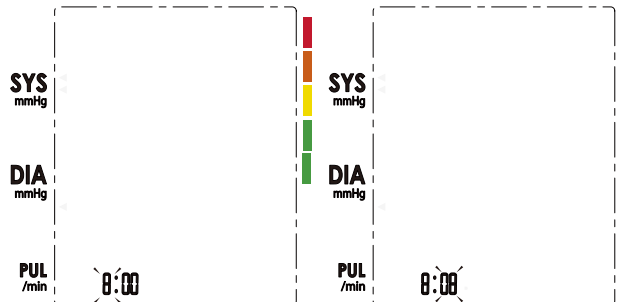


6. After the [MINUTE] is set, the LCD will display "donE" first, then display all the settings you have done and then it will shut off.




BEFORE YOU START

♥ Select the User

1. When the monitor is off, press the "USER" button shortly to enter the user setting mode.

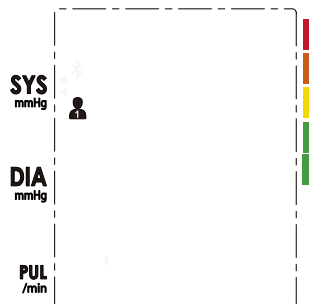

2. Then press the "USER" button again to select the user ID between user 1 and user 2.


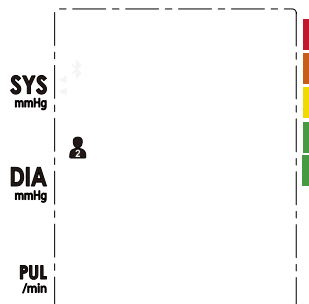


3. After selecting the user ID, press the "START/STOP" button to confirm and it will start to measure.

BEFORE YOU START

♥ Tie the Cuff


1. Remove all jewelry, such as watches and bracelets from your left arm. Note: If your doctor has diagnosed you with poor circulation in your left arm, use your right arm.
2. Roll or push up your sleeve to expose the skin. Make sure your sleeve is not too tight.
3. Hold your arm with your palm facing up and tie the cuff on your upper arm, then position the tube off-center toward the inner side of arm in line with the little finger. Or position the artery mark ϕ over the main artery (on the inside of your arm). Note: Locate the main artery by pressing with 2 fingers approximately 2cm/0.79in above the bend of your elbow on the inside of your left arm. Identify where the pulse can be felt the strongest. This is your main artery.

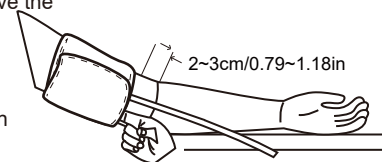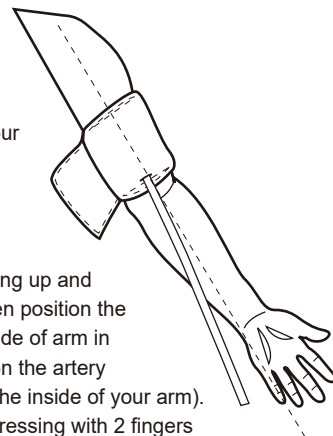

4. The cuff should be snug but not too tight. You should be able to insert one finger between the cuff and your arm.

MEASUREMENT

5. Sit comfortably with your tested arm resting on a flat surface. Place your elbow on a table so that the cuff is at the same level as your heart. Turn your palm upwards. Sit upright in a chair, and take 5-6 deep breaths.
6. Helpful tips for Patients, especially for Patients with Hypertension:
 - Rest for 5 minutes before the first measurement.
 - Wait at least 3 minutes between measurements. This allows your blood circulation to recover.
 - Take the measurement in a silent room.
 - The patient must relax as much as possible and do not move or talk during the measurement procedure.
 - The cuff should maintain at the same level as the right atrium of the heart.
 - Please sit comfortably. Do not cross your legs and keep your feet flat on the ground.
 - Keep your back against the backrest of the chair.
 - For a meaningful comparison, try to measure under similar conditions.
 - For example, take daily measurements at approximately the same time, on the same arm, or as directed by a physician.

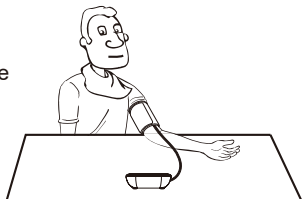


MEASUREMENT

♥ Start the Measurement


1. When the monitor is off, press the “START/STOP” button to turn on the monitor, and it will finish the whole measurement.



LCD display

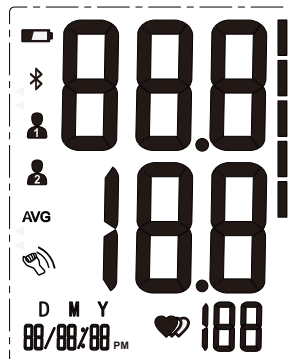


Adjust the zero.

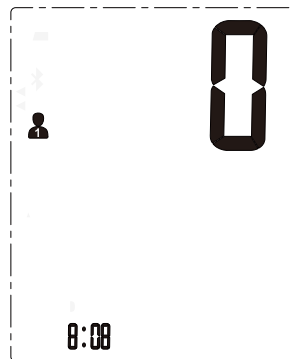


MEASUREMENT

Inflating and measuring.

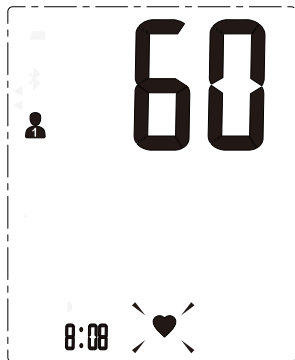

Display and save the results.

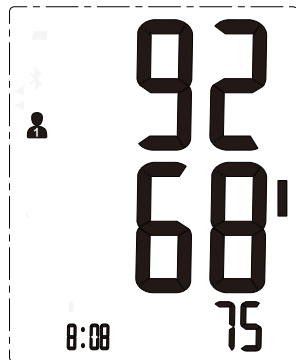


2. Press the "Start/Stop" button to power off, otherwise it will turn off within 1 minute.




DATA MANAGEMENT

♥ Recall the Records

1. When the monitor is off, please press the "MEM" button to show the recent record. If the records are less than 3 groups, it will display the latest record instead. Take user 1 for example.

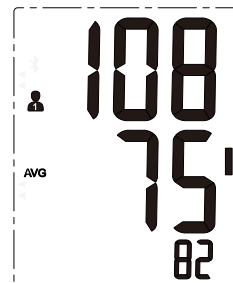

2. Press the "MEM" button to get the record you want.

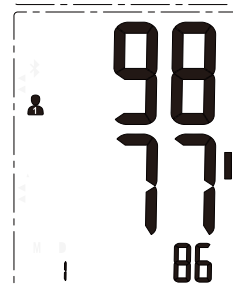

The date and time of the record will be shown alternately.



CAUTION

The most recent record (1) is shown first. Each new measurement is assigned to the first (1) record. All other records are pushed back one digit (e.g., 2 becomes 3, and so on), and the last record (120) is dropped from the list.

♥ Delete the Records

If you did not get the correct measurement, you can delete the results by following the steps below.

A: Delete one record

1. Hold pressing the “MEM” button for 3 seconds when the monitor is in the memory recall mode, the flash display “dELy 1 +USER ID” will show.



2. Press the “USER” button or “MEM” button and the flash displays “dEL n” or “dEL y”.



3. Press the “START/STOP” button shortly to confirm deleting this group result when it shows “dEL y” and display “USER ID + dEL donE”, then the device will show the latest record.



Tips: Press the “START/STOP” when it shows “dEL n”, it will drop out.



B: Delete all records

1. Press and hold the “MEM” button and “USER” button for 3 seconds when the monitor is in the memory recall mode, and the flash display “USER ID+dELy U5Er” will show.



2. Press the “USER” button or “MEM” button and the flash displays “dEL n” or “dEL y”.




3. Press the “START/STOP” button shortly to confirm deleting this group result when it shows “dEL y” and displays “USER ID + dEL donE”, then the device will turn off.



Tips: Press the “START/STOP” when it shows “dEL n”, it will drop out.



4. If there is no record, it will display like the right picture.

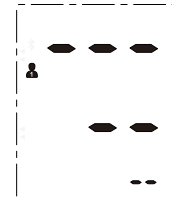

Download

1. Search and download "Renpho Health" from Apple Store / Google Play or scan the QR code below to download the app.




(The images shown are for illustration purposes only and may not be an exact representation due to app update)

2. Open the app. Register with your email address and fill in your personal information (this is required for calculating accurate health measurements). If you already have an account, tap Log in.



Add Device


1. After logging in, you will be directed to the homepage as shown below.
2. Activate Bluetooth on your phone, press the User button of the blood pressure monitor (the Bluetooth icon will flash on the display).
3. A pop-up window will appear to allow you to pair your phone with the device.

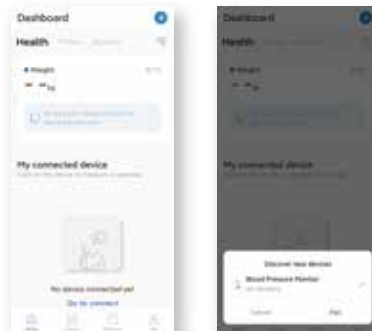


Alternatively, click “+” on the right corner to connect manually.

1. Activate Bluetooth on your smartphone.
2. Click “+” on the right corner.
3. Click “Blood Pressure Monitor (Bluetooth)” to search the device.
4. Press the User button on the blood pressure monitor (the Bluetooth icon will flash on the display)
5. Click ⊕ to connect.

APP OPERATION GUIDE

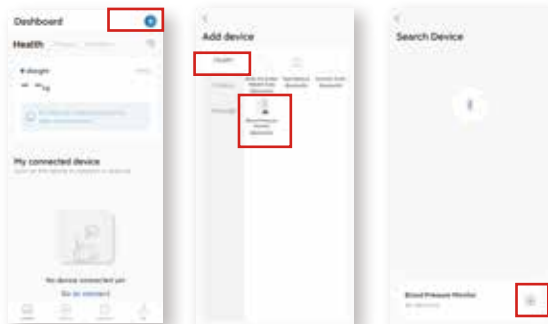

Select the “Blood Pressure Monitor” on the “Device” page to connect for next use. Swipe left and click to delete to disconnect the device.

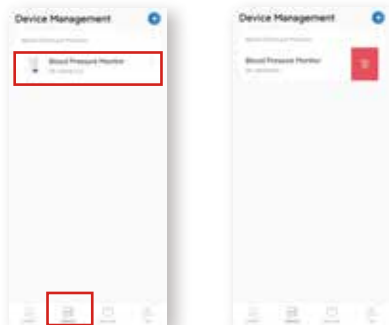


Note: The blood pressure monitor can be paired with one device only. If you want to connect it with another device, please ensure it is fully disconnected from any prior device.

APP OPERATION GUIDE

Add User 1

After pairing, you need to add a user profile if you connect the device for the first time. After filling in the user information as shown below, click “Next step” to save and you will be directed to the homepage (“Records” interface). The “Bluetooth” icon will light up once the device has been paired successfully.

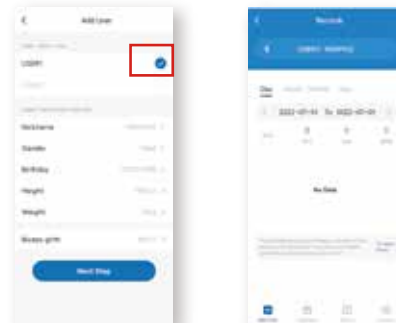

Synchronize the Existing Data

After entering the homepage, you need to synchronize the existing measurement data of the device on the app.



APP OPERATION GUIDE


1. Press the "MEM" button on the blood pressure monitor.
2. Restart the app and enter the "Records" page.
3. The data will be synchronized once the "Bluetooth" icon lights up.

Note: The same measurement data can be synchronized for one account only.



Check Measurement Record


Check your measurement data on a daily, weekly, monthly or yearly basis on the "Records" page.

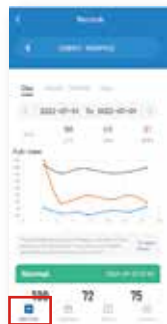

APP OPERATION GUIDE

Click any measurement data to add remarks that will be displayed along with it.

Swipe left and click to delete the data.

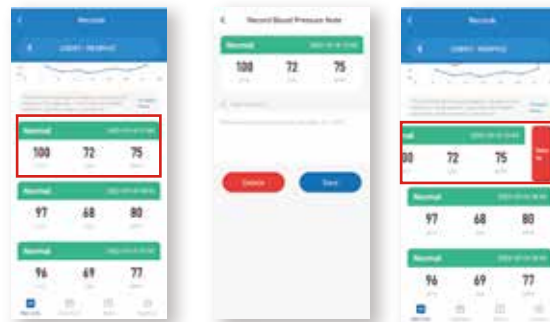


Check Calendar

1. Enter the "Calendar" page to get a holistic view of the dates of measurement.
2. Click the day with blue dot to check the measurement data of that day.
3. Click the data to add remarks or swipe left to delete as mentioned in the part of "Check Measurement Record".




APP OPERATION GUIDE

You can click "Choose Data" to delete multiple data at once.

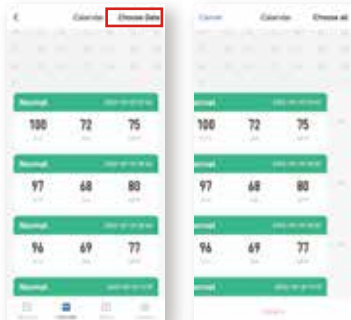

Share

Enter the "Share" page and select the time period to share your measurement results in CSV file.

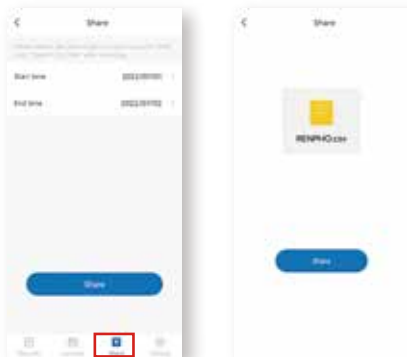

APP OPERATION GUIDE


Setting

Enter the "Setting" page to adjust the following functions.



Add User 2

1. Ensure the device is paired with the app ("Bluetooth" icon solid on the "Records" interface).
2. Enter the "Setting" page and select "Add User". Add the user profile following the in-app instructions.
3. After adding the user, you can switch between "User 1" and "User 2" on the "Records" interface. Ensure the device has been disconnected from the app ("Bluetooth" icon turned off) when you switch the user.

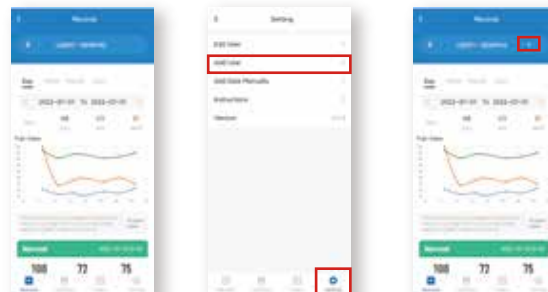

INFORMATION FOR USERS

♥ Tips for Measurement

Measurements may be inaccurate if taken under the following circumstances.



Within 1 hour
after dinner or drinking



Immediate measurement
after tea, coffee, smoking



Within 20 minutes
after taking a bath



When talking or moving your fingers



In a very cold environment



When you want to discharge urine

INFORMATION FOR USERS

♥ Maintenance

Please calibrate the blood pressure monitor in specific institute once every two years to ensure precise measurements. In order to get the best performance, please follow the instructions below for storage.




Put in a dry place and avoid the sunshine



Avoid intense shaking
and collisions



Use wet cloths to remove dirt

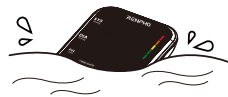

Avoid touching water,
clean it with a dry cloth in case.



Avoid dusty and unstable
temperature environment




Do not attempt to clean the reusable cuff
with water and never immerse the cuff in
water

ABOUT BLOOD PRESSURE

♥ What are systolic pressure and diastolic pressure?

When ventricles contract and pump blood out of the heart, the blood pressure reaches its maximum value in the cycle, which is called systolic pressure. When the ventricles relax, the blood pressure reaches its minimum value in the cycle, which is called diastolic pressure.

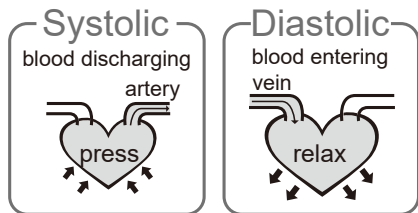

ABOUT BLOOD PRESSURE

♥ What is the standard blood pressure classification?

The chart on the right is the standard blood pressure classification published by American Heart Association (AHA).

This chart reflects blood pressure categories defined by American Heart Association.

Blood Pressure Category	Systolic mmHg (upper#)		Diastolic mmHg (lower#)
Normal	less than 120	and	less than 80
Elevated	120-129	and	less than 80
High Blood Pressure (Hypertension) Stage 1	130-139	or	80-89
High Blood Pressure (Hypertension) Stage 2	140 or higher	or	90 or higher
Hypertensive Crisis (Consult your doctor immediately)	Higher than 180	and/or	Higher than 120

⚠ CAUTION

Please consult a physician if your measuring result falls outside the range. Please note that only a physician can tell whether your blood pressure value has reached a dangerous point.

ABOUT BLOOD PRESSURE

♥ Irregular Heartbeat Detector

An irregular heartbeat is detected when a heartbeat rhythm varies while the device is measuring systolic pressure and diastolic pressure. During each measurement, blood pressure monitor will keep a record of all the pulse intervals and calculate the average value of them. If there are two or more pulse intervals, the difference between each interval and the average is more than the average value of $\pm 25\%$, or there are four or more pulse intervals, the difference between each interval and the average is more than the average value of $\pm 15\%$, then the irregular heartbeat symbol will appear on the display with the measurement result.


⚠ CAUTION

The appearance of the IHB icon indicates that a pulse irregularity consistent with an irregular heart-beat was detected during measurement. Usually this is NOT a cause for concern. However, if the symbol appears often, we recommend you seek medical advice. Please note that the device does not replace a cardiac examination, but serves to detect pulse irregularities at an early stage.

ABOUT BLOOD PRESSURE

♥ Why does my blood pressure fluctuate throughout the day?

1. Individual blood pressure varies multiple times every day. It is also affected by the way you tie your cuff and your measurement position, so please take the measurement under the same conditions.
2. If the person takes medicine, the pressure will vary more.
3. Wait at least 3 minutes for another measurement.

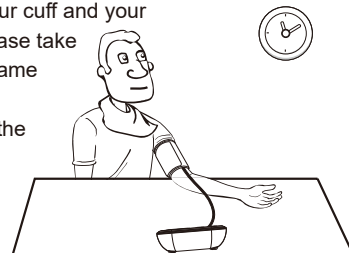

♥ Why do I get a different blood pressure at home compared to the hospital?

The blood pressure is different even throughout the day due to weather, emotion, exercise, etc. Also, there is the “white coat” effect, which means blood pressure usually increases in clinical settings.

What you need to pay attention to when you measure your blood pressure at home:


If the cuff is tied properly. If the cuff is too tight or too loose. If the cuff is tied on the upper arm. If you feel anxious. Taking 2-3 deep breaths before beginning will be better for measuring.

Advice: Relax yourself for 4-5 minutes until you calm down.

ABOUT BLOOD PRESSURE


♥ Is the result the same if measuring on the right arm?

It is ok for both arms, but there will be some different results for different people. We suggest you measure the same arm every time.

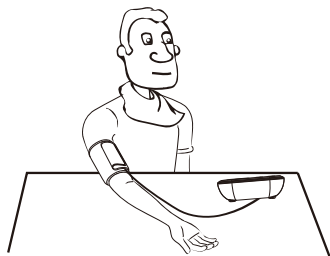

♥ Authorized Components

1. Please use the manufacturer-authorized AC adaptor
(Not included)

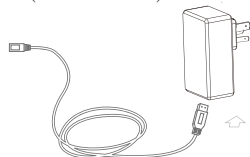

Adapter
Input: 100~240V, 50~60Hz, 0.2Amax
Output: 5V $\equiv$ 1000mA
BLJ06L050100U-U

♥ Contact Information

Please feel free to contact us if there is anything we can help with. We'll try our best to improve the product quality and service. All products come with warranty and lifetime support.

Tel: +1(844) 417-0149

PST 9:00am-4:30pm Mon-Fri

Email: support@renpho.com

TROUBLESHOOTING

PROBLEM	SYMPTOM	CHECK THIS	REMEDY
No power	Display will not light up	Batteries are exhausted	Replace with new batteries
		Batteries are inserted incorrectly	Insert the batteries correctly
		AC adaptor is inserted incorrectly	Insert the AC adaptor tightly
Low batteries	Display is dim or show 	Batteries are low	Replace with new batteries
Error Display will not message	E 01 shows	The cuff is too tight or too loose	Refasten the cuff and then measure again
	E 02 shows	The monitor detected motion, talking, or the pulse is too poor while measuring	Relax for a moment and then measure again
	E 03 shows	The measurement process does not detect the pulse signal	Loosen the clothing on the arm and then measure again
	E 04 shows	The treatment of the measurement failed	Relax for a moment and then measure again
	EExx shows	A calibration error occurred	Retake the measurement. If the problem persists, contact the retailer or our customer service department for further assistance. Refer to the warranty for contact information and return instructions
Warning message	"out" shows	Out of measurement range	Relax for a moment. Refasten the cuff and then measure again. If the problem persists, contact your physician

SPECIFICATIONS

Power supply	Battery powered mode: 6VDC 4*AAA batteries AC adaptor powered mode: 5V=1A (Not included) (Please only use the recommended AC adaptor model).
Display mode	Digital LCD display V.A.2.36in*2.91in (60mm*74mm)
Measurement mode	Oscillographic testing mode
Measurement range	Rated cuff pressure: 0mmHg~299mmHg(0kPa ~ 39.9kPa) Measurement pressure: SYS: 60mmHg~230mmHg (8.0kPa~30.7kPa) DIA: 40mmHg~130mmHg (5.3kPa~17.3kPa) Pulse value: (40-199)beat/minute
Accuracy	Pressure: 41°F-104°F (5°C-40°C) within±3mmHg (0.4kPa) Pulse value: ±5%
Normal working condition	A temperature range of : 41°F to +104°F(+5°C to +40°C) A relative humidity range of 15% to 90%, non-condensing, but not requiring a water vapour partial pressure greater than 50 hPa An atmospheric pressure range of: 700 hPa to 1060 hPa
Storage & transportation condition	Temperature: -4°F to +140°F (-20°C to +60°C) A relative humidity range of ≤ 93%, non-condensing, at a water vapour pressure up to 50hPa

SPECIFICATIONS

Measurement perimeter of the upper arm	About 8.66in~16.53in (22cm~42cm)
Net Weight	Approx.187g (Excluding the batteries)
External dimensions	Approx.3.66in*5.12in*1.28in (93mm*130mm*32.5mm)
Attachment	4*AAA batteries, cuff, user manual, USB cable, carrying case, quick guide
Mode of operation	Continuous operation
Device Classification	Battery Powered Mode: Internally Powered ME Equipment AC Adaptor Powered Mode: Class II ME Equipment
Degree of protection	Type BF applied part
Protection against ingress of water	IP21 It means the device could protected against solid foreign objects of 12.5mm and greater, and protect against vertically falling water drops.
Software Version	A01
Bluetooth Module NO.	LS8261
RF Frequency Range	2400 MHz to 2483.5 MHz
Output Power Range	≤8dBm
Supply Voltage	1.9-3.6 V
Transmitting Distance	10 meters

WARNING: No modification of this equipment is allowed.

♥ EMC Guidance

The ME EQUIPMENT or ME SYSTEM is suitable for home healthcare environments

Warning: Don't near active HF surgical equipment and the RF shielded room of an ME system for magnetic resonance imaging, where the intensity of EM disturbances is high.

Warning: Use of this equipment adjacent to or stacked with other equipment should be avoided because it could result in improper operation. If such use is necessary, this equipment and the other equipment should be observed to verify that they are operating normally.

Warning: Use of accessories, transducers and cables other than those specified or provided by the manufacturer of this equipment could result in increased electromagnetic emissions or decreased electro-magnetic immunity of this equipment and result in improper operation.

Warning: Portable RF communications equipment (including peripherals such as antenna cables and external antennas) should be used no closer than 30 cm (12 inches) to any part of the equipment TMB-1872-B, including cables specified by the manufacturer. Otherwise, degradation of the performance of this equipment could result.

Technical description :

- 1. All necessary instructions for maintaining BASIC SAFETY and ESSENTIAL PERFORMANCE with regard to electromagnetic disturbances for the excepted service life.
- 2. Guidance and manufacturer's declaration -electromagnetic emissions and Immunity

Table 1

Guidance and manufacturer's declaration - electromagnetic emissions	
Emissions test	Compliance
RF emissions CISPR 11	Group 1
RF emissions CISPR 11	Class [B]
Harmonic emissions IEC 61000-3-2	Class A
Voltage fluctuations/ flicker emissions IEC 61000-3-3	Comply

Table 2

Guidance and manufacturer's declaration – electromagnetic Immunity		
Immunity Test	IEC 60601-1-2 Test level	Compliance level
Electrostatic discharge (ESD) IEC 61000-4-2	±8 kV contact ±2 kV, ±4kV, ±8 kV, ±15 kV air	±8 kV contact ±2 kV, ±4kV, ±8 kV, ±15 kV air
Electrical fast transient/burst IEC 61000-4-4	±2 kV for power supply lines ±1 kV signal input/output 100 kHz repetition frequency	±2 kV for power supply lines ±1 kV signal input/output 100 kHz repetition frequency
Surge IEC61000-4-5	±0.5 kV, ±1 kV differential mode ±0.5 kV, ±1 kV, ±2 kV common mode	±0.5 kV, ±1 kV differential mode ±0.5 kV, ±1 kV, ±2 kV common mode

Immunity Test	IEC 60601-1-2 Test level	Compliance level
Voltage dips, short interruptions and voltage variations on power supply input lines IEC 61000-4-11	0 % UT; 0,5 cycle. At 0°, 45°, 90°, 135°, 180°, 225°, 270° and 315°. 0 % UT; 1 cycle and 70 % UT; 25/30 cycles; Single phase: at 0°. 0 % UT; 250/300 cycle	0 % UT; 0,5 cycle. At 0°, 45°, 90°, 135°, 180°, 225°, 270° and 315°. 0 % UT; 1 cycle and 70 % UT; 25/30 cycles; Single phase: at 0°. 0 % UT; 250/300 cycle
Power frequency magnetic field IEC 61000-4-8	30 A/m 50Hz/60Hz	30 A/m 50Hz/60Hz
Conducted RF IEC61000-4-6	3 V 0,15 MHz – 80 MHz 6 V in ISM and amateur radio bands between 0,15 MHz and 80 MHz 80 % AM at 1 kHz	3 V 0,15 MHz – 80 MHz 6 V in ISM and amateur radio bands between 0,15 MHz and 80 MHz 80 % AM at 1 kHz
Radiated RF IEC61000-4-3	10 V/m 80 MHz – 2,7 GHz 80 % AM at 1 kHz	10 V/m 80 MHz – 2,7 GHz 80 % AM at 1 kHz
NOTE U T is the a.c. mains voltage prior to application of the test level.		

Table 3

Guidance and manufacturer's declaration - electromagnetic Immunity							
	Test Frequency (MHz)	Band (MHz)	Service	Modulation	Modulation (W)	Distance (m)	IMMUNITY TEST LEVEL (V/m)
Radiated RF IEC61000-4-3 (Test specifications for ENCLOSURE PORT IMMUNITY to RF wireless communications equipment)	385	380-390	TETRA 400	Pulse modulation b) 18Hz	1.8	0.3	27
	450	430-470	GMRS 460, FRS 460	FM c) ± 5kHz deviation 1kHz sine	2	0.3	28
	710	704-787	LTE Band 13, 17	Pulse modulation b) 217Hz	0.2	0.3	9
	745						
	780						
	810	800-960	GSM 800/900, TETRA 800, iDEN 820, CDMA 850, LTE Band 5	Pulse modulation b) 18Hz	2	0.3	28
	870						
	930						

Radiated RF IEC61000-4-3 (Test specifications for ENCLOSURE PORT IMMUNITY to RF wireless communications equipment)	Test Frequency (MHz)	Band (MHz)	Service	Modulation	Modulation (W)	Distance (m)	IMMUNITY TEST LEVEL (V/m)
	1720	1700-1990	GSM 1800; CDMA 1900; GSM 1900; DECT; LTE Band 1, 3, 4, 25; UMTS	Pulse modulation b) 217Hz	2	0.3	28
	1845						
	1970						
	2450	2400-2570	Bluetooth, WLAN, 802.11 b/g/n, RFID 2450, LTE Band 7	Pulse modulation 217 Hz	2	0.3	28
	5240	5100-5800	WLAN 802.11 a/n	Pulse modulation 217 Hz	0.2	0.3	9
	5500						
	5785						