



Mask/Device Compatibility List

This document specifies the masks compatible with ResMed's positive airway pressure devices and the mask setting, if one is required. For full details regarding the correct use of these devices, please refer to the specific User Guides.

Note: Not all products are available in all regions.

Full Face Masks 1

Device	AcuCare™ F1-4	AirFit™ F40 ⁴	AirFit™ F30i ¹ / AirFit™ F30i Non Magnetic ¹	AirFit™ X30i ^{1,4}	AirTouch™ F30i ^{1,4}	AirFit™ F30
AirCurve™ 11 ST / ST-A	X	Pillows	N/V	N/V	N/V	Full Face
AirCurve™ 11 VPAP Tx	X	Pillows	Full Face	Full Face	Full Face	Full Face
AirCurve™ 11 VAuto / AirCurve™ 11 S	X	Pillows	Full Face	Full Face	Full Face	Full Face
AirCurve™ 11 ASV	X	Pillows	Full Face	Full Face	Full Face	Full Face
AirSense™ 11	N/V	Pillows	Full Face	Full Face	Full Face	Full Face
AirMini™	X	X	X	X	X	✓
AirStart™ 10 / AirSense™ 10	N/V	Pillows	Full Face	Full Face	Full Face	Full Face
AirCurve™ 10 VAuto / AirCurve™ 10 S / AirCurve™ 10 ST	N/V	Pillows	Full Face	Full Face	Full Face	Full Face
AirCurve™ 10 ASV	N/V	Pillows	Full Face	Full Face	Full Face	Full Face
AirCurve™ 10 ST-A	N/V	Pillows ³	Full Face ³	Full Face ³	Full Face ³	Full Face ³
S9 AutoSet™ / S9 AutoSet™ for Her / S9 Elite™	N/V	Pillows	Full Face	Full Face	Full Face	N/V
S9 Escape™ / S9 Escape™ Auto	N/V	Pillows	Full Face	Full Face	Full Face	N/V
C-Series Tango™	N/V	N/V	N/V	N/V	N/V	N/V
S7 Elite™	N/V	N/V	N/V	N/V	N/V	N/V
S8 AutoSet Vantage™	N/V	N/V	N/V	N/V	N/V	N/V
S8 Compact™	N/V	N/V	N/V	N/V	N/V	N/V
S8 Elite™ / S8 Elite™ II	N/V	N/V	N/V	N/V	N/V	N/V
S8 Escape™	N/V	N/V	N/V	N/V	N/V	N/V
S8 Escape™ II	N/V	N/V	N/V	N/V	N/V	N/V
S8 AutoSet™ II	N/V	N/V	N/V	N/V	N/V	N/V
VPAP™ III	N/V	N/V	N/V	N/V	N/V	N/V
VPAP™ III ST-A	N/V	N/V	N/V	N/V	N/V	N/V
VPAP™ Adapt SV	N/V	N/V	N/V	N/V	N/V	N/V
VPAP™ Adapt (S9)	N/V	Pillows	Full Face	Full Face	Full Face	N/V

Device	AcuCare™ F1-4	AirFit™ F40 ⁴	AirFit™ F30i ¹ / AirFit™ F30i Non Magnetic ¹	AirFit™ X30i ^{1,4}	AirTouch™ F30i ^{1,4}	AirFit™ F30
VPAP™ Adapt (S9 ASV)	N/V	Pillows	Full Face	Full Face	Full Face	N/V
VPAP™ Auto 25	N/V	N/V	N/V	N/V	N/V	N/V
VPAP™ Auto (S9)	N/V	Pillows	Full Face	Full Face	Full Face	N/V
VPAP™ Malibu	N/V	N/V	N/V	N/V	N/V	N/V
VPAP™ S / VPAP™ ST	N/V	N/V	N/V	N/V	N/V	N/V
VPAP™ S (S9) / VPAP™ ST (S9)	N/V	Pillows	Full Face	Full Face	Full Face	N/V
VPAP™ ST-A (S9 iVAPS)	N/V	N/V	N/V	N/V	N/V	N/V
VPAP™ COPD (S9)	N/V	N/V	N/V	N/V	N/V	N/V
S9 VPAP™ Tx	N/V	Pillows ²	Full Face ²	Full Face ²	Full Face ²	N/V
VPAP™ Tx	N/V	N/V	N/V	N/V	N/V	N/V
Lumis™ Tx	N/V	N/V	N/V	N/V	N/V	N/V
Stellar™	N/V	N/V	N/V	N/V	N/V	N/V
VS Ultra™ / VS Integra™	✓	N/V	N/V	N/V	N/V	N/V
VS III™	N/V	N/V	N/V	N/V	N/V	N/V
Astral™	FULL FACE	N/V	N/V	N/V	N/V	N/V

¹ SmartStop may not operate as described when using this mask with some CPAP or bi-level devices.

² Excluding PAC and iVAPS modes. Only compatible for pressure range 4 - 25 cm H2O.

³ This combination of mask and device has been determined safe for use based on the evaluation of accuracy of monitoring of pressure, volume, minute ventilation and respiratory rate, and its alarms if applicable. Small variations in performance might be observed, particularly at higher pressure or flow. As per expected clinical practice, the effectiveness of ventilation and alarms should be verified by the clinician.

⁴ SmartStart™ may not operate as described when using this mask with some CPAP or bi-level devices.

N/V = The mask has not been validated for this device.

✓ = Compatible X = Not Compatible

Full Face Masks 2

Device	AirTouch™ F20 / AirTouch™ F20 for Her	AirFit™ F20 / AirFit™ F20 for Her / AirFit™ F20 Non Magnetic	AirFit™ F10 / AirFit™ F10 for Her	Mirage Quattro™	Quattro™ Air / Quattro™ Air for Her Quattro™ FX / Quattro™ FX for Her	Mirage Liberty™	Ultra Mirage™	Hospital Full Face
AirCurve™ 11 ST / ST-A	Full Face	Full Face	Full Face	Full Face	Full Face	X	X	Full Face
AirCurve™ 11 VPAP Tx	Full Face	Full Face	Full Face	Full Face	Full Face	Full Face	Full Face	Full Face
AirCurve™ 11 VAuto / AirCurve™ 11 S	Full Face	Full Face	Full Face	Full Face	Full Face	Full Face	Full Face	Full Face
AirCurve™ 11 ASV	Full Face	Full Face	Full Face	Full Face	Full Face	Full Face	Full Face	Full Face
AirSense™ 11	Full Face	Full Face	Full Face	Full Face	Full Face	Full Face	Full Face	Full Face
AirMini™	✓	✓	X	X	X	X	X	X
AirStart™ 10 / AirSense™ 10	Full Face	Full Face	Full Face	Full Face	Full Face	Full Face	Full Face	Full Face
AirCurve™ 10 VAuto / AirCurve™ 10 S / AirCurve™ 10 ST	Full Face	Full Face	Full Face	Full Face	Full Face	Full Face	Full Face	Full Face
AirCurve™ 10 ASV	Full Face	Full Face	Full Face	Full Face	Full Face	Full Face	Full Face	Full Face
AirCurve™ 10 ST-A	Full Face	Full Face	Full Face	Full Face	Full Face	Full Face	Full Face	Full Face
S9 AutoSet™ / S9 AutoSet™ for Her / S9 Elite™	Full Face	Full Face	Full Face	Full Face	Full Face	Full Face	Full Face	Full Face
S9 Escape™ / S9 Escape™ Auto	Full Face	Full Face	Full Face	Full Face	Full Face	Full Face	Full Face	Full Face
C-Series Tango™	N/V	N/V	No setting required. Consult your clinician before changing to a different mask type.					
S7 Elite™	N/V	N/V	MIR FULL	MIR FULL	MIR FULL	MIR FULL	MIR FULL	MIR FULL

Device	AirTouch™ F20 / AirTouch™ F20 for Her	AirFit™ F20 / AirFit™ F20 for Her / AirFit™ F20 Non Magnetic	AirFit™ F10 / AirFit™ F10 for Her	Mirage Quattro™	Quattro™ Air / Quattro™ Air for Her Quattro™ FX / Quattro™ FX for Her	Mirage Liberty™	Ultra Mirage™	Hospital Full Face
S8 AutoSet Vantage™	MIR FULL	MIR FULL	MIR FULL	MIR FULL	MIR FULL	MIR FULL	MIR FULL	MIR FULL
S8 Compact™	N/V	N/V		No setting required. Consult your clinician before changing to a different mask type.				
S8 Elite™ / S8 Elite™ II	MIR FULL	MIR FULL	MIR FULL	MIR FULL	MIR FULL	MIR FULL	MIR FULL	MIR FULL
S8 Escape™	N/V	N/V		Requires pressure calibration at 12 cm H2O. Consult your clinician and check the user guide.				
S8 Escape™ II	Requires pressure calibration at 12 cm H2O. Consult your clinician and check the user guide.							
S8 AutoSet™ II	MIR FULL	MIR FULL	MIR FULL	MIR FULL	MIR FULL	MIR FULL	MIR FULL	MIR FULL
VPAP™ III	N/V	N/V	MIR FULL	MIR FULL	MIR FULL	MIR FULL	MIR FULL	MIR FULL
VPAP™ III ST-A	N/V	N/V	MIR FULL	MIR FULL	MIR FULL	MIR FULL	MIR FULL	MIR FULL
VPAP™ Adapt SV	N/V	N/V	FULL FACE	FULL FACE	FULL FACE	FULL FACE¹	FULL FACE	FULL FACE
VPAP™ Adapt (S9)	Full Face	Full Face	Full Face	Full Face	Full Face	Full Face	Full Face	Full Face
VPAP™ Adapt (S9 ASV)	Full Face	Full Face	Full Face	Full Face	Full Face	Full Face	Full Face	Full Face
VPAP™ Auto 25	MIR FULL	MIR FULL	MIR FULL	MIR FULL	MIR FULL	MIR FULL	MIR FULL	MIR FULL
VPAP™ Auto (S9)	Full Face	Full Face	Full Face	Full Face	Full Face	Full Face	Full Face	Full Face
VPAP™ Malibu	N/V	N/V	FULL FACE	FULL FACE	FULL FACE	FULL FACE	FULL FACE	FULL FACE
VPAP™ S / VPAP™ ST	MIR FULL	MIR FULL	MIR FULL	MIR FULL	MIR FULL	MIR FULL	MIR FULL	MIR FULL
VPAP™ S (S9) / VPAP™ ST (S9)	Full Face	Full Face	Full Face	Full Face	Full Face	Full Face	Full Face	Full Face
VPAP™ ST-A (S9 iVAPS)	Full Face	Full Face	Full Face	Full Face	Full Face	Full Face	Full Face	Full Face
VPAP™ COPD (S9)	Full Face	Full Face	Full Face	Full Face	Full Face	Full Face	Full Face	Full Face
S9 VPAP™ Tx	Full Face	Full Face	Full Face	Full Face	Full Face	Full Face	Full Face	Full Face
VPAP™ Tx	N/V	N/V	FULL FACE	FULL FACE	FULL FACE	FULL FACE	FULL FACE	FULL FACE
Lumis™ Tx	Full Face	Full Face	Full Face	Full Face	Full Face	Full Face	Full Face	Full Face
Stellar™	FULL FACE	FULL FACE	FULL FACE	FULL FACE	FULL FACE	FULL FACE	FULL FACE	FULL FACE
VS Ultra™ / VS Integra™	N/V	N/V	✓	✓	N/V	✓	✓	✓
VS III™	N/V	N/V	✓	✓	N/V	✓	✓	✓
Astral™	FULL FACE	FULL FACE	N/V	FULL FACE	FULL FACE	FULL FACE	FULL FACE	FULL FACE

N/V = The mask has not been validated for this device.

✓ = Compatible X = Not Compatible

Nasal Masks 1

Device	AirTouch™ N30i¹	AirFit™ N30 for AirMini	AirFit™ N30	AirFit™ N30i¹	AirFit™ N20 / AirFit™ N20 for Her	AirTouch™ N20 / AirTouch™ N20 for Her	AirFit™ N10 / AirFit™ N10 for Her	Swift™ FX Nano / Swift™ FX Nano for Her
AirCurve™ 11 ST / ST-A	N/V	X	Pillows	N/V	Pillows	Pillows	X	X
AirCurve™ 11 VPAP Tx	Pillows	X	Pillows	Pillows	Pillows	Pillows	Pillows	Pillows
AirCurve™ 11 VAuto / AirCurve™ 11 S	Pillows	X	Pillows	Pillows	Pillows	Pillows	Pillows	Pillows
AirCurve™ 11 ASV	Pillows	X	Pillows	Pillows	Pillows	Pillows	Pillows	Pillows
AirSense™ 11	Pillows	X	Pillows	Pillows	Pillows	Pillows	Pillows	Pillows
AirMini™	X	✓	X	X	✓	✓	X	X
AirStart™ 10 / AirSense™ 10	Pillows	X	Pillows	Pillows	Pillows	Pillows	Pillows	Pillows
AirCurve™ 10 VAuto / AirCurve™ 10 S / AirCurve™ 10 ST	Pillows	X	Pillows	Pillows	Pillows	Pillows	Pillows	Pillows
AirCurve™ 10 ASV	Pillows	X	Pillows	Pillows	Pillows	Pillows	Pillows	Pillows

Device	AirTouch™ N30i¹	AirFit™ N30 for AirMini	AirFit™ N30	AirFit™ N30i¹	AirFit™ N20 / AirFit™ N20 for Her	AirTouch™ N20 / AirTouch™ N20 for Her	AirFit™ N10 / AirFit™ N10 for Her	Swift™ FX Nano / Swift™ FX Nano for Her
AirCurve™ 10 ST-A	Pillows³	X	Pillows³	Pillows³	Pillows	Pillows	Pillows	Pillows
S9 AutoSet™ / S9 AutoSet™ for Her / S9 Elite™	Pillows	X	Pillows	Pillows	Pillows	Pillows	Pillows	Pillows
S9 Escape™ / S9 Escape™ Auto	Pillows	X	Pillows	Pillows	Pillows	Pillows	Pillows	Pillows
C-Series Tango™	N/V	X	N/V	N/V	N/V	N/V	N/V	N/V
S7 Elite™	N/V	X	N/V	N/V	N/V	N/V	N/V	N/V
S8 AutoSet Vantage™	N/V	X	N/V	N/V	SWIFT	SWIFT	SWIFT	SWIFT
S8 Compact™	N/V	X	N/V	N/V	N/V	N/V	No setting required. Consult your clinician before changing to a different mask type.	
S8 Elite™ / S8 Elite™ II	N/V	X	N/V	N/V	SWIFT	SWIFT	SWIFT	SWIFT
S8 Escape™	N/V	X	N/V	N/V	N/V	N/V	Requires pressure calibration at 12 cm H2O. Consult your clinician and check the user guide.	
S8 Escape™ II	N/V	X	N/V	N/V	Requires pressure calibration at 12 cm H2O. Consult your clinician and check the user guide.			
S8 AutoSet™ II	N/V	X	N/V	N/V	SWIFT	SWIFT	SWIFT	SWIFT
VPAP™ III	N/V	X	N/V	N/V	N/V	N/V	MIRAGE	MIRAGE
VPAP™ III ST-A	N/V	X	N/V	N/V	N/V	N/V	MIRAGE	MIRAGE
VPAP™ Adapt SV	N/V	X	N/V	N/V	N/V	N/V	N/V	N/V
VPAP™ Adapt (S9)	Pillows	X	Pillows	Pillows	Pillows	Pillows	Pillows	Pillows
VPAP™ Adapt (S9 ASV)	Pillows	X	Pillows	Pillows	Pillows	Pillows	Pillows	Pillows
VPAP™ Auto 25	N/V	X	N/V	N/V	SWIFT	SWIFT	SWIFT	SWIFT
VPAP™ Auto (S9)	Pillows	X	Pillows	Pillows	Pillows	Pillows	Pillows	Pillows
VPAP™ Malibu	N/V	X	N/V	N/V	N/V	N/V	SWIFT	SWIFT
VPAP™ S / VPAP™ ST	N/V	X	N/V	N/V	SWIFT	SWIFT	SWIFT	SWIFT
VPAP™ S (S9) / VPAP™ ST (S9)	Pillows	X	Pillows	Pillows	Pillows	Pillows	Pillows	Pillows
VPAP™ ST-A (S9 iVAPS)	N/V	X	N/V	N/V	Pillows	Pillows	Pillows	Pillows
VPAP™ COPD (S9)	N/V	X	N/V	N/V	Pillows	Pillows	Pillows	Pillows
S9 VPAP™ Tx	Pillows²	X	Pillows²	Pillows²	Pillows	N/V	Pillows	Pillows
VPAP™ Tx	N/V	X	N/V	N/V	N/V	N/V	PILLOWS	PILLOWS
Lumis™ Tx	N/V	X	N/V	N/V	Pillows	Pillows	Pillows	Pillows
Stellar™	N/V	X	N/V	N/V	PILLOWS	PILLOWS	PILLOWS¹	PILLOWS¹
VS Ultra™ / VS Integra™	N/V	X	N/V	N/V	N/V	N/V	N/V	N/V
VS III™	N/V	X	N/V	N/V	N/V	N/V	N/V	N/V
Astral™	N/V	X	N/V	N/V	Pillows	Pillows	N/V	N/V

¹ SmartStop may not operate as described when using this mask with some CPAP or bi-level devices.

² Excluding PAC and iVAPS modes. Only compatible for pressure range 4 - 25 cm H2O.

³ This combination of mask and device has been determined safe for use based on the evaluation of accuracy of monitoring of pressure, volume, minute ventilation and respiratory rate, and its alarms if applicable. Small variations in performance might be observed, particularly at higher pressure or flow. As per expected clinical practice, the effectiveness of ventilation and alarms should be verified by the clinician.

N/V = The mask has not been validated for this device.

✓ = Compatible X = Not Compatible

Nasal Masks 2

Device	Mirage™ FX / Mirage™ FX for Her	Hospital Nasal	Mirage Micro™	Ultra Mirage™ II	Mirage Activa™	Mirage Activa™ LT	Mirage™ SoftGel	Mirage Vista™	Meridian	Mirage Kidsta™	Mirage Micro™ for Kids	Pixi™
AirCurve™ 11 ST / ST-A	Nasal	Nasal	X	X	X	Nasal	X	X	X	X	X	Pediatric
AirCurve™ 11 VPAP Tx	Nasal	Nasal	X	X	X	Nasal	X	X	X	X	X	N/V
AirCurve™ 11 VAuto / AirCurve™ 11 S	Nasal	Nasal	X	X	X	Nasal	X	X	X	X	X	N/V
AirCurve™ 11 ASV	Nasal	Nasal	X	X	X	Nasal	X	X	X	X	X	N/V
AirSense™ 11	Nasal	Nasal	N/V	N/V	N/V	Nasal	N/V	N/V	N/V	N/V	N/V	N/V
AirMini™	X	X	X	X	X	X	X	X	X	X	X	X
AirStart™ 10 / AirSense™ 10	Nasal	Nasal	Nasal	Nasal	Nasal	Nasal	Nasal	Nasal	Nasal	Nasal	N/V	N/V
AirCurve™ 10 VAuto / AirCurve™ 10 S / AirCurve™ 10 ST	Nasal	Nasal	Nasal	Nasal	Nasal	Nasal	Nasal	Nasal	Nasal	Nasal	N/V	N/V
AirCurve™ 10 ASV	Nasal	Nasal	Nasal	Nasal	Nasal	Nasal	Nasal	Nasal	Nasal	Nasal	N/V	N/V
AirCurve™ 10 ST-A	Nasal	Nasal	Nasal	Nasal	Nasal	Nasal	Nasal	Nasal	Nasal	Nasal	N/V	N/V
S9 AutoSet™ / S9 AutoSet™ for Her / S9 Elite™	Nasal	Nasal	Nasal	Nasal Ultra	Nasal	Nasal	Nasal	Nasal	Nasal	N/V	N/V	N/V
S9 Escape™ / S9 Escape™ Auto	Nasal	Nasal	Nasal	Nasal Ultra	Nasal	Nasal	Nasal	Nasal	Nasal	N/V	N/V	N/V
C-Series Tango™	No setting required. Consult your clinician before changing to a different mask type.								N/V	N/V	N/V	N/V
S7 Elite™	STANDARD	STANDARD	STANDARD	ULTRA	STANDARD	STANDARD	STANDARD	STANDARD	N/V	N/V	N/V	N/V
S8 AutoSet Vantage™	STANDARD	STANDARD	STANDARD	ULTRA	ACTIVA	ACTIVA	ACTIVA	STANDARD	STANDARD	N/V	N/V	N/V
S8 Compact™	No setting required. Consult your clinician before changing to a different mask type.									N/V	N/V	N/V
S8 Elite™ / S8 Elite™ II	STANDARD	STANDARD	STANDARD	ULTRA	ACTIVA	ACTIVA	ACTIVA	STANDARD	STANDARD	N/V	N/V	N/V
S8 Escape™	Requires pressure calibration at 12 cm H2O. Consult your clinician and check the user guide.									N/V	N/V	N/V
S8 Escape™ II	Requires pressure calibration at 12 cm H2O. Consult your clinician and check the user guide.									N/V	N/V	N/V
S8 AutoSet™ II	STANDARD	STANDARD	STANDARD	ULTRA	ACTIVA	ACTIVA	ACTIVA	STANDARD	STANDARD	N/V	N/V	N/V
VPAP™ III	STANDARD	N/V	STANDARD	ULTRA	STANDARD	STANDARD	STANDARD	STANDARD	N/V	N/V	N/V	N/V
VPAP™ III ST-A	STANDARD	N/V	STANDARD	ULTRA	STANDARD	STANDARD	STANDARD	STANDARD	N/V	STANDARD	STANDARD	N/V
VPAP™ Adapt SV	VISTA¹	VISTA	VISTA¹	ULTRA	ACTIVA	ACTIVA	ACTIVA	VISTA	VISTA	N/V	N/V	N/V
VPAP™ Adapt (S9)	Nasal	Nasal	Nasal	Nasal Ultra	Nasal	Nasal	Nasal	Nasal	Nasal	N/V	N/V	N/V
VPAP™ Adapt (S9 ASV)	Nasal	Nasal	Nasal	Nasal Ultra	Nasal	Nasal	Nasal	Nasal	Nasal	N/V	N/V	N/V
VPAP™ Auto 25	STANDARD	STANDARD	STANDARD	ULTRA	ACTIVA	ACTIVA	ACTIVA	STANDARD	STANDARD	N/V	N/V	N/V
VPAP™ Auto (S9)	Nasal	Nasal	Nasal	Nasal Ultra	Nasal	Nasal	Nasal	Nasal	Nasal	N/V	N/V	N/V
VPAP™ Malibu	VISTA¹	MERIDIAN	MERIDIAN	ULTRA	N/V	N/V	N/V	VISTA	MERIDIAN	N/V	N/V	N/V
VPAP™ S / VPAP™ ST	STANDARD	STANDARD	STANDARD	ULTRA	ACTIVA	ACTIVA	ACTIVA	STANDARD	STANDARD	N/V	N/V	N/V
VPAP™ S (S9) / VPAP™ ST (S9)	Nasal	Nasal	Nasal	Nasal Ultra	Nasal	Nasal	Nasal	Nasal	Nasal	N/V	N/V	N/V
VPAP™ ST-A (S9 iVAPS)	Nasal	Nasal	Nasal	Nasal Ultra	Nasal	Nasal	Nasal	Nasal	Nasal	N/V	N/V	Pediatric
VPAP™ COPD (S9)	Nasal	Nasal	Nasal	Nasal Ultra	Nasal	Nasal	Nasal	Nasal	Nasal	N/V	N/V	Pediatric
S9 VPAP™ Tx	Nasal	Nasal	Nasal	Nasal Ultra	Nasal	Nasal	Nasal	Nasal	Nasal	N/V	N/V	Pediatric
VPAP™ Tx	NASAL	NASAL	NASAL	NASAL-ULTRA	NASAL	NASAL	NASAL	NASAL	N/V	NASAL	N/V	N/V
Lumis™ Tx	Nasal	Nasal	Nasal	Nasal Ultra	Nasal	Nasal	Nasal	Nasal	Nasal	N/V	N/V	Pediatric
Stellar™	NASAL	NASAL	NASAL	ULTRA	NASAL	NASAL	NASAL	NASAL	N/V	NASAL	N/V	Pediatric

Device	Mirage™ FX / Mirage™ FX for Her	Hospital Nasal	Mirage Micro™	Ultra Mirage™ II	Mirage Activa™	Mirage Activa™ LT	Mirage™ SoftGel	Mirage Vista™	Meridian	Mirage Kidsta™	Mirage Micro™ for Kids	Pixi™
VS Ultra™ / VS Integra™	N/V	N/V	N/V	N/V	N/V	N/V	N/V	N/V	N/V	N/V	N/V	N/V
VS III™	N/V	✓	✓	✓	✓	✓	N/V	✓	N/V	✓	N/V	N/V
Astral™	Nasal	Nasal	Nasal	Nasal Ultra	N/V	Nasal	Nasal	Nasal	N/V	Nasal	Nasal	Pediatric

¹ SmartStop may not operate as described when using this mask with some CPAP or bi-level devices.

N/V = The mask has not been validated for this device.

✓ = Compatible X = Not Compatible

Nasal Pillows

Device	AirFit™ P30i ¹	AirFit™ P10 For AirMini™	AirFit™ P10 / AirFit™ P10 for Her	Swift™ FX / Swift™ FX for Her	Swift™ FX Bella / Swift™ FX Bella Gray	Swift™ LT / Swift™ LT for Her	Mirage Swift™ II
AirCurve™ 11 ST / ST-A	N/V	X	Pillows ²	Pillows	Pillows	X	X
AirCurve™ 11 VPAP Tx	Pillows	X	Pillows ²	Pillows	Pillows	Pillows	Pillows
AirCurve™ 11 VAuto / AirCurve™ 11 S	Pillows	X	Pillows ²	Pillows	Pillows	Pillows	Pillows
AirCurve™ 11 ASV	Pillows	X	Pillows ²	Pillows	Pillows	Pillows	Pillows
AirSense™ 11	Pillows	X	Pillows	Pillows	Pillows	Pillows	Pillows
AirMini™	X	✓	X	X	X	X	X
AirStart™ 10 / AirSense™ 10	Pillows	X	Pillows ²	Pillows	Pillows	Pillows	Pillows
AirCurve™ 10 VAuto / AirCurve™ 10 S / AirCurve™ 10 ST	Pillows	X	Pillows ²	Pillows	Pillows	Pillows	Pillows
AirCurve™ 10 ASV	Pillows	X	Pillows ²	Pillows	Pillows	Pillows	Pillows
AirCurve™ 10 ST-A	Pillows ⁴	X	Pillows ²	Pillows	Pillows	Pillows	Pillows
S9 AutoSet™ / S9 AutoSet™ for Her / S9 Elite™	Pillows	X	Pillows	Pillows	Pillows	Pillows	Pillows
S9 Escape™ / S9 Escape™ Auto	Pillows	X	Pillows	Pillows	Pillows	Pillows	Pillows
C-Series Tango™	N/V	X	No setting required. Consult your clinician before changing to a different mask type.				
S7 Elite™	N/V	X	MIRAGE	MIRAGE	MIRAGE	MIRAGE	MIRAGE
S8 AutoSet Vantage™	N/V	X	SWIFT	SWIFT	SWIFT	SWIFT	SWIFT
S8 Compact™	N/V	X	No setting required. Consult your clinician before changing to a different mask type.				
S8 Elite™ / S8 Elite™ II	N/V	X	SWIFT	SWIFT	SWIFT	SWIFT	SWIFT
S8 Escape™	N/V	X	Requires pressure calibration at 12 cm H2O. Consult your clinician and check the user guide.				
S8 Escape™ II	N/V	X	Requires pressure calibration at 12 cm H2O. Consult your clinician and check the user guide.				
S8 AutoSet™ II	N/V	X	SWIFT	SWIFT	SWIFT	SWIFT	SWIFT
VPAP™ III	N/V	X	MIRAGE ²	MIRAGE	MIRAGE	MIRAGE	MIRAGE
VPAP™ III ST-A	N/V	X	MIRAGE ²	MIRAGE	MIRAGE	MIRAGE	MIRAGE
VPAP™ Adapt SV	N/V	X	N/V	N/V	N/V	N/V	N/V
VPAP™ Adapt (S9)	Pillows	X	Pillows ²	Pillows	Pillows	Pillows	Pillows
VPAP™ Adapt (S9 ASV)	Pillows	X	Pillows ²	Pillows	Pillows	Pillows	Pillows
VPAP™ Auto 25	N/V	X	SWIFT ²	SWIFT	SWIFT	SWIFT	SWIFT
VPAP™ Auto (S9)	Pillows	X	Pillows ²	Pillows	Pillows	Pillows	Pillows
VPAP™ Malibu	N/V	X	SWIFT ²	SWIFT	SWIFT	SWIFT	SWIFT
VPAP™ S / VPAP™ ST	N/V	X	SWIFT ²	SWIFT	SWIFT	SWIFT	SWIFT

Device	AirFit™ P30i ¹	AirFit™ P10 For AirMini™	AirFit™ P10 / AirFit™ P10 for Her	Swift™ FX / Swift™ FX for Her	Swift™ FX Bella / Swift™ FX Bella Gray	Swift™ LT / Swift™ LT for Her	Mirage Swift™ II
VPAP™ S (S9) / VPAP™ ST (S9)	Pillows	X	Pillows ²	Pillows	Pillows	Pillows	Pillows
VPAP™ ST-A (S9 iVAPS)	N/V	X	Pillows ²	Pillows	Pillows	Pillows	Pillows
VPAP™ COPD (S9)	N/V	X	Pillows ²	Pillows	Pillows	Pillows	Pillows
S9 VPAP™ Tx	Pillows ³	X	Pillows ²	Pillows	Pillows	Pillows	Pillows
VPAP™ Tx	N/V	X	PILLOWS ²	PILLOWS	PILLOWS	PILLOWS	PILLOWS
Lumis™ Tx	N/V	X	Pillows ²	Pillows	Pillows	Pillows	Pillows
Stellar™	N/V	X	PILLOWS ^{1,2}	PILLOWS ¹	PILLOWS ¹	PILLOWS ¹	PILLOWS ¹
VS Ultra™ / VS Integra™	N/V	X	N/V	N/V	N/V	N/V	N/V
VS III™	N/V	X	N/V	N/V	N/V	N/V	N/V
Astral™	N/V	X	N/V	Pillows	Pillows	Pillows	Pillows

¹ SmartStop may not operate as described when using this mask with some CPAP or bi-level devices.

² Accuracy of tidal volume/minute ventilation may reduce using the AirFit P10 mask in bilevel modes, particularly at higher pressure supports.

³ Excluding PAC and iVAPS modes. Only compatible for pressure range 4 - 25 cm H2O.

⁴ This combination of mask and device has been determined safe for use based on the evaluation of accuracy of monitoring of pressure, volume, minute ventilation and respiratory rate, and its alarms if applicable. Small variations in performance might be observed, particularly at higher pressure or flow. As per expected clinical practice, the effectiveness of ventilation and alarms should be verified by the clinician.

N/V = The mask has not been validated for this device.

✓ = Compatible X = Not Compatible