

BARIATRIC 4-IN-1 COMMUNE & SHOWER CHAIR LVA3105

vive
vivehealth.com




To see all FAQs
in one place,
visit vivehealth.link/nky

OWNER'S MANUAL

BARIATRIC 4-IN-1 COMMODE & SHOWER CHAIR

WHAT'S INCLUDED

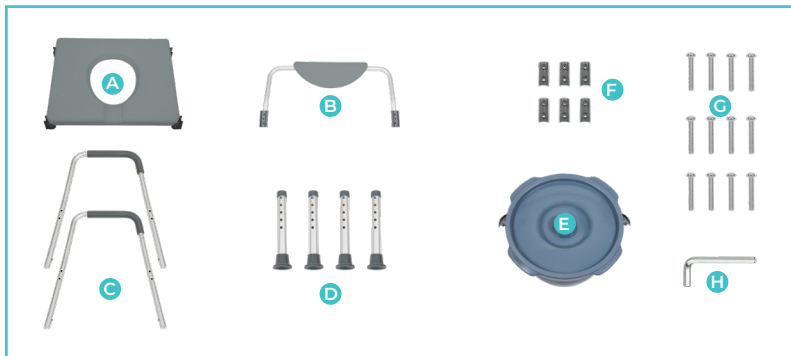

- Seat (A)
- Backrest Assembly (B)
- 2x Armrests (C)
- 4x Feet (D)
- Commode Bucket (E)
- 6x Brackets (F)
- 12x Screws (G)
- Hex key (H)

ASSEMBLY INSTRUCTIONS

1. Attach all four feet (D) to the bottom of each armrest (C). Repeat for both armrests.



2. Position the armrests (C) onto the seat base (A). Ensure the padded sections face forward. Use two brackets (F) and 4 screws (G) to secure the armrest into place. Repeat for the opposite side to complete your standing base.



3. Once both armrests are installed and upright, secure the backrest (B) assembly to the frame using the remaining brackets.



4. Slide the commode bucket underneath the seat. The handle should face the back of the chair.



5. Use the Push pins on each leg to adjust to your desired height. Make sure all four legs are placed at the same height.

WARNINGS ⚠

- Ensure feet are dry and free of soap residue before each use.
- Maximum weight capacity: 550 lbs (249 kg).
- Do not stand on the chair.
- Do not lean excessively on the backrest - this may cause tipping.
- Never attempt to stand from the chair onto a wet or slippery surface.
- Before each use, verify all four feet are locked at equal height.
- Periodically check all screws to ensure they are fully tightened.

FREQUENTLY ASKED QUESTIONS

QUESTION	ANSWER
Is the chair mold and mildew resistant?	Yes, the materials are resistant, but regular cleaning is recommended for hygiene and durability.
How should the chair be cleaned?	Wipe down the chair using mild soap, bathroom surface cleaner, or a disinfectant solution. Avoid abrasive cleaners.

CARE INSTRUCTIONS

- Rinse and dry thoroughly after use in wet environments.
- Store in a dry location when not in use.
- Do not expose to prolonged direct sunlight.
- Do not use harsh chemicals or bleach on any components.

WARRANTY INFORMATION

Vive Health Limited Warranty – 60 Days

Vive Health offers a 60-day limited warranty on the Bariatric 4-in-1 Commode & Shower Chair. This warranty covers defects in materials and workmanship under normal use and conditions.

What is Covered:

- Manufacturing defects or material failures.
- Replacement or repair at no cost if the product is deemed defective.

What is Not Covered:

- Normal wear and tear.
- Damage resulting from misuse, improper care, or unauthorized modifications.
- Accidental damage such as cracks, dents, or stains.
- Use outside of the product's intended function.

Submitting a Warranty Claim:

To initiate a claim, contact Vive Health Customer Support at:

- Web: service@vivehealth.com
- Tel. no. 1-800-487-3808

Provide the following:

- Proof of purchase (e.g., order confirmation or receipt)
- A clear description of the issue
- Photos or videos supporting your claim

NOTES

This image shows a full page of blank, lined paper. The paper has a light cream or off-white color. It features approximately 20 horizontal blue lines spaced evenly apart, typical of standard notebook paper. There are no margins, text, or other markings on the page.



HAVE MORE QUESTIONS?

Check out our list of Frequently Asked Questions at
[vhealth.link/nky](https://vivehealth.com/health.link/nky) for helpful answers.



And if that doesn't answer your question, our
customer service team would love to help!
Feel free to connect with them by phone,
e-mail, or chat on our website

service@vivehealth.com
1-800-487-3808
vivehealth.com

Distributed by



8955 Fontana Del Sol Way
Naples, FL 34109
Made in China