

**SUNSTRIDE ROLLATOR
ALL-TERRAIN EDITION**
MOB1093

vive®
MOBILITY



To see all FAQs in
one place, visit
vhealth.link/8w4

vivehealth.com

SUNSTRIDE ROLLATOR: ALL-TERRAIN EDITION

WHAT'S INCLUDED

- Rollator Frame
- Rollator Bag



ATTACHING/DETACHING FRONT WHEELS





Want to watch us do this?
Check out [**vhealth.link/bl**](https://vhealth.link/bl)
for an easy instructional video.

1. Fold the front wheel down from the folded up position and line the wheel forks hole up with the holes in the frame.



2. Insert the screw knob and lock the wheel into place. Repeat steps 1 and 2 for the other front wheel.

FOLDING/UNFOLDING INSTRUCTIONS

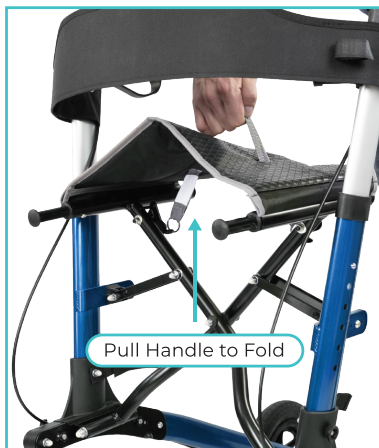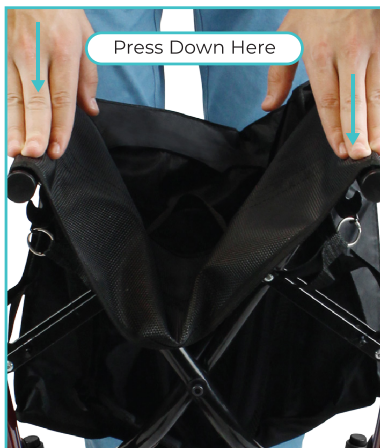

Straps have been provided around the seat area to keep the legs in place when in the folded position. To unfold the legs, undo the straps and move the leg down into a horizontal position and line up the hole on the leg with the hole on the central frame that connects the legs to the uprights. Place the provided locking twist knob screw into the hole, threading it from the outside inward. Repeat for the other leg.



INSERT THE UPRIGHT ARMS

Take the upright arm that corresponds with the brake line for the correct back wheel (AKA don't cross the brake lines) and place the upright into the opening at the top of the frame. Make sure the arm clicks into place using the locking pin and assess what height works best for you.



LOCKING THE BRAKES

The parking brake is used to help prevent the Rollator from moving when not in use, or to prevent it from moving while sitting or standing. To engage the parking brake, push the parking brake handle down until it locks into a diagonal position, locking it into place. To disengage the parking brake, simply lift up on the handle.



ADDING A BAG

Loop the Bag onto the front two extended tabs on the frame near the seat.



SPECIFICATIONS

- Product Dimensions:
Wheels: Front - 12", Rear - 10"
Height: 35" - 40"
Base: 32-26"
Seat: 18" - 10"
- Product Weight: 21lbs
- Max user weight: 300lbs

WARNING

- Do not use strong corrosive cleaner to clean the product's surface.
- Product should not be left outside in rainy conditions.
- Product may rust if regularly exposed to moisture.
- Before sitting down on the seat, please lock both brakes to prevent the rollator from sliding.
- Do not hang any heavy weighted items on either of the armrests as it will cause the rollator to be unstable.
- Please fold the rollator before storage.
- This item is meant to be a walking aid. This means that it is not built to support the full weight of ones body, it is only meant to assist you while walking. If a large amount of force is placed on this item, even when brakes are in locked position, this can cause the item to move out of its position.
- Perform weekly checks on your tire pressure as it becomes dangerous to use / sit on if tire pressure drops below 25psi. Keep tire pressure between 25-30psi at all times. With 35psi being the maximum fill limit.

NOTES:

[illegible]



HAVE MORE QUESTIONS?

Check out our list of Frequently Asked Questions at [vhealth.link/8w4](https://vivehealth.com/faq) for helpful answers.



And if that doesn't answer your question, our customer service team would love to help! Feel free to connect with them by phone, e-mail, or chat on our website

service@vivehealth.com
1-800-487-3808
vivehealth.com

Distributed by



8955 Fontana Del Sol Way
Naples, FL 34109
1-800-487-3808
www.vivehealth.com