

ANKLE SLEEVES WITH STRAPS

SUP3078

vive
vivehealth.com



To see all FAQs
in one place,
visit vhealth.link/i5a

OWNER'S MANUAL

ANKLE SLEEVES WITH STRAPS

WHAT'S INCLUDED

- 2x Ankle Sleeve
- 2x Ankle Sleeve Straps
- 1x Instruction Manual

INSTRUCTIONS FOR USE

Intended Use

The medical device is designed to provide support, compression, and/or immobilization for musculoskeletal injuries, post-operative recovery, and chronic conditions. It is intended to aid in the relief of pain, reduce swelling, and promote healing of specific body areas. The device may be used under the guidance of a healthcare professional for rehabilitation, pain management, or daily support.

User Profile

The primary users of bracing and splints are adults ages 18+, healthcare professionals, including physicians, orthopedic specialists, physical therapists, and other qualified practitioners. Secondary users may include patients or caregivers, who are instructed by healthcare professionals on the proper application and use of the devices.

Environment

The medical devices are intended for use in various healthcare settings. Suitable for both indoor and outdoor applications within typical residential, commercial, and occupational settings, addressing the requirements of daily living scenarios.

APPLICATION INSTRUCTIONS

1. Slide the compression sleeve onto your desired ankle.
2. Secure the strap by attaching the hook on the strap to the loop on the sleeve under your foot's arch. (Straps are optional.)



3. After attaching, stand to hold the strap in place as you wrap the strap around your ankle.



4. Pull the strap diagonally across your ankle, wrap it around your leg, and attach it to the loop patch on the front of the sleeve.



5. Repeat on the other side, crossing the strap to form an 'X' on the front of the sleeve.



Performance Characteristics

Main purpose: Stabilize and compress the Ankle joint.

Compression: Adjustable compression through strategic strap placement.

Targeted application: Foot/Ankle

Portability: Lightweight and compact design for easy transport.

Contraindications

- Do not use if you have circulatory problems without consulting a doctor.
- Do not use over open wounds or infected areas.
- Do not use if you have known allergies to the materials.
- Discontinue use if numbness or tingling occurs.
- Consult healthcare provider before use if you have diabetes.

CARE INSTRUCTIONS

- Machine Wash Sleeve only.
- Tumble dry on low.
- Hand wash included straps with cold water and mild detergent.

WARNINGS

- Consult your doctor before using.
- Stop use if pain occurs or worsens.
- Assistance may be required during the application of this support.

- Do not over tighten, doing so may lead to restricted blood circulation.
- If you experience any pain, swelling, sensation changes, or any unusual reaction while using this product, please consult your doctor immediately.

TECHNICAL SPECIFICATIONS

Materials: 60%Nylon, 20%Rubber, 20%Spandex

Latex Free: Yes

MANUFACTURE AND PRODUCT INFORMATION

Manufacturer



Vive Health

8955 Fontana Del Sol Way
Naples FL 34109

service@vivehealth.com

1-800-487-3808

vivehealth.com

EC	REP
----	-----

Compliance Management Solutions Europe

2 Bulgaria Str, 2850 Petrich, Bulgaria.

SIGNS AND SYMBOLS

	Medical device indication ISO/DIS 15223- 1:2020 Reference no. 5.7.7 Indicates the item is a Medical Device.
	CE marking Communauté Européenne Product traded on the extended Single Market in the European Economic Area (EEA); has been assessed to meet high safety, health, and environmental protection requirements.
	Consult instructions for use ISO 7000-1641 Indicates the need for the user to consult the Instructions for Use.
	Manufacturer ISO 7000-3082 To identify the manufacturer of the Medical Device.
	Date of manufacture ISO 7000-2497 To indicate the date on which the Medical Device was manufactured.
	Importer To identify the importer of the Medical Device.
	Caution ISO 15223-1, Clause 5.4.4 To indicate the need for the user to consult the Instructions for Use for important cautionary information such as warnings and precautions that cannot, for a variety of reasons, be presented on the Medical Device itself.
	SUP3078
	Indicates a medical device that has not been subjected to a sterilization process.
	Latex Free
	

IMPORTANT SAFETY NOTICE!!

If you experience any serious incident related to the use of this device, immediately discontinue use and report it to:

1. The manufacturer: Vive Health (8955 Fontana Del Sol Way, Naples FL 34109)
2. Contact your national health authority or medical device regulator.
3. Your healthcare provider.

A serious incident is any incident that directly or indirectly led, might have led, or might lead to:

- Death of a patient, user, or other person.
- Temporary or permanent serious deterioration of a patient's, user's, or other person's state of health.
- A serious public health threat .

For any questions or concerns that are not serious incidents, please contact Vive Health customer service at.

service@vivehealth.com

1-800-487-3808

vivehealth.com

NOTES

[illegible]



HAVE MORE QUESTIONS?

Check out our list of Frequently Asked Questions at [vhealth.link/i5a](https://vivehealth.com/health.link/i5a) for helpful answers.



And if that doesn't answer your question, our customer service team would love to help! Feel free to connect with them by phone, e-mail, or chat on our website

service@vivehealth.com
1-800-487-3808
vivehealth.com

Distributed by



8955 Fontana Del Sol Way
Naples, FL 34109
Made in China