

BARIATRIC CARBON FIBER ROLLATOR

MOB1112

vive®
MOBILITY



To see all FAQs in
one place, visit
vivehealth.link/m1g

vivehealth.com


BARIATRIC CARBON FIBER ROLLATOR

INSTRUCTIONS

OPEN THE LOCKING LATCH

When you receive your rollator it will be locked in the folded position.

Open the latch on the seat to unlock and unfold your rollator.

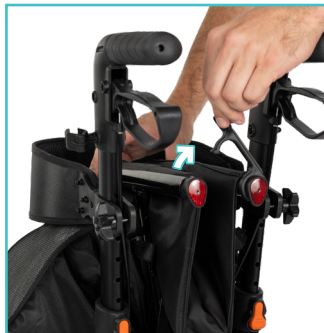

FOLDING AND UNFOLDING THE ROLLATOR

Pull up on the seat handle to unlock the latch and fold up your rollator.



Push down on the seat until you here a small click of the latch locking to unfold the rollator.



MOVING THE ARM SUPPORTS UP AND DOWN

Push in the orange locking button to unlock the lock that holds the arm supports in place. Move each arm support up and down making sure they are both placed at the same height.



ADD CANE HOLDER

Place the cane holder on the knob at the base of the Left Upright.



MOVING THE BACKREST UP AND DOWN

The backrest height can be adjusted according to your needs. Just twist the locking knob on each side to loosen the backrest and position it up or down as needed. Twist back to lock it in place.




ADD THE BAG

Place the bag under the seat using the bag holding pegs.



LOCKING THE BRAKES

Locking the brakes is an important feature of this rollator, especially when using the seat portion of the Rollator. To lock the brakes press DOWN on the brake lever on both sides until it clicks and holds in the downward position.

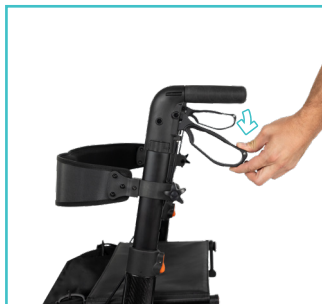

WARNINGS

- 450lbs weight capacity.
- Do not use strong corrosive cleaner to clean the product's surface.
- Product should not be left outside in rainy conditions. Product may rust if regularly exposed to moisture.
- Before sitting down on the seat, please lock both brakes to prevent the rollator from sliding.
- Do not hang any heavy weighted items on either of the armrests as it will cause the rollator to be unstable.
- This item is meant to be a walking aid. This means that it is not meant to support the full weight of one's body, it is only meant to assist you while walking. If a large amount of force is placed on this item, even when brakes are in locked position, this can cause the item to move out of its position.



HAVE MORE QUESTIONS?

Check out our list of Frequently Asked Questions at [vhealth.link/mfg](https://vivehealth.com/mfg) for helpful answers.



And if that doesn't answer your question, our customer service team would love to help! Feel free to connect with them by phone, e-mail, or chat on our website

service@vivehealth.com
1-800-487-3808
vivehealth.com

Distributed by



8955 Fontana Del Sol Way
Naples, FL 34109
Made in China