



# **BARIATRIC TRANSPORT WHEELCHAIR**

**Owner's Manual**  
**MOB1102**

[vivehealth.com](https://vivehealth.com)



To see all of the FAQs in one place  
visit [vhealth.link/wps](https://vhealth.link/wps)

## FOLDING / UNFOLDING THE CHAIR



For a video demonstration,  
check out **[vhealth.link/25c](https://vhealth.link/25c)**

From a stable position, grip the armrests or the chair frame under the sides of the seat and pull the frame away from the center of the chair until the fabric seat is fully unfolded.



## ATTACHING / REMOVING THE FOOT RESTS



For a video demonstration,  
check out the link here: **[vhealth.link/ey4](https://vhealth.link/ey4)**

Fit the foot rest over and down onto the two metal placement pins as shown.



Then rotate each foot rest forward until the silver clasp locks into place and the foot rest is securely fastened to the chair frame.



To remove the foot rests, pull the wide end of the silver clasp in towards the middle of the chair. You will feel the clasp release its grip on the frame, and the foot rest will swing out where it can be lifted off the two metal placement pins.

### **Adjusting the Height of the Foot Rests**

Release the locking tab, then adjust the height by pressing in the locking pin and then releasing the pin at the desired height. Make sure you re-lock the locking tab and that your heights for each leg are at equal heights (unless you require a specific length on each leg rest).



## LOCKING THE HANDBRAKES

The parking brake is used to help prevent the Wheelchair from moving when not in use, or to prevent it from moving while sitting or standing. Apply the brakes by pushing outward on the brake lever until it locks in place. Pull back on the lever to release the lock.



## MAINTENANCE

### Tire Maintenance

Tires should be cleaned occasionally with a damp cloth. Replace if tires become severely worn or cracking appears. Front casters should be checked periodically for cracking and lubricated occasionally. Rear wheels should be inspected at least once a month to ensure proper tightness.

### General Maintenance

Wipe metal parts with a soft cloth at least once a week. All aluminum parts can be cleaned with auto wax or similar products. If exposed to moisture, dry immediately. Do not use an abrasive cleanser that will scratch the finish.

## CAUTION

Please note that this chair is designed to use for transporting another individual and is not meant for self transportation. This Transport Chair is designed to safely be a stable center of gravity but it is important to try and keep a stable center of gravity of all times while performing activities that may change your center of gravity or a passengers center of gravity. Never reach out so far that is would require you to shift in the seat or bend over out of the chair.

When you have to reach for objects that require you to lean, be certain your front wheels are pointing forward as this will stabilize the chair.

## Ramps

Be sure of your own capabilities, limitations and strength before attempting to push another individual up a ramp. Correct techniques on ramp negotiation should be shown to you by a qualified professional. If you must stop on an incline, avoid any and all sudden or abrupt movement when you resume movement. Because of backward angle of the chair, sudden abrupt movement could cause instability. Do not attempt to change direction when moving in incline or decline, this could result in tipping.

### WARNING

- Wheelchair will support up to a 450 lb individual. Do not exceed chair weight limit.
- Do not attempt to use wheelchair without reading carefully all instructions in this manual.
- This wheelchair has not been approved as a seat that can be used within vehicles of any kind.
- Always transfer wheelchair user to a vehicle manufacturers approved seat and use safety restraints available.
- Do not operate this wheelchair on roadways, streets or any surfaces with vehicular traffic.
- Do not attempt to operate chair on inclines without proper assistance.
- Wheelchair should be on a smooth, stable and level area with the hand brakes engaged by the transporter before transferring the passenger in or out of the wheelchair.
- When transferring in or out of wheelchair, do not stand on or apply weight to foot rests.
- Do not lean over or apply weight to the top of wheelchair back. This can cause wheelchair to tip over.
- Do not stand on wheelchair.
- Abruptly changing directions while going down an incline could cause instability and/or increase the possibility of tipping over.
- The armrests are removable by pressing in the locking pin and pulling up on the armrests. Only use this function while the brakes are applied. Never move around in the chair without both armrests locked into the chair.

## HAVE MORE QUESTIONS?

Check out our list of Frequently Asked Questions at **vhealth.link/wps** for helpful answers.



And if that doesn't answer your question, our customer service team would love to help! Feel free to connect with them by phone, e-mail, or chat on our website.



**service@vivehealth.com**



**1-800-487-3808**



**vivehealth.com**

Distributed by

**vive  
health**

8955 Fontana Del Sol Way  
Naples, FL 34109  
Made in China