

BENDABLE LOOFAH INSTRUCTIONS

REPLACING THE LOOFAHS (No tools required)

- Remove the used loofah tip by unscrewing and then discarding. Screw the new loofah tip on and hand tighten. No tool required.
- We recommend replacing the loofah every 2-3 months.

Care Instructions: Rinse well after each use. Shake off any excess water. Hang to dry in a well-ventilated area.



BENDING THE HANDLE

Readjustments may be required after extended use.

Option 1: Using hands.

Option 2: Using a countertop.

1. Position the middle part of the handle onto a firm straight edge such as a countertop, table, or the seat of a chair. One half should be supported on the flat surface and the other half should be hanging.
2. While holding the flat end down with one hand, bend the opposite end to the desired angle.

