



CARBON FIBER ROLLATOR

Owner's Manual
MOB1086

vivehealth.com



To see all of the FAQs in one place
visit vhealth.link/jpc

INSTRUCTIONS

Open the Locking Latch

When you receive your rollator it will be locked in the folded position.



Open the latch on the seat to unlock and unfold your rollator.



Folding and Unfolding the Rollator:

Pull up on the seat handle to unlock the latch and fold up your rollator.



Push down on the seat until you here a small click of the latch locking to unfold the rollator.



Moving the arm supports up and down:

Pull up on the silver locking latch on each arm to unlock the arm supports.



Move the armrests to the desired position. When lined up correctly, the line below the height adjustment number on the arm will sit just above the locking latch for each position (notice the image below for height adjustment "5"). You will also hear a small click when the hole is lined up for the correct locking position. Once you have found your desired height, close down the silver locking tab.



Moving the Backrest up and down:

The backrest height can be adjusted according to your needs. Just twist the locking knob on each side to loosen the backrest and position it up or down as needed. Twist back to lock it in place.



Add the Bag

Place the bag under the seat using the bag holding pegs.



LOCKING THE BRAKES

Locking the brakes is an important feature of this rollator, especially when using the seat portion of the Rollator. To lock the brakes press **DOWN** on the brake lever on both sides until it clicks and holds in the downward position.




WARNINGS



- 300lbs weight capacity.
- Do not use strong corrosive cleaner to clean the product's surface.
- Product should not be left outside in rainy conditions. Product may rust if regularly exposed to moisture.
- Before sitting down on the seat, please lock both brakes to prevent the rollator from sliding.
- Do not hang any heavy weighted items on either of the armrests as it will cause the rollator to be unstable.
- This item is meant to be a walking aid. This means that it is not meant to support the full weight of one's body, it is only meant to assist you while walking. If a large amount of force is placed on this item, even when brakes are in locked position, this can cause the item to move out of its position.

HAVE MORE QUESTIONS?

Check out our list of Frequently Asked Questions at **vhealth.link/jpc** for helpful answers.

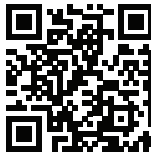

And if that doesn't answer your question, our customer service team would love to help! Feel free to connect with them by phone, e-mail, or chat on our website.



service@vivehealth.com



1-800-487-3808



vivehealth.com

Distributed by

**vive
health**

8955 Fontana Del Sol Way
Naples, FL 34109
Made in China