

CARBONGLIDE ROLLATOR

MOB1124

vive®
MOBILITY



To see all FAQs in
one place, visit
vivehealth.link/8mu

vivehealth.com

CARBONGLIDE ROLLATOR

WHAT'S INCLUDED

- A. 1x Rollator Frame (with Uprights attached)
- B. 1x Adjustable Backrest
- C. 1x Bag

INSTRUCTIONS FOR USE

Product Description


The Vive CarbonGlide is a lightweight rollator made from carbon fiber for users up to 300lbs.

ASSEMBLY INSTRUCTIONS

1. Unfold your (A) Rollator Frame by pushing down on the sides of the seat.
2. Thread each of the Uprights that are attached to the Rollator Frame through the holes on the adjustable backrest.



3. Insert the Uprights into the Rollator Frame and determine which height works best for you. Adjust the heights by pressing the black button at the top of the frame on each side. Make sure you set your heights evenly.

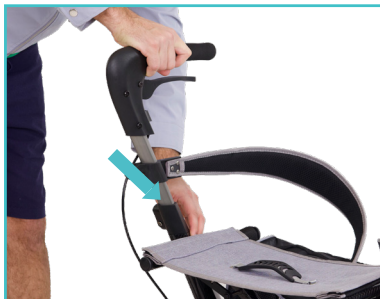

4. Adjust the height of your backrest by using the locking knobs on the side to loosen the clamp. Tighten the knobs back down when you find the right placement for the backrest.



5. Place the bag onto the front pegs of the rollator.



USING THE BRAKES

Use the brakes by pulling up on the brake lever of each handle. It is important to try and use both brakes at the same time so the flow of the brakes is even and the rollator don't spin.



Locking the Brakes:

Push down on each handle until the handle sticks into a locked position.

**FOLDING THE UNIT**

Pull up on the handle on the seat to fold the unit.

**WARNINGS**

- 300lbs weight capacity.
- Do not use strong corrosive cleaner to clean the product's surface.
- Product should not be left outside in rainy conditions
- Product may rust if regularly exposed to moisture.

- Before sitting down on the seat, please lock both brakes to prevent the rollator from sliding.
- Do not hang any heavy weighted items on either of the armrests as it will cause the rollator to be unstable.
- This item is meant to be a walking aid. This means that it is not meant to support the full weight of one's body, it is only meant to assist you while walking. If a large amount of force is placed on this item, even when brakes are in locked position, this can cause the item to move out of its position.

MAINTENANCE

- Please conduct regular inspections on any wires or screws on the product.
- To clean just wipe with damp rag, do not use any corrosive cleaning products.
- For indoor storage only

MANUFACTURER INFORMATION

Vive Health
8955 Fontana Del Sol Way
Naples FL 34109

service@vivehealth.com
1-800-487-3808
vivehealth.com

IMPORTANT SAFETY NOTICE

If you experience any serious incident related to the use of this device, immediately discontinue use and report it to:

1. The manufacturer: Vive Health
2. Contact your national health authority
3. Your healthcare provider

A serious incident is any incident that directly or indirectly led, might have led, or might lead to:

- Death of a patient, user, or other person
- Temporary or permanent serious deterioration of health
- A serious public health threat

For non-serious concerns, contact Vive Health customer service:

service@vivehealth.com
1-800-487-3808
vivehealth.com



HAVE MORE QUESTIONS?

Check out our list of Frequently Asked Questions at [vhealth.link/8mu](https://vivehealth.com/vhealth.link/8mu) for helpful answers.



And if that doesn't answer your question, our customer service team would love to help! Feel free to connect with them by phone, e-mail, or chat on our website

service@vivehealth.com
1-800-487-3808
vivehealth.com

Distributed by



8955 Fontana Del Sol Way
Naples, FL 34109
Made in China