

CATHETER LEG BAG HOLDER SLEEVES

LVA3078



To see all the FAQ's,
visit vhealth.link/ysv



QUICK START GUIDE

WHAT'S INCLUDED

- x2 Catheter Leg Bag Holder Sleeves
- x1 Mesh Wash Bag
- x1 Elastic securing strap

APPLICATION

Note:

The Catheter Leg Bag Holder Sleeves can be positioned around the calf or thigh depending on the needs of the user

Calf Application:

1. Begin to slide the sleeve up the leg, just under the knee.



2. Once positioned on the Calf properly, carefully begin to put your catheter bag into the pocket on the side of the sleeve.



vive

vivehealth.com

Distributed by



8955 Fontana Del Sol Way
Naples, FL 34109
1-800-487-3808
www.vivehealth.com

3. Ensure that the outlet port is pushed through the outlet port hole on the bottom of the Catheter Leg Sleeve.



4. Use the included strap to secure the sleeve at the top of the calf. It can be cut to size if needed.



Thigh Application:

1. Begin to slide the sleeve up the leg to the mid thigh.



2. Once positioned on the thigh properly, carefully begin to put your catheter bag into the pocket on the side of the sleeve.



3. Ensure that the outlet port is pushed through the outlet port hole on the bottom of the Catheter Leg Sleeve.



4. Use the included strap to secure the sleeve at the top of the thigh. It can be cut to size if needed.



Note:

Elastic strap can be cut to size. Cut in between the holes.



CARE

Machine wash with cold water. Tumble dry on low

WARNINGS ⚠

- Consult your doctor or physician before use.
- For external use only.
- Inspect regularly for wear and tear.
- Do not use if damaged
- Keep away from heat sources