

CORE CARBON FIBER ROLLATOR

MOB1132

vive®
MOBILITY



To see all FAQs
in one place,
visit vivehealth.link/2xs


vivehealth.com

CORE CARBON FIBER ROLLATOR


INSTRUCTIONS

FOLDING AND UNFOLDING THE ROLLATOR:

Pull up on the seat handle to unlock the latch and fold up your rollator.

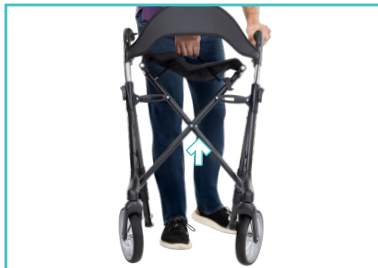

Push down on the seat until you hear a small click of the latch locking to unfold the rollator.

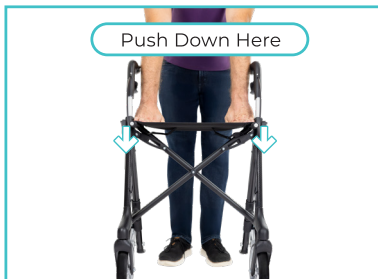

NOTE:

Your rollator may come with an elastic that holds the rollator in the closed position. This elastic will be located around the rear of the seat and will need to be removed before unfolding.

MOVING THE ARM SUPPORTS UP AND DOWN

Push in the locking button to unlock the lock that holds the arm supports in place. Move each arm support up and down making sure they are both placed at the same height.



ADD THE BAG

Place the bag under the seat using the bag holding pegs.



Installing the Backrest: Remove the two screws located just below the handles with the supplied allen key. Place one side to the backrest against the inside of upright arms. NOTE: The shape of the metal piece will fit under the plastic almost like a puzzle piece. Secure the metal piece back on with the screw that was previously removed. Do this for each side.



LOCKING THE BRAKES

Locking the brakes is an important feature of this rollator, especially when using the seat portion of the Rollator. To lock the brakes press DOWN on the brake lever on both sides until it clicks and holds in the downward position.



To use the brakes normally just pull up on the brake lever.

WARNINGS ⚠

- 300lbs weight capacity.
- Do not use strong corrosive cleaner to clean the product's surface.
- Product should not be left outside in rainy conditions. Product may rust if regularly exposed to moisture.
- Before sitting down on the seat, please lock both brakes to prevent the rollator from sliding.
- Do not hang any heavy weighted items on either of the armrests as it will cause the rollator to be unstable.
- This item is meant to be a walking aid. This means that it is not meant to support the full weight of one's body, it is only meant to assist you while walking. If a large amount of force is placed on this item, even when brakes are in locked position, this can cause the item to move out of its position.



HAVE MORE QUESTIONS?

Check out our list of Frequently Asked Questions at [vhealth.link/2xs](https://vivehealth.com/health.link/2xs) for helpful answers.



And if that doesn't answer your question, our customer service team would love to help! Feel free to connect with them by phone, e-mail, or chat on our website

service@vivehealth.com
1-800-487-3808
vivehealth.com

Distributed by



8955 Fontana Del Sol Way
Naples, FL 34109
Made in China