

CORE CARBON FIBER ROLLATOR

MOB1113

vive®
MOBILITY



To see all FAQs in
one place, visit
vivehealth.link/33t

vivehealth.com

CORE CARBON FIBER ROLLATOR

INSTRUCTIONS

FOLDING AND UNFOLDING THE ROLLATOR:

Pull up on the seat handle to unlock the latch and fold up your rollator.



Push down on the seat until you hear a small click of the latch locking to unfold the rollator.



MOVING THE ARM SUPPORTS UP AND DOWN:

Push in the locking button to unlock the lock that holds the arm supports in place. Move each arm support up and down making sure they are both placed at the same height.



ADD THE BAG

Place the bag under the seat using the bag holding pegs.



LOCKING THE BRAKES

Locking the brakes is an important feature of this rollator, especially when using the seat portion of the Rollator. To lock the brakes press **DOWN** on the brake lever on both sides until it clicks and holds in the downward position.



WARNINGS ⚠

- 275lbs weight capacity.
- Do not use strong corrosive cleaner to clean the product's surface.
- Product should not be left outside in rainy conditions. Product may rust if regularly exposed to moisture.
- Before sitting down on the seat, please lock both brakes to prevent the rollator from sliding.
- Do not hang any heavy weighted items on either of the armrests as it will cause the rollator to be unstable.
- This item is meant to be a walking aid. This means that it is not meant to support the full weight of one's body, it is only meant to assist you while walking. If a large amount of force is placed on this item, even when brakes are in locked position, this can cause the item to move out of its position.

NOTES:

This image shows a single sheet of white paper with horizontal blue ruling lines. The lines are evenly spaced and run across the width of the page. There are approximately 20 lines visible. The paper has a slight shadow on the right side, suggesting it's resting on a surface.



HAVE MORE QUESTIONS?

Check out our list of Frequently Asked Questions at [vhealth.link/33t](https://vivehealth.com/33t) for helpful answers.



And if that doesn't answer your question, our customer service team would love to help! Feel free to connect with them by phone, e-mail, or chat on our website

service@vivehealth.com
1-800-487-3808
vivehealth.com

Distributed by



8955 Fontana Del Sol Way
Naples, FL 34109
Made in China