

COLD COMPRESSION BANDAGE

RHB3054BLU

ORIGINAL FORMULA
DR. PAT'S



To see all the FAQs
in one place,
visit vhealth.link/g6k

OWNER'S MANUAL

COLD COMPRESSION BANDAGE

WHAT'S INCLUDED

- 1x Cooling wrap (4" x 10.5')
- 2x Wrap hooks



INSTRUCTIONS FOR USE

Product Description

The Cold Compression Bandage is designed to provide immediate cooling relief and compression therapy for acute injuries, sprains, strains, and post-exercise recovery. The wrap activates upon application, requiring no refrigeration, making it ideal for on-the-go use and first aid applications. Please note that the cooling effect can only be used once, though the wrap can continue to be used afterwards for compression support.

Intended Use

Cold packs for hot or cold therapy are designed to provide targeted relief for muscle aches, injuries, and inflammation. The primary purpose is to alleviate pain, reduce swelling, and promote recovery for individuals experiencing muscle discomfort or injuries.

User Profile

This device is intended for adult users seeking pain relief and inflammation reduction through cold therapy and compression. Suitable for athletes, active individuals, and those recovering from minor injuries or experiencing muscle soreness.

Environment

The medical devices are intended for use in various healthcare settings. Suitable for both indoor and outdoor applications within typical residential, commercial, and occupational settings, addressing the requirements of daily living scenarios.

Performance Characteristics

Main purpose: Provides one-time cooling effect through evaporation technology combined with compression for pain relief and swelling reduction.

Compression: Yes, adjustable compression for targeted support.

Targeted application: Can be applied to various body parts requiring cold therapy and compression.

Adjustability: Flexible design with hooks for secure attachment and custom fit.

Portability: Lightweight and compact design for easy transport and storage, ideal for on-the-go use.



CONTRAINDICATIONS

- Do not use if you have known sensitivity to menthol or borneol.
- Do not use on broken or damaged skin.
- Do not use if you have circulation problems.
- Do not use if you have decreased skin sensitivity.
- Avoid use on areas with recent trauma or acute inflammation without medical consultation.

HOW TO USE

1. If needed, the bandage can be cut to a shorter length for better fit.
2. Gently wrap the bandage around the affected area, ensuring even coverage.



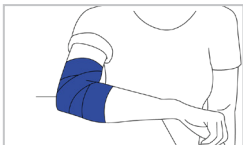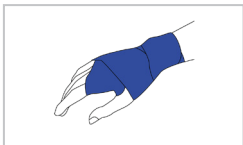

3. Use the provided hooks to secure the bandage in place.



4. Adjust compression level for comfort should be snug but not too tight.
5. Wash hands thoroughly after applying the wrap.
6. Leave in place for up to 20 minutes or as directed by a healthcare provider.
7. Remove if you experience any discomfort or skin irritation.
8. Note: After first use, the cooling effect will be depleted, but the wrap can still be used for compression support.

SPECIAL STORAGE & HANDLING CONDITIONS

- Store in a cool, dry place away from direct sunlight.
- Keep in original packaging when not in use.

- Do not expose to extreme temperatures
- Keep out of reach of children.
- Product has a shelf life of 2 years from date of manufacture.

WARNINGS

- Do not overtighten.
- Stop use if pain worsens.
- Do not use on an open wound.
- Consult doctor before use.
- If you experience any swelling, discomfort, or pain, stop use immediately.
- Do not use for longer than recommended duration.
- Do not apply additional heat sources over the bandage.
- Avoid contact with eyes.
- For external use only.
- Do not ingest the cooling ingredients.
- Do not microwave.
- Do not put in refrigerator.
- The cooling effect is single-use only and cannot be reactivated.
- After the cooling effect is depleted, the wrap can still be used for compression.

MANUFACTURE AND PRODUCT INFORMATION

Manufacturer



Vive Health

8955 Fontana Del Sol Way
Naples FL 34109
service@vivehealth.com
1-800-487-3808
vivehealth.com



Compliance Management Solutions Europe

2 Bulgaria Str,
2850 Petrich, Bulgaria.

SIGNS AND SYMBOLS

	Manufacturer ISO 7000-3082 To identify the manufacturer of the Medical Device.
	Date of manufacture ISO 7000-2497 To indicate the date on which the Medical Device was manufactured.
	Importer To identify the importer of the Medical Device.
	Country of Manufacture To identify the country manufacture of products.
	Indicates a medical device that has not been subjected to a sterilization process.
	Date of manufacture ISO 7000-2497 To indicate the date on which the Medical Device was manufactured.
	Medical device indication ISO/DIS 15223- 1:2020 Reference no. 5.7.7 Indicates the item is a Medical Device.
	 (01)00818323021493(241)SLUP1080WH1
	Indicates a medical device that needs protection from light sources.

IMPORTANT SAFETY NOTICE!!

If you experience any serious incident related to the use of this device, immediately discontinue use and report it to:

1. The manufacturer: Vive Health (8955 Fontana Del Sol Way, Naples FL 34109)
2. Contact your national health authority or medical device regulator
3. Your healthcare provider

A serious incident is any incident that directly or indirectly led, might have led, or might lead to:

- Death of a patient, user, or other person
- Temporary or permanent serious deterioration of a patient's, user's, or other person's state of health
- A serious public health threat

For any questions or concerns that are not serious incidents, please contact Vive Health customer service at: service@vivehealth.com 1-800-487-3808 vivehealth.com

ORIGINAL FORMULA
DR. PAT'S

HAVE MORE QUESTIONS?

Check out our list of Frequently Asked Questions at
vhealth.link/g6k for helpful answers.



And if that doesn't answer your question, our
customer service team would love to help!
Feel free to connect with them by phone,
e-mail, or chat on our website

service@vivehealth.com
1-800-487-3808
vivehealth.com

Distributed by

**vive
health**

8955 Fontana Del Sol Way
Naples, FL 34109
Made in China