

EASYRINSE SHOWER CHAIR

LVA3095

vive
vivehealth.com



To see all FAQs
in one place,
visit vivehealth.link/9y9

OWNER'S MANUAL

EASYRINSE SHOWER CHAIR

INSTRUCTIONS



Want to watch us do this?
Check out vhealth.link/k36
for an easy instructional video.

STEP 1

Install both of the H-shaped upper leg frames to the bottom side of the Seat Base. Make sure the pins on each leg depress into the hole locking them in place.



STEP 2

Install the 4 leg pieces onto the bottoms of the Upper Leg Frames that were installed in the first step. Make sure the foot shape on each leg is facing outward from the center.



STEP 3

Install bracket onto base.

**NOTE:**

For Storage easily lift up the front of the seat to fold the chair.



WARNING

- Make sure Shower Chair feet are wiped clean and dry from shower scum before each use.
- Maximum User Weight: 300lbs
- Never try to stand up from the chair onto a wet surface.
- Do not stand on a this device for any reason.
- Do not lean over or apply weight to the top of the chair back. This can cause the chair to tip over.
- Do periodically checks on all bolts to make sure they are still firmly tightened.
- Make sure all four feet are locked in at the same height before each use.

FAQ's

Q: Is it mold and mildew resistant?

A: Yes, the chair is resistant to mold and mildew, but should still be cleaned to maximize longevity and hygiene.

Q: How do I clean it?

A: The chair can be wiped down with mild soap, an ordinary bathroom surface cleaner, or disinfectant. Also check to make sure water is draining properly through the holes in the rubber feet of the Leg Extensions, and isn't accumulating inside.

NOTES

[illegible]



HAVE MORE QUESTIONS?

Check out our list of Frequently Asked Questions
at vhealth.link/9y9 for helpful answers.



And if that doesn't answer your question, our
customer service team would love to help!
Feel free to connect with them by phone,
e-mail, or chat on our website

service@vivehealth.com
1-800-487-3808
vivehealth.com

Distributed by



8955 Fontana Del Sol Way
Naples, FL 34109
Made in China