

FLEECE CRUTCH PADS WITH HAND GRIPS

CSH1102BLK



To see all the FAQ's,
visit vhealth.link/sy5


vive
vivehealth.com

QUICK START GUIDE

INSTRUCTIONS


Step 1

Open up the fastening straps.

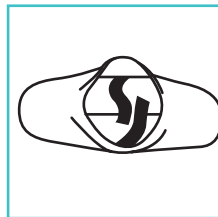


Step 2

Stretch cover over the crutches.

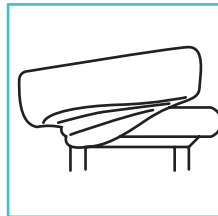

Step 3


Attach the fastening straps for
a snug fit.

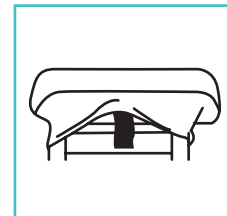

HAND GRIP INSTALLATION

Step 1:

Open up the fastening straps.

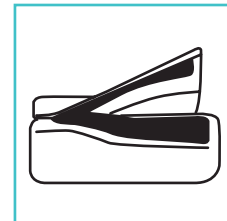

QUICK START GUIDE

vive
vivehealth.com


Distributed by

vive
health

8955 Fontana Del Sol Way
Naples, FL 34109
1-800-487-3808
www.vivehealth.com


Step 2:

Stretch cover over the crutches.

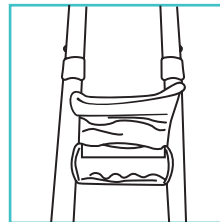

Step 3:

Attach the fastening straps for a snug fit.

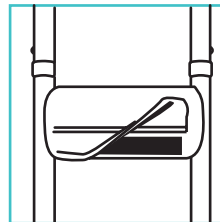

MAINTENANCE & CARE

- Machine wash in cold water on a gentle cycle.
- Use mild detergent.
- Do not use bleach.
- Tumble dry on low heat or hang to air dry.
- Do not iron.