


FULL ALTERNATING PRESSURE PAD SYSTEM

LVA3058BLK

vive
vivehealth.com

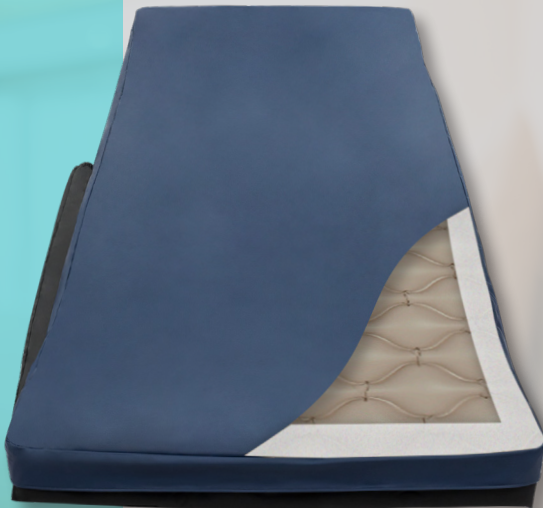

To see all FAQs
in one place,
visit vivehealth.link/i8t

OWNER'S MANUAL

FULL ALTERNATING PRESSURE PAD SYSTEM

WHAT'S INCLUDED

- 1x Pressure Pad
- 1x Inflation Tubes
- 1x Alternating Pressure Pump
- Patch Kit
- 1x Foam Frame

QUICK SETUP



Check out vhealth.link/fml for an easy instructional video.

1. Unroll the Foam Frame and lay it flat on your mattress frame or your floor. Then unroll the pressure pad and lay it flat inside the foam frame. Make sure the inflation ports are facing up and positioned at the head or foot of the bed, depending on where you plan to place the Pressure Pump.



2. Use the built in metal hooks to hang the pump on the bed frame. You may also lay the pump flat on a table or floor.



3. Connect the inflation tubes to the side of the pump and to the inflation ports on the pressure pad.



4. Plug the pump in.

5. Turn the Pump on and switch the static button on to speed up inflation process. Wait until the mattress has been fully inflated before turning the static switch off and using the pad.



6. Twist the variable pressure dial to adjust the level of firmness to your desired setting.



HOW IT WORKS



To see how it works
Check out vhealth.link/o9l

The Vive Alternating Pressure Pad has 130 bubble air cells. The air cells are divided into two different sections. Each section is connected separately to the pressure pump. The pressure pump automatically alternates between sections at 6 minutes intervals.

The pressure dial on the front of the pressure pump allows you to adjust the pad pressure from firm to soft. Regardless of the pressure setting the pump will always alternate in 6 minute intervals.

CLEANING INSTRUCTIONS

Pressure Pad/Pressure Hose

Use cloth with a mild detergent solution or a surface detergent/ disinfectant for cleaning the pad and hose.

REPAIR GUIDE



Want to watch us do this?
Check out **vhealth.link/mjd** for an easy instructional video.

SMALL PUNCTURES: Use the patch kit.

1. Fully deflate the unit.
2. Take the patch and add glue around the edges.
3. Firmly place the patch over the puncture.
4. Hold for 20 seconds until glue is dried.
5. Let stand for 30 minutes.

SPECIFICATIONS

Pump

- Interval Duration: 6 Mins.
- Air Output: 6-7 L/Min.
- Pressure Range: 35~130 mmHg
- Electrical Power Supply: 220V / 60Hz
- Foam Frame Dimensions: 79" x 34.6" x 4.3"

Mattress

- Dimensions: 36" x 78" x 3" (inflated)
- Materials: PVC (EN-71) 0.3mm

WARNING ⚠

- Weight Capacity: 300 lbs.
- Do not use corrosive cleaning products such as industrial degreasers or acetic solvents.
- Keep away from sharp objects as they may puncture the pressure pad.



HAVE MORE QUESTIONS?

Check out our list of Frequently Asked Questions at vhealth.link/i8t for helpful answers.



And if that doesn't answer your question, our customer service team would love to help! Feel free to connect with them by phone, e-mail, or chat on our website

service@vivehealth.com
1-800-487-3808
vivehealth.com

Distributed by



8955 Fontana Del Sol Way
Naples, FL 34109
Made in China