

INSTRUCTIONS FOR USE

VIVECARE Hydrogel Wound Gel

Product Description

Hydrogel wound gel is an amorphous hydrogel that gently rehydrates necrotic tissue, facilitating autolytic debridement while loosening and absorbing slough and exudate. It can clean the tissue surface, even when surfaces are difficult to access, such as deep wound cavities, fissures, and wound pockets. It can also be used to provide the optimal moist wound management environment during the later stages of wound closure.

Indications

Hydrogel wound gel is indicated as a primary wound filler for non to minimally draining partial wounds, including:

- Diabetic skin ulcers
- Venous stasis ulcers
- First and second degree burns
- Post-surgical incisions
- Cuts
- Abrasions

Intended Use

It is used to absorb wound exudate or discharge water to the wound and to cover the wound. It may also be used for the removal of necrotic tissue in wounds.

Function Principle

The hydroxyethyl cellulose combines with water to form a gel, which provides water to the wound and absorbs some of the seepage. Glycerin acts as a moisturizing ingredient, locking in moisture. Allantoin and schizophyllan can promote cell growth and speed wound healing. Sodium benzoate is used as a preservative to ensure that the product does not deteriorate during its shelf life.

Contraindications

1. Do not use on individuals with a known sensitivity to the gel or its components.
2. Treatment of any wound should be part of a well-defined plan and under the supervision of a health care professional.
3. Bleeding should be controlled before applying wound gel.
4. Do not use on severely draining wounds.
5. Do not use for infusion or injection.
6. Do not ingest.

Instructions for Use

1. Clean the wound with a sterile saline solution.
2. Apply wound gel directly to the wound at a 1/4 inch thickness, avoiding overlap onto the surrounding skin.
3. Apply a cover dressing that manages the wound drainage and maintains a moist wound bed.
4. Recommended to reapply every 24-72 hours. Change the dressing if drainage shows through the back.

Warnings

1. Consult a healthcare professional before use, especially for chronic wounds or if you have underlying health conditions. For single use only.
2. If signs of infection are detected, a qualified healthcare professional should be contacted to decide on the next course of action.
3. If signs of an allergic reaction are detected, discontinue use of the product and seek advice from a qualified healthcare professional.
4. Hydrogel wound gel must NOT be ingested and must be kept away from children and animals.

Storage

Store at room temperature away from light.

Distributed by:

VIVE HEALTH
8955 Fontana Del Sol Way Naples, FL 34109
1-800-487-3808
Made in China

Shelf Life:

Please see the expiration date on the packaging.