

INSTRUCTIONS FOR USE

VIVECARE Medical Paper Tape

DESCRIPTION

Medical Paper Tape is a lightweight, breathable, and hypoallergenic adhesive tape designed for securing dressings, tubing, or medical devices to the skin. Made from porous, medical-grade materials, it is ideal for use on sensitive skin and areas requiring frequent dressing changes.

INDICATIONS

- For external use only.
- Temporary use for securing bandages, dressings, or medical devices such as IV lines.
- Suitable for sensitive skin, including pediatric and elderly patients.

APPLICATION INSTRUCTIONS

1. Clip or trim hair if necessary. Clean and thoroughly dry the skin to remove any oil or lotion that could compromise adhesion.



2. Tear the tape to the desired length, minimizing contact with the adhesive side to preserve its effectiveness.



3. Apply the tape to the skin without stretching or creating tension.



4. Press firmly to activate the adhesive and ensure full contact with the skin.



REMOVAL INSTRUCTIONS

1. Loosen the edge of the tape. If it's difficult to grip, press a small piece of tape onto the corner to create a handle for lifting.



2. Stabilize the skin with one finger at the peel line to minimize tugging on the skin.



3. Remove the tape "low and slow," pulling it in the direction of hair growth and keeping the tape parallel to the skin surface. (This reduces skin irritation and hair pull.)

WARNINGS ⚠

- Do not apply to open wounds or broken skin.
- Do not encircle a limb completely with tape.
- Do not use the product after the expiration date.
- Discontinue use if irritation or allergic reactions occur, and consult a healthcare provider.
- Avoid pulling or stretching the tape during application to prevent skin trauma.
- Minimize contact with the adhesive surface to preserve adhesive effectiveness.
- Avoid gaps and wrinkles, which can allow moisture to seep between the tape and the skin, tubing, or dressing.
- When securing dressings, the tape should extend at least one inch beyond the edge of the dressing to ensure it stays in place.
- For single-use only; discard after use to maintain hygiene.

SYMBOLS

	Manufacturer
	Batch code
	Unique Device Identification
	Not made with natural rubber latex
	Temperature limit
	Keep dry
	Keep away from sunlight
	Do not use if package is damaged
	Single use
	Non sterile

STORAGE

Avoid storage in direct sunlight/fluorescent lighting. Keep in a cool (15–25°C or 59–77°F), dry and well-ventilated area.

SHELF LIFE

3 years from the date of manufacture.

Distributed by:
VIVE HEALTH
8955 Fontana
Del Sol Way
Naples, FL 34109
1-800-487-3808
Made in China