

MOBILIZATION BELT

RHB3019BLK

WHAT'S INCLUDED

- 1x Mobilization Strap
- 1x Removable Pad



To see all the FAQ's,
visit vhealth.link/o4i

vive
vivehealth.com

QUICK START GUIDE

INSTRUCTIONS



For a video demonstration,
visit vhealth.link/vw5

1. Position the mobilization strap around the patient's joints, such as the neck, shoulder, back, knee, leg, or ankle, ensuring that the soft pad is comfortably against the skin.



2. Adjust the length to achieve a snug yet comfortable fit. Once the desired length is attained, secure the strap using the quick push-button.





Distributed by



8955 Fontana Del Sol Way
Naples, FL 34109
1-800-487-3808
www.vivehealth.com

3. Proceed with the manual therapy session.

CARE

- Wipe clean with a damp cloth.
- Do not machine wash or dry.

WARNINGS

- This product is intended for use by trained healthcare providers. Ensure proper training before using this product, as incorrect use may result in injury to the patient or caregiver.
- Do not exceed weight limits of 250 lbs, as overloading can lead to equipment failure and potential harm to the patient.
- Inspect the product for signs of wear, tear, or damage before each use.

- Secure the product tightly around the patient's body to prevent slipping or shifting.
- Use caution when considering the limitations of patients.
- Avoid applying pressure to sensitive areas.
- Never leave patients unattended while using this product.
- Discontinue use of this product immediately if any discomfort occurs.