

SHOULDER IMMOBILIZER

SUP3049BLK

vive
vivehealth.com



To see all FAQs
in one place,
visit vivehealth.link/u82

OWNER'S MANUAL

SHOULDER IMMOBILIZER

APPLICATION (Assistance may be needed)

1. Place the arm sling with the pocket facing your chest.
2. Put the sling over your head and pull straps from the back of the sling to the front. Forming a figure eight like shape on the back of the user. This will prevent undesired abduction.
3. Secure the hooks to the front of the arm sling and or thread through the loops for added support. (These can be placed however is most comfortable to the user).



4. Add the strap around the body to secure the arm in place.



CARE

Hand wash with cold water and mild detergent, Hang dry.

WARNINGS ⚠

- Consult your doctor before use.
- Stop use if the pain worsens.
- Assistance may be required during application of this support.
- Do not overtighten, doing so may lead to restricted blood circulation.
- If you experience any pain, swelling, sensation changes, or any unusual reaction while using this product, please consult your physician immediately.



HAVE MORE QUESTIONS?

Check out our list of Frequently Asked Questions at vhealth.link/u82 for helpful answers.



And if that doesn't answer your question, our customer service team would love to help! Feel free to connect with them by phone, e-mail, or chat on our website

service@vivehealth.com
1-800-487-3808
vivehealth.com

Distributed by



8955 Fontana Del Sol Way
Naples, FL 34109
Made in China