

OSTOMY BAG ONE-PIECE

LVA3051

vivecare®



To see all FAQs
in one place,
visit vhealth.link/rng




OWNER'S MANUAL

OSTOMY BAG ONE-PIECE




BEFORE YOU BEGIN

- Gather supplies: ostomy bag, measuring guide, scissors, warm washcloth.
- Optional supplies: barrier ring, skin prep wipe, adhesive remover, pouch deodorant.
- Close the bag's outlet before starting to prevent leakage.

Hook & Loop Closure:

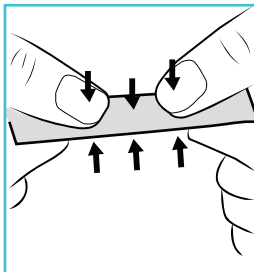
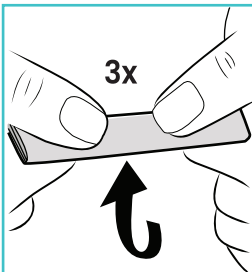
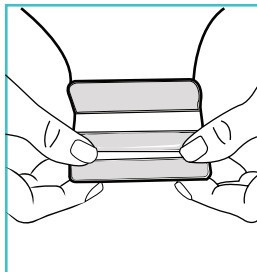


Clamp Closure:

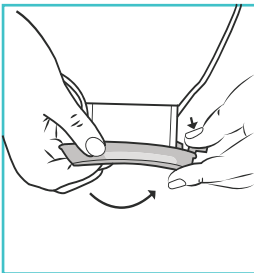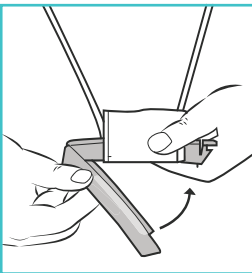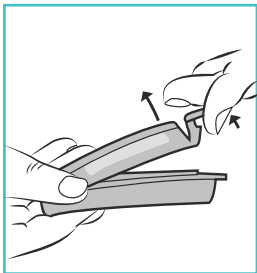


MEASURING YOUR STOMA

1. Use the included measuring guide card.
2. Size the opening about an 1/8 inch larger than your stoma.
3. Trace the size onto the skin barrier.
4. Cut the opening using ostomy or regular scissors.

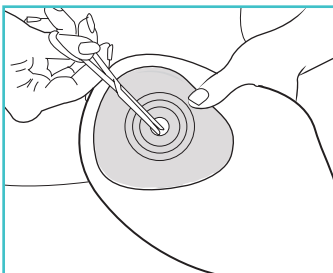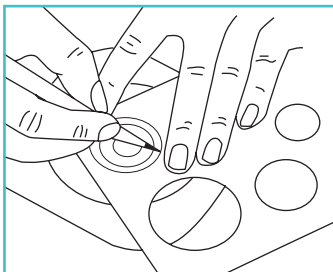
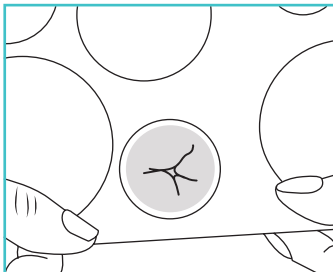


APPLYING THE OSTOMY BAG

1. Clean stoma area with warm water and mild soap.
2. Dry thoroughly.

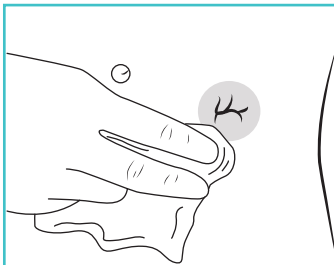

3. If needed, apply skin protective film:
 - Apply to clean, dry skin.
 - Let dry completely.
 - Helps protect skin from adhesive and output.
4. Center the opening and apply gentle pressure around the barrier for 60 seconds to ensure proper adhesion.

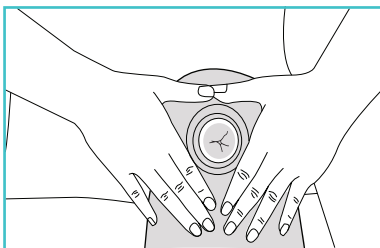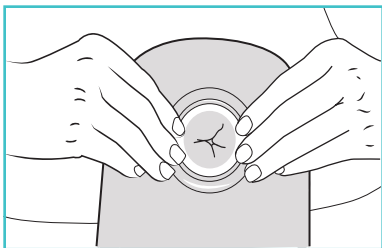

USING A BARRIER RING (IF NEEDED)

Use if you have:

- Redness around stoma
- Uneven skin surface
- Need custom fit
- Stool leakage concerns

Apply ring either:

- Around stoma first, then apply bag, OR
- On bag first, then apply entire system

DAILY MANAGEMENT

Drainage

- Empty when 1/3 full.
- Do not exceed half full as weight can compromise seal.
- Position over toilet with clear view.
- Have toilet paper ready.
- Clean outlet opening (1 inch inside and outside) before resealing.
- Add pouch deodorant if needed for odor control.

Regular Checks


- Monitor adhesive seal.
- Check for any leakage.
- Inspect stoma and surrounding skin.
- Empty regularly before pouch gets too full.

WEAR TIME & CHANGING

- Change every 3-5 days depending on output.
- Change sooner if:
 - Leakage occurs
 - Burning or itching develops
 - Barrier begins lifting

REMOVING THE BAG

1. Hold skin with one hand.
2. Slowly peel adhesive with other hand.
3. Use adhesive remover wipe if needed.
4. Clean area with warm washcloth.
5. Ensure skin and stoma are completely clean.
6. Measure stoma before applying new bag (size can change).

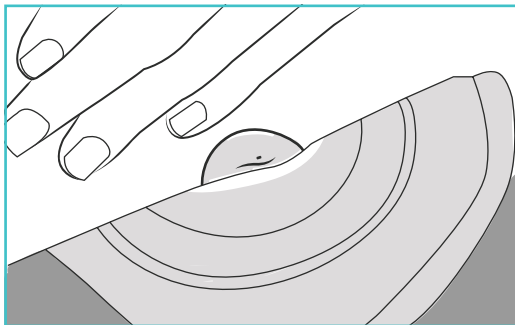

IMPORTANT NOTES

- One-piece bags cannot be reused once detached from skin.
- Empty bag contents into toilet
- Dispose of used bag in trash (not in toilet)
- Use barrier ring if seeing redness (indicates possible leakage)



WHEN TO USE SKIN PROTECTION

Apply skin protective film if you have:

- Sensitive skin
- Peristomal skin irritation
- Frequent barrier changes
- History of skin problems

WARNING SIGNS

Contact healthcare provider if:

- Persistent skin irritation
- Consistent leakage issues
- Changes in stoma size/appearance
- Unusual bleeding

CAUTIONS

- Apply only to clean, dry skin.
- Do not use oils or lotions as they affect adhesion.
- Replace barrier every 3-4 days.
- Do not fill beyond 1/2 capacity (strongly recommended to drain at $\frac{1}{3}$ capacity).
- Do not flush bag or recycle.
- Do not use if allergic to adhesives.
- Consult healthcare provider for skin irritation.

WARRANTY

30-day warranty against defects. Contact customer service with proof of purchase.

NOTES

This image shows a full page of blank, lined paper. It has a light cream or off-white background color. There are approximately 20 horizontal blue lines spaced evenly across the page, typical of standard notebook paper. The lines extend from the left edge to the right edge. There are no margins, text, or other markings on the page.

NOTES

This image shows a single sheet of white paper with horizontal blue ruling lines. The lines are evenly spaced and run across the width of the page. There are no margins, text, or other markings on the paper.

NOTES

This image shows a single sheet of white paper with horizontal blue ruling lines. The lines are evenly spaced and run across the width of the page. There are no margins, text, or other markings on the paper.



HAVE MORE QUESTIONS?

Check out our list of Frequently Asked Questions at
vhealth.link/rng for helpful answers.



And if that doesn't answer your question, our
customer service team would love to help!
Feel free to connect with them by phone,
e-mail, or chat on our website

service@vivehealth.com
1-800-487-3808
vivehealth.com

Distributed by



8955 Fontana Del Sol Way
Naples, FL 34109
Made in China