

INSTRUCTIONS FOR USE

VIVECARE Ostomy Adhesive Barrier Ring

PRODUCT DESCRIPTION


The Ostomy Adhesive Barrier Ring is made of medical-grade pressure-sensitive adhesive with carboxymethyl cellulose (CMC). It is designed to improve the fit of an ostomy pouching system, providing a secure seal, longer wear time, and added protection against leakage.

INDICATIONS



The Ostomy Adhesive Barrier Ring helps create a protective seal between the stoma and the appliance, preventing leakage and skin irritation. It is designed to conform to the skin, filling in uneven areas and ensuring a snug fit for better adhesion and comfort.

CARE INSTRUCTIONS


1. Clean the skin around the stoma thoroughly and ensure it is completely dry.

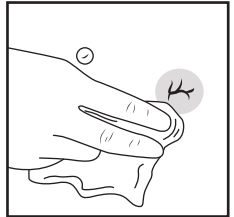


2. Peel off the release liner on the front and back.

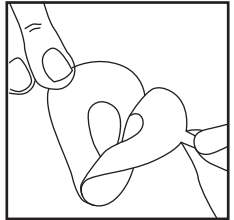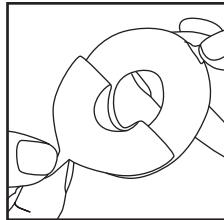


3. Stretch and shape the ostomy adhesive barrier ring to fit the size of the stoma.

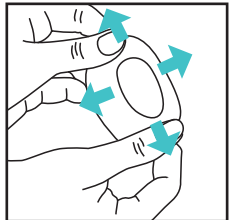

4. Carefully place the ostomy adhesive barrier ring around the stoma, ensuring a snug fit and a flat surface.

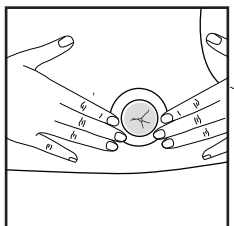

5. Place the ostomy bag over the stoma and press firmly around the edges to ensure a secure, leak-proof seal.

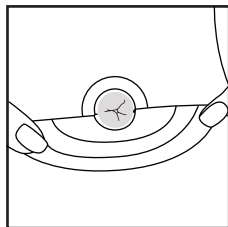

DISPOSAL

- Place used ostomy barrier rings in a sealed bag before disposal.
- Do NOT flush the product down the toilet.

FREQUENCY OF CHANGE

- Change the ring as needed, depending on output levels, leakage, or skin irritation.
- The product may remain in place for up to 7 days, but replace sooner if it becomes loose, soiled, or causes discomfort.

STORAGE

- Store in a cool, dry place, away from direct sunlight and humidity.
- Keep at room temperature (below 80% relative humidity) and in a well-ventilated area.
- Avoid exposure to corrosive gases or extreme heat.

ALLERGIES OR IRRITATION

- Do NOT use if you have known allergies to any of the ingredients in this product.
- If you experience redness, irritation, or skin discomfort, discontinue use and consult a healthcare professional.

WARNINGS

- Maintain proper hygiene when handling the ring to reduce the risk of infection.
- Do NOT reuse or clean a used ring—this may compromise adhesion and increase the risk of infection or leakage.
- Do NOT modify, cut, or alter the ring, as this may reduce its effectiveness.
- If irritation, discomfort, or signs of an allergic reaction occur, discontinue use immediately and consult a healthcare provider.
- Keep out of reach of children.

Distributed by:

VIVE HEALTH

8955 Fontana Del Sol Way Naples, FL 34109

1-800-487-3808

Made in China

140225