

OSTOMY BAG TWO-PIECE

LVA3052

vivecare®



To see all FAQs
in one place,
visit vhealth.link/810

OWNER'S MANUAL


OSTOMY BAG TWO-PIECE

BEFORE YOU BEGIN

- Gather supplies: skin barrier, ostomy bag, measuring guide, scissors, mild soap and water
- Optional supplies: barrier ring, skin prep wipe, stoma wipes, pouch deodorant, disposal bag


PREPARE THE WORK AREA

1. Wash your hands thoroughly.
2. Clean the stoma and surrounding skin with warm water and mild soap (avoid oils or lotions). Pat dry completely.

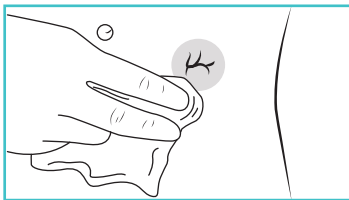


MEASURING YOUR STOMA

1. Use the included measuring guide card.
2. Size the opening about an 1/8 inch larger than your stoma.

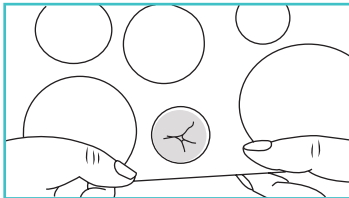


3. Trace the size onto the skin barrier.

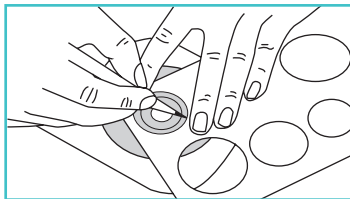


APPLYING THE SKIN BARRIER

1. Cut the opening about an 1/8in larger than your stoma using ostomy or regular scissors.


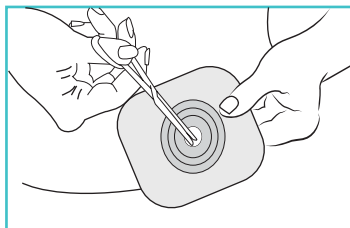


2. Remove the liner from the skin barrier.

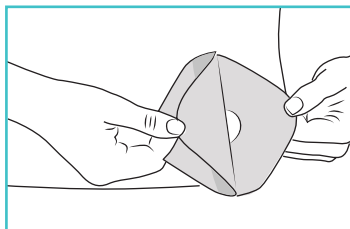

3. If needed, apply skin prep wipe and allow it to dry completely to protect skin from adhesive and stoma output.

- Center the skin barrier opening over the stoma. Apply gentle pressure from the bottom of the skin barrier upwards, and wait for about 60 seconds for optimal adhesion.


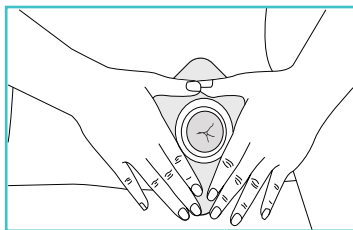
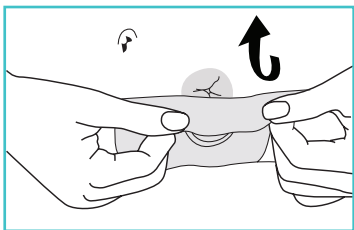


APPLYING THE OSTOMY BAG




1. Fasten the Bag Securely:

Ensure the pouch is clean and the closure at the bottom (if drainable) is securely fastened.

Hook & Loop Closure:

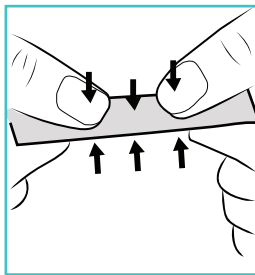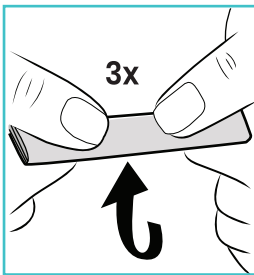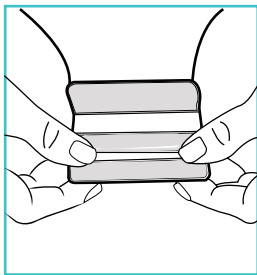


Clamp Closure:

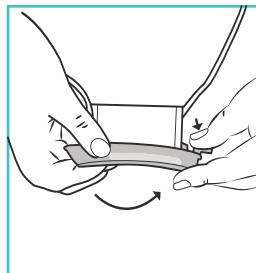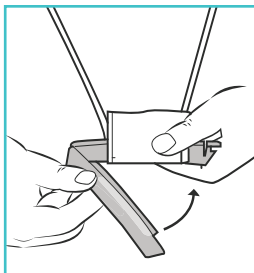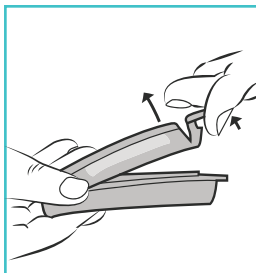

2. Align the Skin Barrier and Bag:

- Align the flange of the bag with the flange of the skin barrier.
- Press the bag onto the skin barrier from the bottom upwards, until it clicks or seals securely.

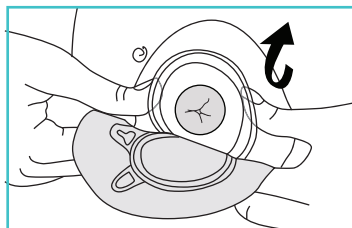

3. Check the Seal:

Gently tug on the pouch to ensure it is attached firmly.

USING A BARRIER RING (IF NEEDED)

Use if you have:

- Redness around stoma
- Uneven skin surface
- Need custom fit
- Stool leakage concerns

Apply ring either:

- Around stoma first, then apply skin barrier, OR
- On the adhesive side of the skin barrier first, then apply entire system

DAILY MANAGEMENT

Drainage

- Empty when 1/3 full.
- Do not exceed half full as weight can compromise seal.
- Position over toilet with clear view.
- Have toilet paper ready.
- Clean outlet opening (1 inch inside and outside) before resealing.
- Add pouch deodorant if needed for odor control.

Regular Checks


- Monitor adhesive seal.
- Check for any leakage.
- Inspect stoma and surrounding skin.
- Empty regularly before pouch gets too full.

WEAR TIME & CHANGING

- Change every 3-5 days depending on output.
- Change sooner if:
 - Leakage occurs
 - Burning or itching develops
 - Barrier begins lifting


REMOVING BAG

Remove the bag by pulling the tab on the ring.

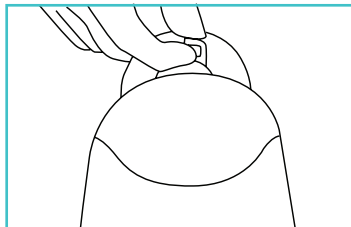

REMOVING THE SKIN BARRIER

1. Hold skin with one hand.
2. Slowly peel adhesive with other hand.
3. Use adhesive remover wipe if needed.
4. Clean area with warm washcloth.
5. Ensure skin and stoma are completely clean.
6. Measure stoma before applying new bag (size can change).

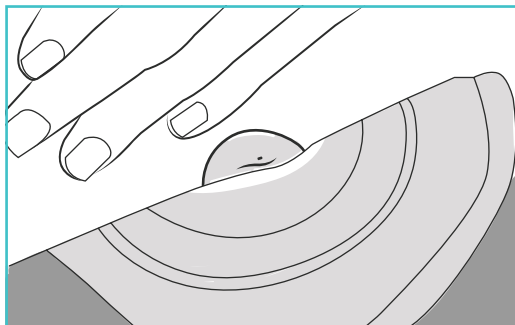

IMPORTANT NOTES

- Empty bag contents into toilet.
- Dispose of used bag in trash (not in toilet).
- Use barrier ring if seeing redness (indicates possible leakage)



WHEN TO USE SKIN PROTECTION

Apply skin protective film if you have:

- Sensitive skin
- Peristomal skin irritation
- Frequent barrier changes
- History of skin problems

WARNING SIGNS

Contact healthcare provider if:

- Persistent skin irritation
- Consistent leakage issues
- Changes in stoma size/appearance
- Unusual bleeding

CAUTIONS

- Apply only to clean, dry skin.
- Do not use oils or lotions as they affect adhesion.
- Replace barrier every 3-4 days.
- Do not fill beyond 1/2 capacity (strongly recommended to drain at $\frac{1}{3}$ capacity).
- Do not flush bag or recycle.
- Do not use if allergic to adhesives.
- Consult healthcare provider for skin irritation.

WARRANTY

30-day warranty against defects. Contact customer service with proof of purchase.

This image shows a full page of white paper with horizontal blue ruling lines. The lines are evenly spaced and run across the width of the page, providing a template for writing or drawing. There are no margins, text, or other markings on the page.

NOTES

This image shows a single sheet of white paper with horizontal blue ruling lines. The lines are evenly spaced and run across the width of the page. There are approximately 20 lines visible. The paper has a slight shadow on the right side, suggesting it's resting on a surface.



HAVE MORE QUESTIONS?

Check out our list of Frequently Asked Questions at vivehealth.link/810 for helpful answers.



And if that doesn't answer your question, our customer service team would love to help! Feel free to connect with them by phone, e-mail, or chat on our website

service@vivehealth.com
1-800-487-3808
vivehealth.com

Distributed by



8955 Fontana Del Sol Way
Naples, FL 34109
Made in China