

PORTABLE STAND ASSIST

LVA3045BLK



To see all FAQs in one place,
visit [vhealth.link/40s](https://vivehealth.link/40s)



QUICK START GUIDE

Thank you for your purchase. Please note your ez-sit was created to help assist you when trying to stand up from a chair that does not already have armrests to assist you. Please read and follow these simple rules below to help assist in your safety and maximize the best use for your new device.

First thing to note for your safety is hand placement. Please be cautious of where you place your hands as every chair is built differently. To maximize your safety, never place your hands at the very front of the armrests to help yourself stand up. Place your hands back just enough to make sure you have a full grip of the armrest, this also will better position your hands so that it best balances your ability to push off the device.

GOOD:



BAD:



Next you must always make sure your strap is tight and properly placed well for the type of chair you are using.

vive
vivehealth.com

Distributed by



8955 Fontana Del Sol Way
Naples, FL 34109
1-800-487-3808
www.vivehealth.com

CAUTION:

Never use the EZ-Sit portable stand assist if you have not secured your strap or checked to make sure your strap is still tightly placed in the best position for your specific chair.

Here is an example of a tightly placed strap on a straight back chair:



Here is an example of a strap that has been secured tightly underneath the chair as this chair's back is curved. This particular way of securing your device makes it the safest to use for this type of chair.



On your own personal chairs it might take a few attempts to figure out whether going behind or underneath the chair is best for that specific chair.

WARNING ⚠

Never use your EZ-Sit on a chair that doesn't fit the EZ-Sit's seat and or straps. If your device hangs over the edge of the chair then it presents a tipping hazard and is not safe to use.

**NOTE:**

We have included a stretch strap to help secure together the armrests while attempting to carry the device.

MAX USER WEIGHT:

275lbs

WARRANTY:

One Year Limited warranty