

# RECLINING WHEELCHAIR WITH HEADREST

MOB1103

**vive**®  
MOBILITY



To see all FAQs in  
one place, visit  
[vivehealth.link/ore](https://vivehealth.link/ore)

[vivehealth.com](https://vivehealth.com)

# RECLINING WHEELCHAIR WITH HEADREST

## INSTRUCTIONS



For a video demonstration,  
check out [vhealth.link/9aw](https://vhealth.link/9aw)

### TO UNFOLD

Push downward on the edges of the seat until the seat is flat.



### FOLDING

Pull up on the middle of the seat and then push the armrests inward.



## INSTALLING THE ACCESSORIES TO YOUR CHAIR

### INSTALL THE HEADREST

Add the top portion to the to the backrest of the chair so the velcro is on the back side.



Add the headrest cushion and run the straps to the back side and attach to the velcro.



### INSTALL THE BACKREST SUPPORT BAR

Place the bar onto the two threaded pipes protruding from the backrest, then install the lock knobs.



## INSTALL THE FOOTRESTS

Slide the footrests onto the two vertical pins on the front of the frame, below the seat. Rotate them inward so the silver locking piece clicks into place stabilizing the footrest.



## USING YOUR NEW CHAIR

### USING THE BRAKES

Push forward on the brake handle to lock the brakes.



## RECLINING THE BACKREST

Pull on both the rear levers that resemble brake handles to release the lock holding the backrest in place. Simultaneously and slowly move the backrest up or down anywhere from 0 to nearly 90 degrees.



## ELEVATING AND LOWERING THE LEG RESTS

To elevate the leg rest, simply push up on the leg rest for your patient to the desired height, or if your patient has the leg strength they can raise it with their foot.

To lower the leg rest carefully push outward on the leg rest level and slowly lower the leg rest down by hand.

**WARNING:** The lever completely releases the lock holding the leg rest in place, and it does not have any form of a hydraulic for a “slow drop”. If you’re not holding the leg rest it will completely drop at full speed back to the lowest position. Be careful when releasing the leg rest from higher positions.



## WARNINGS ⚠

- Wheelchair will support up to a 300 lb individual. Do not exceed the chairs weight limit.
- Do not attempt to use the wheelchair without reading carefully all instructions in this manual.
- This wheelchair has not been approved as a seat that can be used within vehicles of any kind.
- Always transfer wheelchair users to a vehicle manufacturers approved seat and use safety restraints available.
- Do not operate this wheelchair on roadways, streets or any surfaces with vehicular traffic.

- Wheelchair should be on a smooth, stable and level area with the hand brakes engaged by the transporter before transferring the passenger in or out of the wheelchair. When transferring in or out of the wheelchair, do not stand on or apply weight to foot rests.
- Do not lean over or apply weight to the top of the wheelchair back. This can cause the wheelchair to tip over.
- Do not stand on the wheelchair.
- Abruptly changing directions while going down an incline could cause instability and/or increase the possibility of tipping over.
- Always apply the hand brakes when stopped on a slope.
- Do not move the wheelchair while in the reclined position. Especially do not move the wheelchair while in the reclined position and on a slope. Having the chair reclined on a slope greatly increases the chance for tipping.
- When reclining a patient, always check the stability of the chair. If a patient has shifted slightly and is not balanced in the correct position of the chair it can become a tipping hazard especially if the chair is fully reclined.

## HAVE MORE QUESTIONS?

Check out our list of Frequently Asked Questions at  
[vhealth.link/ore](https://vhealth.link/ore) for helpful answers.



And if that doesn't answer your question, our  
customer service team would love to help!  
Feel free to connect with them by phone,  
e-mail, or chat on our website

[service@vivehealth.com](mailto:service@vivehealth.com)  
1-800-487-3808  
[vivehealth.com](https://vivehealth.com)

Distributed by

**vive**  
health

8955 Fontana Del Sol Way  
Naples, FL 34109  
Made in China