

RISE UP TOILET RAIL

LVA3082

vive
vivehealth.com




To see all FAQs in
one place,
visit vivehealth.link/lky

OWNER'S MANUAL

RISE UP TOILET RAIL

WHAT'S INCLUDED

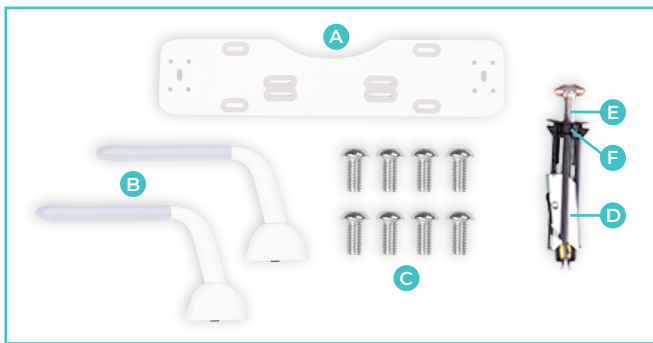

- (A) Flat Base Panel
- (B) 2x Swing Back Arms
- (C) 8x Small Screw
- (D) 2x Locking Insert
- (E) 2x Long Screw
- (F) 2x Washer



ASSEMBLY AND INSTALLATION INSTRUCTIONS



for an easy instructional video
Check out vhealth.link/9qz

NOTE:

Before starting the installation process, check your toilet seat to make sure it has a standard connection like what is seen in the images for step. The first step in the process will be to uninstall your current toilet seat.

Step 1:

After uninstalling your toilet seat as mentioned in the note above, you can assemble the Swing Back Arms (B) to the Flat Base Panel (A). Make sure that the side with the grey non-slip stickers are on the underside, away from the Swing Back Arms.




Step 2:

Secure the 2 Swing Back Arms to the top of the Flat Base Panel using the 8 Small Screws (C) (four on each side). Use the Allen Key to Secure the Screws.



Step 3:

Unscrew the Long Screw (E) from each of the Locking Inserts (D). Set the Long Screw and the Washers (F) aside for the moment.

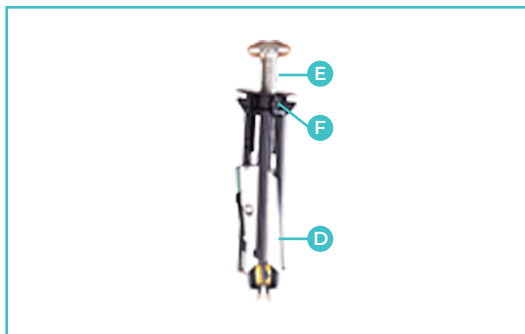

Step 4:

Insert the Locking Inserts into the two back holes in the toilet.



Step 5:

Place the Toilet Rail onto the toilet which you removed the toilet seat from. Line the holes on the Base up with the holes on the toilet for the toilet seat placement. Then place the toilet seat over it and line it up along the same holes.



Step 6:

Insert the Long Screw (E) through a Washer (F) and then Insert it into the Locking Insert (D) that was inserted into the toilet (Threading it through both the Base and the Toilet Seat Tabs on the back of the Toilet Seat).



Step 7:

Tighten down the screws on each side. **NOTE:** It is going to feel like you are not tightening it. That's normal. It is going to take roughly **60 full rotations** on each screw to tighten it all the way.



NOTE:

You can check your progress on the sides, as you turn the screw the gold square piece in the Locking Insert will slowly move upward and it will push up the silver piece which locks everything in place. Once the silver piece on each side sits up horizontally along the underside of the toilet you are done with installation.

Unlocked:**Locked:**

CAUTION: The Toilet Rail is meant to be used as a safety frame and stability assist only, it is NOT designed or intended to bear an individual's full body weighing. Periodically check all of the screws and fasteners to make sure that the rail is securely attached, retighten if necessary. Failure to use the Safety Rail as intended may result in damage or personal injury.

Weight Capacity: 250lbs each arm. 450lbs total weight capacity.

Warning: Toilet rail is meant to be used with both arms and can totally support a 450lbs individual. It is not recommended to use the arms singularly and the weight capacity lowers to 250lbs. Using in this manner also can greatly stress the base of the toilet.



HAVE MORE QUESTIONS?

Check out our list of Frequently Asked Questions at
[vhealth.link/lky](https://vivehealth.com/faq) for helpful answers.



And if that doesn't answer your question, our
customer service team would love to help!
Feel free to connect with them by phone,
e-mail, or chat on our website

service@vivehealth.com
1-800-487-3808
vivehealth.com

Distributed by

vive
health

8955 Fontana Del Sol Way
Naples, FL 34109
Made in China