

ROLLATOR WITH CUP HOLDER TRAY

MOB1119

vive®
MOBILITY



To see all FAQs in
one place, visit
vivehealth.link/64b

vivehealth.com

ROLLATOR WITH CUP HOLDER TRAY

WHAT'S INCLUDED



- (A) 1x Rollator Frame w/ Pre-attached Bag and Tray
- (B) 2x Front Legs
- (C) 2x Rear Legs attached to the Upright Handle (Pre-installed) (Attached by the brake line)
- (D) Hex Head Bolt
- (E) Locking Knob

ASSEMBLY INSTRUCTIONS

Want to watch us do this instead? Just go to the following link to see how to assemble and set up your Three Wheel Rollator with Seat.



To see all FAQs in one place,
visit **vhealth.link/wh1**

Step 1

Unfold the Rollator Frame by pulling down on the rear folding bar.



Step 2

Install the Front Wheels (B) to the front of the frame. You will see the tubing for this part is more rounded compared to the oval back legs.



Then insert each leg into the rear of the frame. If done correctly the wheel will be on the outside of the frame. Be sure to keep the brake lines from getting twisted so they can nicely follow the frame up on each side to the brake lever.




Step 3

To adjust the height handle of your rollator first, uninstall the pre-installed screw and locking knob, then pull up on the upright handles and decide a height thats best for you, makes sure the handles align with the holes on the side of the frame and then Insert the Hex Head Screw (D) into the Hex Hole from the outside of the rollator protruding inwards. Then secure it tightly with the locking knob (E). You will notice the hex head fits nearly flush into the hexagon shaped hole.



Step 4

You can add or remove the tray using the velcro on each side of the bag.

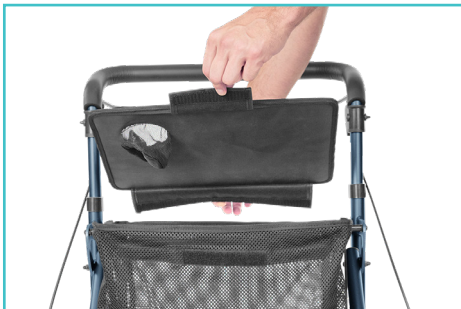

USING THE BRAKES

There are two (2) different brake functions on the Rollator: an operating brake and a parking brake.

The operating brakes are intended to help maneuverability around corners or other objects, as well as provide a somewhat firm base for leaning on when needed. These brakes are engaged and released using the brake bar underneath the top handle bar. You can also lock the brakes by pushing down on this brake bar.



MAINTENANCE

The only maintenance required for your Rollator is to keep it clean with a soft, damp cloth. Do not use any abrasive or volatile cleaners.

TROUBLESHOOTING

Occasionally, the brake cables can become loose. To tighten them, simply use a phillips head screwdriver to tighten the adjuster screw where the cable is connected to the brake handle. DO NOT attempt to use the Rollator if the brake cables are broken.

If you have any problems with the Rollator, including assembly, use, or maintenance, please contact service@vivehealth.com. Please report to vivehealth.com if any unexpected events occur.

ADDITIONAL INFORMATION AND SAFETY WARNINGS ⚠

- The Rollator has a weight limit of 300 lbs, and should not be used to support more weight than that.
- The Rollator is intended as a mobility aide and should not be used for any other purpose. Failure to follow this warning can result in damage to the equipment and/or severe injury.

- Do not attempt to use the Rollator if either of the brake cables are loose, disconnected, or broken.
- The Rollator is intended for adult use only. Children should not attempt to use this equipment.
- Do not store the Rollator outside or in areas of extreme moisture, cold, or heat as these can damage the components over time, making them unsafe to use.

WARNING: No modification of this equipment from its original specifications is allowed.

HAVE MORE QUESTIONS?

Check out our list of Frequently Asked Questions at
[vhealth.link/64b](https://vivehealth.com/64b) for helpful answers.



And if that doesn't answer your question, our
customer service team would love to help!
Feel free to connect with them by phone,
e-mail, or chat on our website

service@vivehealth.com
1-800-487-3808
vivehealth.com

Distributed by

vive
health

8955 Fontana Del Sol Way
Naples, FL 34109
Made in China